# Symphony™

with PersonalFit PLUS breast shield

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Thank you for choosing the Symphony breast pump. Breast milk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 60 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breast pumps that perfectly meet breastfeeding mothers' needs. For mothers who insist on the best for their babies and themselves. Because you care.

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AR  
231–194

# Table of Contents

<b>1. Intended purpose / population / contraindication</b>	<b>5</b>
<b>2. Meaning of symbols</b>	<b>6</b>
<b>3. Important safety information</b>	<b>8</b>
3.1 Warnings	8
<b>4. Product description</b>	<b>12</b>
<b>5. Cleaning</b>	<b>14</b>
5.1 Tubing and Symphony cap	14
5.2 Before first use / after each use	15
5.3 Before first use / once a day	16
5.4 Breast pump – Cleaning and disinfection	17
<b>6. Preparing for pumping</b>	<b>18</b>
6.1 Card and cord protector	18
6.2 Mains operation	18
6.3 Changing the program card	19
6.4 Assembling the pump set	20
<b>7. Expression</b>	<b>24</b>
7.1 Selecting the right program	24
7.2 Symphony Standard card: Operation	26
7.3 Set comfort vacuum	27
7.4 Symphony PLUS card: Operation INITIATE program	28
7.5 Symphony PLUS card: Operation MAINTAIN program	30
7.6 Expression by single pumping	32
7.7 Expression by double pumping	34
<b>8. Storing and thawing breast milk</b>	<b>36</b>
8.1 Storage	36
8.2 Freezing	36
8.3 Thawing	36
<b>9. Troubleshooting</b>	<b>37</b>
9.1 Troubleshooting	37
9.2 Error codes	38
<b>10. Warranty and maintenance / disposal</b>	<b>39</b>
<b>11. Complete care during breastfeeding time</b>	<b>41</b>
<b>12. Technical specifications</b>	<b>233</b>

# 1. Intended purpose / population / contraindication

## Intended purpose

The Symphony breast pump is a powered breast pump to be used by lactating women to express and collect milk from their breasts.

## Indications for use

The Symphony breast pump is indicated to express and collect milk from the breasts of a lactating woman. It is also indicated to initiate, build and maintain a mother's milk supply to support the provision of human milk for infants who are not able to directly or effectively breastfeed (e.g. latch-on problems such as flat or inverted nipples, cleft palate and prematurity). The breast pump removes milk from affected breast/s to alleviate the symptoms of engorgement and milk stasis as well as support the healing process of sore or cracked nipples and mastitis.

## Contraindications

There are no known contraindications for the Symphony breast pump.

## Product description

For adequate initiation and building of milk supply, the breasts should be drained

- early: within 1–3 hours after birth
- frequently: every 2–3 hours
- effectively: baby with a healthy sucking behaviour or double pumping.

It is recommended to record the date, time and volumes pumped.

The Symphony breast pump is provided with the Symphony PLUS program card, which contains two pump suction pattern programs. The goal of the INITIATE program is to initiate milk production with Medela's initiation technology. The MAINTAIN program is designed to build and maintain milk production with the 2-Phase Expression technology.

In some cases, the Symphony breast pump is provided with the Standard program card to generate the pump suction patterns. This program is designed to build and maintain milk production with 2-Phase Expression technology. While the Standard card can be used to initiate lactation, Medela recommends using the INITIATE program of the Symphony PLUS program card.

The 2-Phase Expression technology mimics the baby's natural sucking rhythm. Short, stimulating cycles in the stimulation phase are followed by longer ones in the expression phase.

## 2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to observe these instructions can lead to injury to the user or damage to the breast pump. There are several warning symbols/word combinations. These have the following meanings:

**Warning**  
Can lead to serious injury or death.

**Note**  
Can lead to material damage.

**Caution**  
Can lead to minor injury.

**Info**  
Useful or important information that is not related to safety.

This symbol indicates that an item can technically be recycled.

This symbol indicates the material from which an item is made.

This symbol indicates that the device should be kept away from sunlight.

This symbol indicates a fragile device that should be handled with care.

This symbol indicates the temperature limits for operation, transport and storage.

This symbol indicates the humidity limits for operation, transport and storage.

This symbol indicates the atmospheric pressure limits for operation, transport and storage.

This symbol indicates that the device should be kept dry.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation (EC) No. 1935/2004.

This symbol indicates unique GS1 Global Trade Item Numbers (GTIN).

### Symbols on the device and packaging

This symbol indicates that the instructions for use should be followed.

Indicates compliance with the EU requirements concerning medical devices.

This symbol indicates the manufacturer.

This symbol indicates that the device should not be disposed of with unsorted municipal waste (for EU only).

This symbol indicates compliance with additional USA and Canada safety requirements for medical electrical equipment.

This symbol indicates a type BF applied part.

This symbol indicates the manufacturer's serial number for the device.

This symbol indicates the manufacturer's catalogue number for the device.

IP21

This symbol indicates the degree of protection against ingress of foreign objects and moisture.

This symbol indicates the date of manufacture.

Indicates equipment meeting the safety requirements specified for Class II equipment according to IEC 61140.

This symbol indicates the fuse (to identify fuse boxes or their location).

This symbol indicates that the fuse box is safety tested.

This symbol indicates that the cover of the mains socket should be opened.

Indicates a medical device.

Authorized representative in the European Community / European Union.

Importer.

On/off button

Let-down button

Vacuum adjustment knob

## Symbols on the display

Vacuum level indicator, stimulation phase – Standard program card and Symphony PLUS program card (MAINTAIN program).

Vacuum level indicator, expression phase – Standard program card and Symphony PLUS program card (MAINTAIN program).

Vacuum level indicator – Symphony PLUS program card (INITIATE program).

## 3. Important safety information

8

### 3.1 Warnings

Any serious incident that has occurred in relation to the Symphony breast pump should be reported to Medela AG and the relevant Competent Authority.

Failure to follow the instructions/safety information below can lead to danger from the device.

#### READ ALL INSTRUCTIONS BEFORE USE

##### **WARNING: To reduce the risk of electric shock:**

- | Keep the device dry. Never immerse in water or other liquids!
- | Do not spray or pour liquid directly onto the pump unit.
- | Do not place or store the device where it can fall or be pulled into a sink or tub.
- | Do not use the Symphony breast pump while bathing or showering.
- | Do not reach for any electrical device if it has fallen into water. Unplug it immediately.

##### **WARNING: To avoid fire, electric shock or serious burns:**

- | The Symphony breast pump is not heat-resistant: Keep it away from radiators and naked flames.
- | Do not expose the motor unit to direct sunlight.
- | Do not operate the Symphony breast pump where aerosol (spray) products are being used or where oxygen is being administered.
- | Do not use the device in the presence of a flammable anaesthetic mixture with air, or with oxygen or nitrous oxide.
- | Do not thaw frozen breast milk and do not warm breast milk in a microwave or in a pan of boiling water. Microwaving can cause severe burns to the baby's mouth from hot spots that develop in the milk during heating. Microwaving can also change the composition of breast milk.
- | Use only the power cord that comes with the Symphony breast pump. Make sure that its rated voltage is compatible with the power source.
- | Never operate an electrical device if it has a damaged power cord, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- | Always inspect the power cord wires prior to use for damage or exposed wire. If any damage is found, stop using the power cord immediately. Contact the local Medela representative or rental agency.

- I The Symphony breast pump should never be left unattended when plugged into a power source.
- I Separation from the mains is only assured by disconnecting the power cord from the wall socket.
- I Position the pump unit such that it is easy to disconnect the power cord from the wall socket.
- I Keep the power cord away from heated surfaces.

** WARNING: To avoid health risks and reduce risk of injury:**

- I Clean all reusable parts that come in contact with the breast and breast milk prior to first use as shown in chapter 5.
- I Clean parts that come in contact with the breast and breast milk immediately after use.
- I The pump sets are personal care items intended for use by a single user only. They should not be shared between mothers.
- I Do not continue pumping for more than two consecutive pumping sessions if no results are achieved.
- I If problems or pain occur, consult a lactation specialist.
- I Never use the pump during pregnancy, unless prescribed/advised by a physician, as pumping can induce labour.
- I For mothers who are infected with Hepatitis B, Hepatitis C or Human Immuno-deficiency Virus (HIV): Pumping breast milk will not reduce or remove the risk of transmitting the virus to the baby through breast milk.
- I Inspect all components of the Symphony breast pump as well as all parts of the pump set(s) before each use.
- I Never use a device, components or parts that are damaged.
- I Never use a device, components or parts if dirt, mould or other contamination is visible.
- I Replace mouldy, damaged or worn parts.
- I There are no user-serviceable parts inside the Symphony pump unit. Repairs must be performed only by an authorised service agency. Do not repair the parts yourself! No modifications to the device are permitted.
- I Use the Symphony breast pump only for its intended purpose as described in this manual.

### 3. Important safety information

10

#### **⚠ WARNING: To avoid health risks and reduce risk of injury:**

- I Use only original Medela accessories.
- I Do not use attachments not recommended by Medela, as this may cause hazards.
- I Do not modify any parts of the pump or the pump set.
- I Do not use the Symphony breast pump while operating a motor vehicle.
- I Do not drive while using hands-free pumping.
- I Never drop or insert any object into any opening or tubing.
- I Do not use the pump outdoors.
- I Supervision is necessary when the Symphony breast pump is used in the vicinity of children.
- I Never leave packaging and components unattended. Keep them out of reach of children.
- I The Symphony breast pump is EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 3rd Edition and 60601-1-2:2014 4th Edition according to clause 7 and 8.9.
- I Radio frequency communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations as well as walkie-talkies can affect the Symphony breast pump. Due to electromagnetic compatibility (EMC), a distance of at least 30 cm between the Symphony breast pump and the wireless equipment is recommended.
- I The Symphony electric breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Symphony electric breast pump should be monitored to verify normal operation in the configuration in which it will be used.

#### **Important**

- I **Plastic bottles and component parts become brittle when frozen and may break when dropped.**
- I **Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.**
- I **Take appropriate care in handling bottles and components.**
- I **Do not use the breast milk if bottles or components become damaged.**

Failure to follow the instructions/safety information given below can lead to minor injury.

**⚠ CAUTION: To avoid risk of contamination:**

- I Wash your hands thoroughly with soap and water before touching the breast pump, the pump set(s) and breasts. Avoid touching the inside of containers or lids.
- I Only use drinking-quality tap or bottled water for cleaning.
- I Do not store wet or damp parts, as mould may develop.

**⚠ CAUTION: To avoid health risks and reduce risk of injury:**

- I Pump only with the breast pump in an upright position.
- I Do not use the Symphony breast pump while sleeping or overly drowsy.
- I Pressing shields too firmly onto the breasts may affect the milk flow.
- I Do not express with a vacuum that is too high and uncomfortable (painful). The pain, along with potential breast and nipple trauma, may decrease the milk output.
- I Do not attempt to remove the breast shield from the breast while pumping. Turn the breast pump off and break the seal between the breast and the breast shield with a finger, then remove the breast shield from the breast.
- I If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with a finger and remove the breast shield from the breast.
- I Do not shorten the tubing between the breast shield and the pump.
- I Do not use abrasive washing-up liquids/detergents when cleaning the breast pump, pump components or pump set parts.
- I Never put the breast pump unit in water, as this can cause permanent damage to the breast pump.
- I Do not use Medela Symphony pump sets with a non-compatible breast pump.
- I Do not use the Medela Symphony breast pump with non-compatible pump sets.
- I Do not use the pump sets for any other purpose than expressing breast milk.

**⚠ CAUTION: To reduce risk of electrocution:**

- I Plug the power cord into the breast pump first and then into the wall socket.
- I Unplug the breast pump before cleaning it.

## 4. Product description

12

### Symphony single or double pump set (applied parts)\*

The Symphony single or double pump set is an accessory for the Symphony breast pump. The pump set must be used by only one mother; use by more than one mother could pose a health risk. The single pump set contains one piece of each part, while the double pump set contains two.

The pump set contains the PersonalFit PLUS breast shield, size S (21 mm), M (24 mm) or L (27 mm). For other breast shield sizes, see chapter 11. For more details about the correct breast shield size, see the related information inside the pump set box.

**PersonalFit PLUS  
breast shield**

**connector and  
connector lid**


**milk bottle, printed**

**multi lid**


**membrane**

**tubing**

**Symphony cap**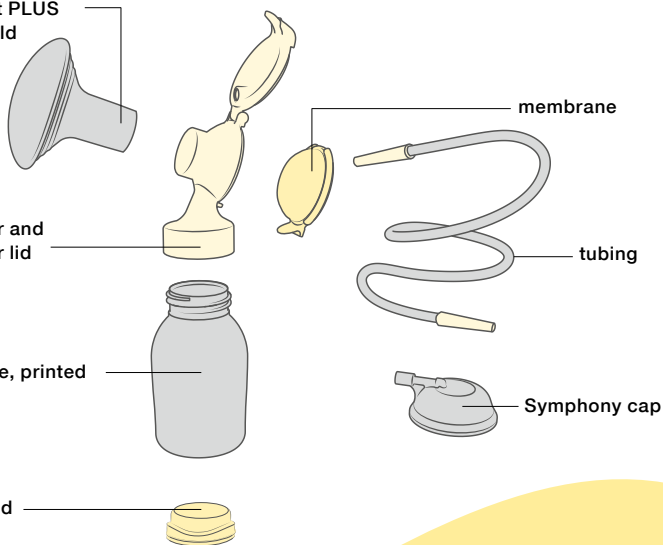

## Rental pump required: Symphony breast pump

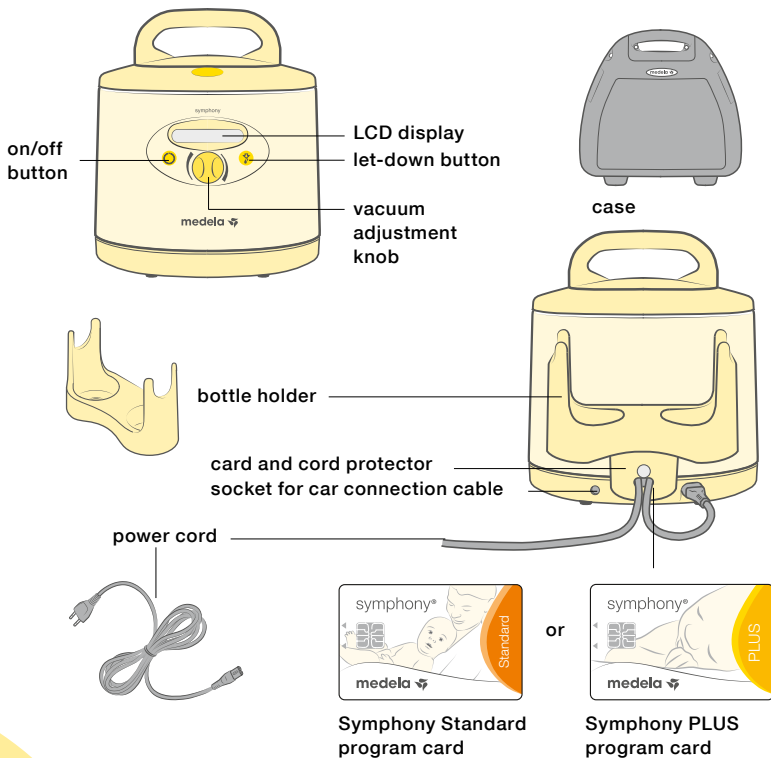\* Accessories available for order can be found in chapter 11. Articles may not be available in all countries. For further information about Medela products, visit [www.medela.com](http://www.medela.com) and look for location finders.

## 5. Cleaning

14

### Caution

- I Only use drinking-quality water for cleaning.
- I Take apart and wash all parts that come in contact with the breast and breast milk immediately after use to avoid drying up of milk residues and prevent growth of bacteria.


### Note

- I Before using the pump set for the first time, you must clean it thoroughly. Perform the steps described in chapter 5.2. and chapter 5.3.
- I The pump set requires no maintenance.
- I Take care not to damage parts of the pump set during cleaning.
- I If the individual parts of the pump set are cleaned in a dishwasher, they may be discoloured by food pigments. This has no impact on their function.
- I The membrane prevents milk from flowing into the tubing, however inspect the tubing and the Symphony cap after each pumping session for visible residues. If there are visible residues, follow the instructions in chapter 5.1.


### 5.1 Tubing and Symphony cap

1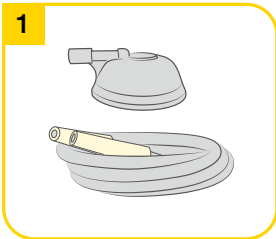

- I **No visible residues**  
No cleaning needed.
- I **With visible residues**  
Clean the tubing and the cap as described in chapter 5.2, step 2a. Then disinfect by boiling them in water for at least 5 minutes. Dry the cap as described in chapter 5.2, step 3. For the tubing, shake out the water drops and let the pump run with the tubing attached until the tubing is dry.

## 5.2 Before first use/after each use

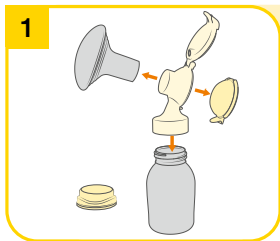
Disassemble the pump set into its individual parts:

- bottle
- multi lid
- breast shield
- connector and connector lid
- membrane

Check all parts for damage.

Replace damaged or worn parts.

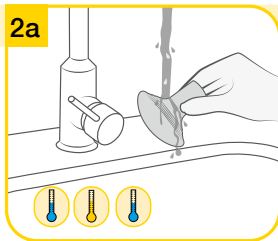- I Rinse all parts with cold, clear drinking-quality water (approx. 20 °C).
- I Clean these parts with plenty of warm, soapy water (approx. 30 °C). Use a commercially available washing-up liquid, preferably without artificial fragrances and colouring (pH neutral).
- I Rinse the parts with cold, clear drinking-quality water (approx. 20 °C).

### **2b** Dishwasher

**As an alternative to the previous step 2a,** you can also clean the pump set in the dishwasher. Place all parts on the top rack or in the cutlery section. Use a commercially available dishwashing detergent.

### **3** Drying / storage

Dry with a clean cloth or leave to dry on a clean cloth.  
Put the clean parts in a clean storage bag or a clean environment. Do not store parts in an airtight container/bag. It is important that all residual moisture dries.

## 5. Cleaning

16

### Caution

- I Do not spray or pour liquid directly onto the pump.
- I Only use drinking-quality water for cleaning.
- I Take apart and wash all parts that come in contact with the breast and breast milk immediately after use to avoid drying up of milk residues and prevent growth of bacteria.
- I **For chapter 5.4 Breast pump – Cleaning and disinfection:**  
Unplug the breast pump before cleaning.


### Note

- I **For chapter 5.3, step 2:**  
When boiling, a teaspoon of citric acid may be added to the water to avoid a build-up of limescale.

### 5.3 Before first use / once a day

1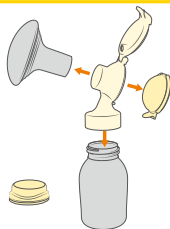

Disassemble the pump set into its individual parts:


- bottle
- multi lid
- breast shield
- connector and connector lid
- membrane

Check all parts for damage.

Replace damaged or worn parts.

2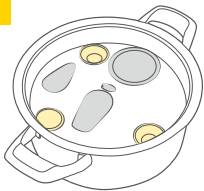

Cover all parts with water and boil at least for 5 minutes.

3

Drying/storage

Dry with a clean cloth or leave to dry on a clean cloth.  
Put the clean parts in a clean storage bag or a clean environment. Do not store parts in an airtight container/bag. It is important that all residual moisture dries.

## 5.4 Breast pump – Cleaning and disinfection


Clean and disinfect the pump as needed.

1

To **clean** the breast pump, wipe the housing with a **clean**, damp (not wet) cloth. Use drinking-quality water and basic washing-up liquid or a non-abrasive detergent.

2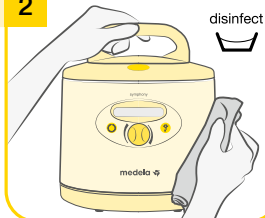

To **disinfect** the breast pump, wipe the pump housing with a **clean**, damp (not wet) cloth. You may use an alcohol-based disinfecting agent.

## 6. Preparing for pumping

18

### Warning

#### For chapter 6.2 Mains operation:

- I Use only the power cord that comes with the Symphony breast pump.
- I Make sure the voltage of the power cord is compatible with the power source.

### Info


- I Use the pump only with the appropriate program card (Symphony Standard or Symphony PLUS).

#### Optional for pump with battery:

- I The built-in battery must be charged for 12 hours on receipt of the device.
- I The pump can be used during charging.
- I If the pump is to remain unused for a long time, charge the battery every two months (for 12 hours each time).
- I The optimum service life of the battery is ensured if it is not fully discharged on a regular basis or if the pump is charged while not in use.


Pumping time (fully-charged battery)	approx. 60 min.
Charging time (for full charge)	12 hours

### 6.1 Card and cord protector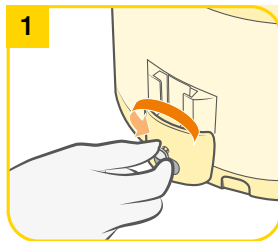


To remove, unscrew the card and cord protector and slide it out of the guide rail.

### 6.2 Mains operation

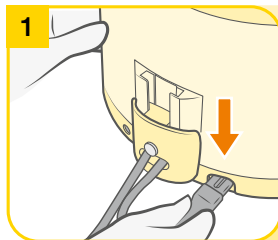Insert the power cord into the power socket on the back of the pump.


2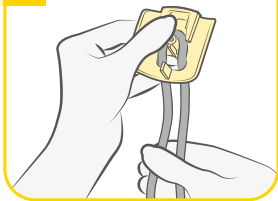

Insert the cord into the card and cord protector in a loop around the pin.


→ Make sure there is enough play to plug the cord in.

3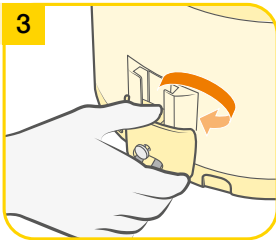


Push the card and cord protector over the guide rail on the back until it stops. Tighten the screw.

2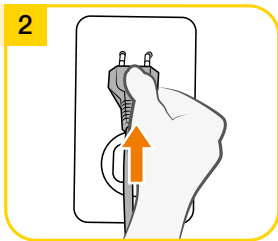

Plug the power cord into the mains socket.

1

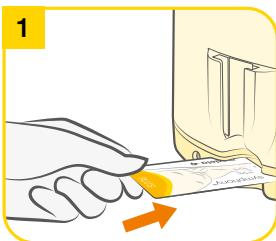As soon as the card compartment is revealed (step 1 in chapter 6.1), the card can be removed and replaced or changed.

### 6.3 Changing the program card

## 6. Preparing for pumping

20

### Caution

- Wash hands thoroughly (at least one minute) with soap and water before touching the breast, the clean pump parts and clean pump set. Dry your hands with a fresh towel or a single-use paper towel.

### Note

- Use Medela original accessories only.
- Check pump set parts for wear or damage before use. Replace damaged or worn parts.
- All parts must be completely dry before use.


### Info

- Carry out all the steps with care and assemble the pump set correctly. Otherwise a good vacuum may not be achieved.


### 6.4 Assembling the pump set

1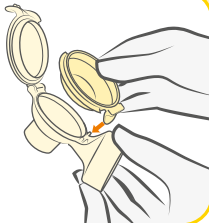


Carefully place the membrane into the connector with the flap into the opening of the connector. Make sure that the membrane forms a seal around the edge of the connector.

4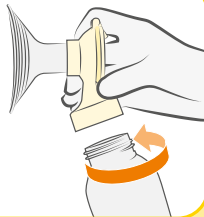


Screw the connector onto the bottle. Choose a bottle size that suits your needs.

2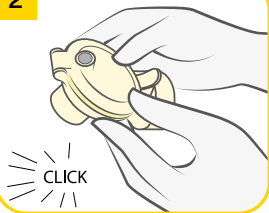

Close the connector lid until it audibly clicks.

3

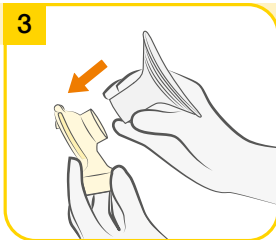
Carefully push the breast shield onto the connector. Choose a breast shield size that suits your needs.

5

Insert the tubing adapter into the corresponding opening in the connector lid.

6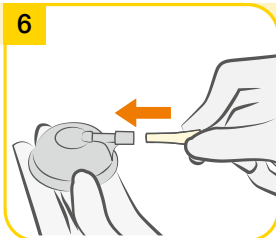


Insert the tubing adapter into the opening in the Symphony cap.

## 6. Preparing for pumping


22

### i Info


- Always close the lid when pumping. The lid keeps the Symphony cap pushed down to prevent a loss of vacuum that could occur if the Symphony cap came off.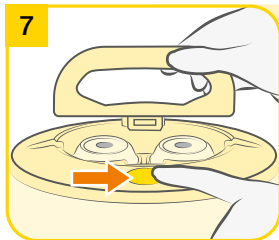


7  
Open the lid of the breast pump:  
Press the oval button on the top of the Symphony pump and lift the handle.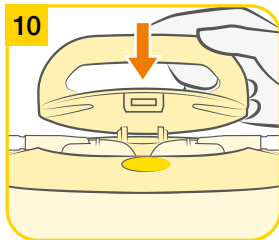

10  
Close the lid.

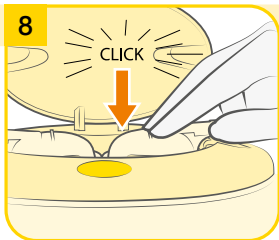Position the Symphony cap firmly on the breast pump membrane to form a seal.  
→ Make sure that the Symphony cap engages and lies flush to the surface.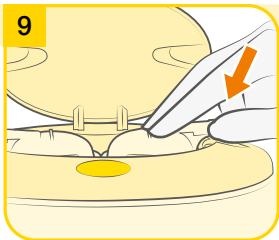

Centre the tubing connector in the recess in the pump housing.

## 7. Expression

### 24 7.1 Selecting the right program

The Symphony breast pump is installed with either the Symphony PLUS program card or Symphony Standard program card which generate the pump suction patterns.


The Symphony PLUS program card contains two pump programs. The aim of the INITIATE program is to initiate milk production with Medela's initiation technology. The MAINTAIN program is designed to build and maintain milk production with the 2-Phase Expression technology.

The Standard program card is designed to build-up and maintain milk production with the 2-Phase Expression technology. This program is identical to the MAINTAIN program. While the Standard card can be used to initiate lactation, Medela recommends using the INITIATE program of the Symphony PLUS program card.

Contact your midwife or breastfeeding and lactation consultant if you are experiencing difficulties in initiating lactation or for further information on which program to use.

#### Which program card is installed?

To determine which program card is installed remove the program card according to chapter 6.3. Then reinsert the card.

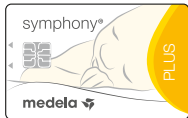**Symphony PLUS  
program card**

or

**Symphony Standard  
program card**

### When the Symphony Standard card is installed:

You will not need to select a pumping program. The card contains the 2-Phase Expression technology and will start with the stimulation phase. See chapter 7.2 for further information.

### When the Symphony PLUS card is installed:

Use the INITIATE program directly after birth until the mother has reached secretory activation:

- I **For fully pump-dependent mothers:** This can be indicated by pumping at least 20 ml in total in each of her last three pumping sessions. If not all pumping volumes have been recorded, this can be indicated by other physiological markers of secretory activation, such as breast fullness.
- I **When the mother is breastfeeding and pumping:** This can be indicated by other physiological markers of secretory activation, such as breast fullness.

OR

- I Throughout the first five days after birth.

If milk production has not started after five days, change to the MAINTAIN program. Contact your midwife or breastfeeding and lactation consultant if you are unsure if your milk has come in and which program you should use. See chapter 7.4 for further information.

Use the MAINTAIN program after milk coming in:

- I Once you have pumped at least 20 ml in total in each of your last three pumping sessions, OR
  - I Beginning on day six after birth, whichever of the two options comes first.
- See chapter 7.5 for further information.

## 7. Expression

26

### Note


- Always disconnect the Symphony from the power source after expression.

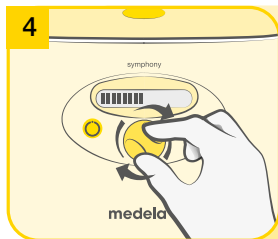
### Info

- The program stores the last vacuum setting in the stimulation phase.
- For the Standard program:** Press the “Let-down” button  to revert from the expression phase to the stimulation phase if necessary. If milk has not begun to flow after two successive stimulation phases, you should take a break and try again in 15–30 minutes. Also, you could try breast massage and then pump using the expression phase.
- If the pump has been running for 30 minutes without any manipulations (e.g. adjustment of the vacuum), the Symphony will automatically switch off.


## 7.2 Symphony Standard card: Operation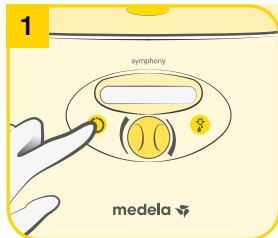


Press the On/off button  to switch on the breast pump. The Standard program will automatically start with the stimulation phase after pressing the On/off button.

Set the comfort vacuum for the expression phase (see chapter 7.3). The bars on the display show the vacuum level. The more bars, the stronger the vacuum.

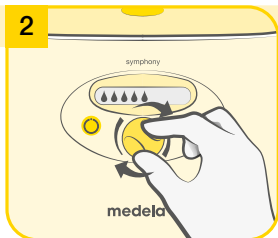
Set the comfort vacuum for the stimulation phase (see chapter 7.3). The drops on the display show the vacuum level. The more drops, the stronger the vacuum.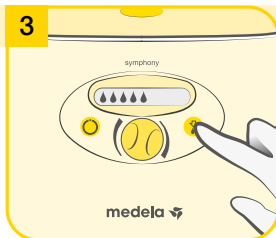


The pump will automatically switch to the expression phase after two minutes.  
 → If the milk starts to flow sooner, press the Let-down button  to change to the expression phase.

### 7.3 Set comfort vacuum

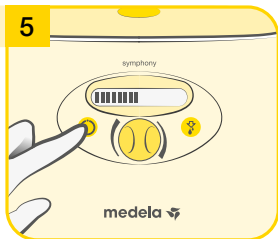Continue pumping until the breast feels well drained and soft all over (Medela recommends a pumping time of at least 15 minutes). Switch off the pump by pressing the On/off button .

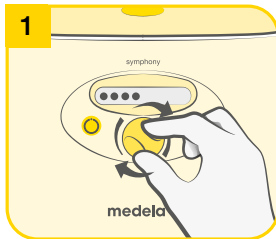Set the comfort vacuum by rotating the vacuum adjustment knob. First rotate to the right to increase the vacuum until you feel a slight discomfort. Then rotate to the left to reduce the vacuum until it becomes comfortable.

## 7. Expression

28

### Note


- I Use the INITIATE program directly after birth:
  - Until you have pumped at least 20 ml in total in each of your last three pumping sessions, OR
  - Throughout the first five days. If milk production has not started after five days, change to the MAINTAIN program (see chapter 7.5).
- I Always disconnect the Symphony from the power source after expression.


### Info

- I The INITIATE program runs automatically for 15 minutes. It is important to complete the entire INITIATE program. The pump switches off automatically.
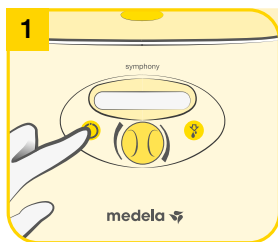
### 7.4 Symphony PLUS card: Operation INITIATE program


Press the On/off button  to switch on the breast pump. The display text shows "INITIATE press .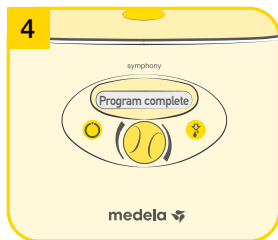


The INITIATE program runs automatically for 15 minutes with stimulation, expression and pause phases. An acoustic signal indicates the end of the program. The display shows "Program complete". The pump switches off automatically.

→ It is important to complete the entire INITIATE program.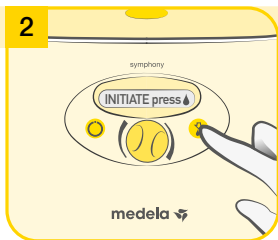

Press the “Let-down” button  within 10 seconds. The display text will change to “INITIATE running”.

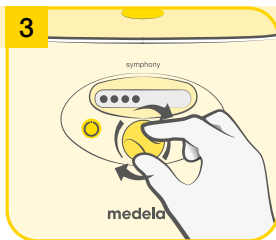Set the comfort vacuum (see chapter 7.3). The bullets on the display show the vacuum level. The more bullets, the stronger the vacuum.

## 7. Expression

30

### Note


- I Use the MAINTAIN program after milk coming in:
  - Once you have pumped at least 20 ml in total in each of your last three pumping sessions, OR
  - Beginning on day six, whichever of the two options comes first.
- I Always disconnect the Symphony from the power source after expression.

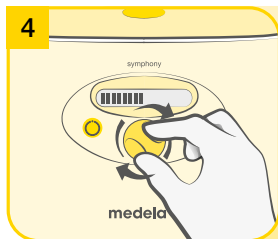
### Info

- I The program stores the last vacuum setting in the stimulation phase.
- I **For the MAINTAIN program:**
  - Press the “Let-down” button  to revert from the expression phase to the stimulation phase if necessary. If milk has not begun to flow after two successive stimulation phases, you should take a break and try again in 15–30 minutes. Also, you could try breast massage and then pump using the expression phase.
- I If the pump has been running for 30 minutes without any manipulations (e.g. adjustment of the vacuum), the Symphony will automatically switch off.


### 7.5 Symphony PLUS card: Operation MAINTAIN program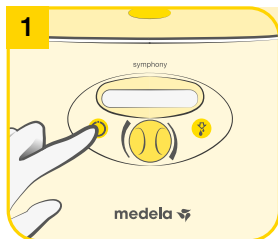


Press the On/off button  to switch on the breast pump.  
The MAINTAIN program will automatically start with the stimulation phase after pressing the On/off button.

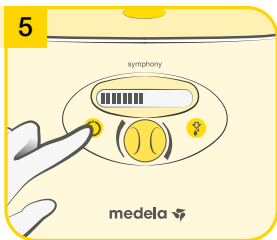
Set the comfort vacuum for the expression phase (see chapter 7.3). The bars on the display show the vacuum level. The more bars, the stronger the vacuum.

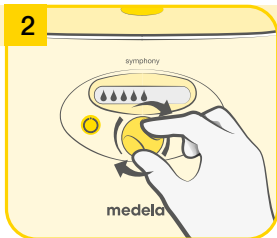Set the comfort vacuum for the stimulation phase by rotating the vacuum adjustment knob (see chapter 7.3). The drops on the display show the vacuum level. The more drops, the stronger the vacuum.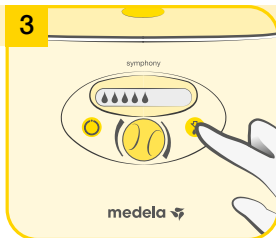

The pump will automatically switch to the expression phase after two minutes.  
 → If the milk starts to flow sooner, press the Let-down button  to change to the expression phase.

Continue pumping until the breast feels well drained and soft all over (Medela recommends a pumping time of at least 15 minutes). Switch off the pump by pressing the On/off button .

## 7. Expression

32

### Caution

- I Wash hands thoroughly (at least one minute) with soap and water before touching the breast, the clean pump and the clean pump set parts. Dry your hands with a fresh towel or a single-use paper towel.
- I Wipe the breast with a warm cloth (do not use alcohol) before pumping.
- I Do not hold the pump set by the bottle during pumping. This can lead to blockage of the milk ducts and engorgement.
- I Contact the midwife or breastfeeding and lactation consultant if only minimal or no milk can be expressed or if expression is painful.

### Note


- I Always disconnect the Symphony from the power source after expression.


### Info

- I Do not kink the tubing during expression.

### 7.6 Expression by single pumping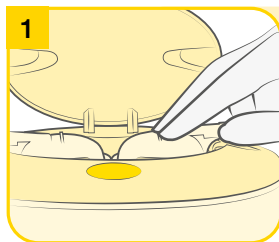


Prepare one bottle and pump set for expression. Mount only one Symphony cap on the breast pump (see chapter 6.4).

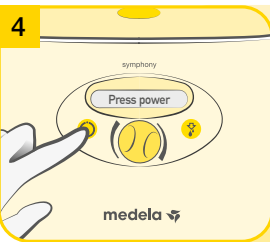Switch on the breast pump with . Proceed with the program selection (see chapter 7.1). Operate the pump as is appropriate for the selected program (see chapters 7.2, 7.4 and 7.5).


2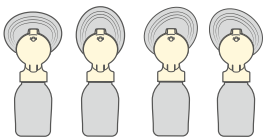


The PersonalFit PLUS breast shield is oval and can be rotated (360°) and placed in the desired position for the most comfortable fit for you.

3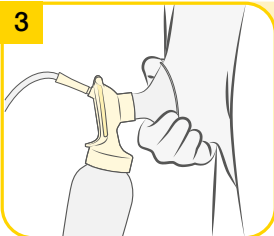

Place the breast shield on the breast so that the nipple is properly **centred** in the tunnel. Hold the breast shield onto the breast with the thumb and index finger. Support the breast with the palm of the hand.

5

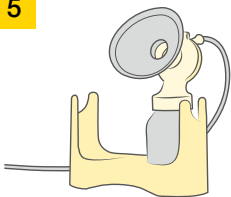When expression is finished, place the bottle in the stand or bottle holder in order to prevent the bottle from tipping over.

6

Clean in accordance with chapter 5.

Close the bottle with the lid. To store the breast milk, follow the instructions in chapter 8.

## 7. Expression

34

### Caution

- I Wash hands thoroughly (at least one minute) with soap and water before touching the breast, the clean pump and the clean pump set parts. Dry your hands with a fresh towel or a single-use paper towel.
- I Wipe the breast with a warm cloth (do not use alcohol) before pumping.
- I Do not hold the pump set by the bottle during pumping. This can lead to blockage of the milk ducts and engorgement.
- I Contact the midwife or breastfeeding and lactation consultant if only minimal or no milk can be expressed or if expression is painful.

### Note


- I Always disconnect the Symphony from the power source after expression.


### Info

- I **Double pumping** saves time and increases the energy content of the milk. Milk production may be increased and maintained over the long term. Medela recommends double pumping.
- I Do not kink the tubing during expression.

### 7.7 Expression by double pumping

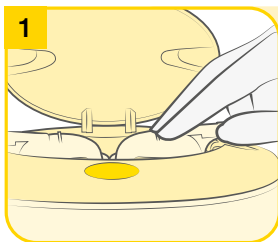
Prepare two bottles and pump sets for expression.  
Mount the Symphony caps on the breast pump (see chapter 6.4).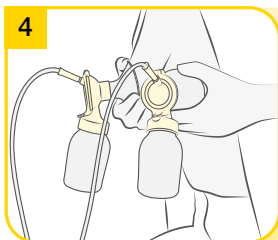


Place the breast shield of the second bottle on the other breast so that the nipple is properly **centred** in the tunnel. Operate the pump as appropriate for the selected program (see chapters 7.2, 7.4 and 7.5).

2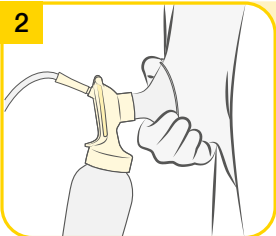


Place the breast shield of one bottle on the first breast so that the nipple is properly **centred** in the tunnel. The PersonalFit PLUS breast shield is oval and can be rotated (360°) and placed in the desired position for the most comfortable fit for you. To learn how to hold the breast shield correctly see chapter 7.6, step 3.

3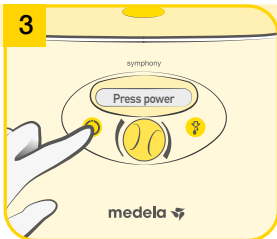

Switch on the breast pump with . Proceed with the program selection (see chapter 7.1).

5

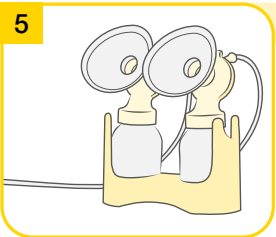When expression is finished, place the bottles in the stand or bottle holder in order to prevent the bottles from tipping over.

6

Clean in accordance with chapter 5.

Close the bottles with the lid. To store the breast milk, follow the instructions in chapter 8.

## 8. Storing and thawing breast milk

### 8.1 Storage

36

#### Storage guidelines for freshly expressed breast milk

	Room temperature 16 to 26 °C (60 to 78 °F)	Refrigerator 4 °C (39 °F) or lower	Freezer -18 to -20 °C (0 to -4 °F)	Thawed breast milk
Healthy term baby	≤ 6 hours	≤ 5 days	≤ 6 months at -18 °C (0 °F) ≤ 12 months at -20 °C (-4 °F)	Thawed to room temperature: use within max. 4h Thawed to fridge temperature: use within 24h
NICU baby	≤ 4 hours	≤ 4 days	Optimal: ≤ 3 months	Do not refreeze!

- I These guidelines for storing and thawing of breast milk are a recommendation. Contact your midwife or breastfeeding and lactation consultant for further information.
- I Do not store breast milk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).

### 8.2 Freezing

- I You can freeze expressed breast milk in milk bottles or “Breast milk storage bags”. Do not fill the bottles or bags more than  $\frac{3}{4}$  full to allow space for possible expansion.
- I Label the bottles or “Breast milk storage bags” with the date of expression and volume of milk expressed.

### 8.3 Thawing

#### Warning

Do not thaw frozen breast milk or do not warm breast milk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

- I In order to preserve the components of the breast milk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or “Breast milk storage bag” under warm water (max. 37 °C / 98.6 °F).
- I Gently swirl the bottle or “Breast milk storage bag” to mix any fat that has separated. Avoid shaking or stirring the milk.

## 9. Troubleshooting

### 9.1 Troubleshooting

Problem	Solution
<b>If the motor is not running</b>	<p>Check whether mains power/battery is available.</p> <p>Check that the device is switched on.</p> <p>Check whether the program card is inserted correctly in the card compartment on the back of the breast pump.</p>
<b>If there is low or no suction</b>	<p>Assemble the pump set in accordance with the instructions in chapter 6.4.</p> <p>Check that all connections of the pump set and breast pump are secure.</p> <p>Express in accordance with chapter 7. Make sure that the breast shield forms a complete seal around the breast.</p> <p>See chapter 6.4 „Assembling the pump set“.</p> <ul style="list-style-type: none"><li>  The ends of the tubing must be firmly connected to the connector lid and the opening in the Symphony cap.</li><li>  The Symphony cap must be firmly fitted to the breast pump membrane to form a seal.</li><li>  The membrane must be intact.</li><li>  The membrane must be correctly placed into the connector and the connector lid properly closed.</li><li>  Do not kink or trap the tubing during expression.</li><li>  All components must be completely dry.</li></ul>

# 9. Troubleshooting

## 9.2 Error codes

### Error codes

The following describes error codes that may be displayed indicating a problem with the unit, potential causes, and steps to correct the problem.

	Potential problems	Actions
Invalid card	Program card is not a Symphony card	– Check if the correct program card is being used
	Program card is incorrectly inserted	– Check that card is positioned correctly
	Program card is damaged and not working	– Replace program card
No card	Program card is not inserted	– Insert program card
	Program card is not completely inserted	– Push card in all the way
Battery low	Battery needs charging soon, and will beep every 20 seconds (15 minutes of pumping time remaining from when the indication started)	– Be sure power cord is plugged into the back of the pump
Motor overload	Motor is drawing too much current	– Contact the rental station if this happens repeatedly
Error	Potential fault in electronics of gear control	– Contact the rental station if this happens repeatedly
Power fault	Potential fault in electronics of power supply	– Contact the rental station if this happens repeatedly

## 10. Warranty and maintenance / disposal

### Warranty and maintenance

Three-year warranty for the unit from the date of purchase, excluding pump set and accessories. Six month warranty for the batteries. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorised persons. Routine checks, servicing and battery exchange may only be carried out by Medela authorised personnel.

The engineering of the Medela Symphony breast pump has been verified by independent test institutes for compliance with the EN/IEC 60601-1 standard. Copies of such certificates may be obtained on request. Because of the method of construction used, Medela does not expect electrical safety to be affected at any time during the life of the product – provided that it is used properly in accordance with its intended purpose as well as these instructions for use, and that the device is serviced and repaired at an authorised service centre. There are no prescribed or recommended electrical safety checks or recurrent tests to be carried out.

Manufacturer's information provided in accordance with EN/IEC 62353:2014 "Medical electrical equipment – Recurrent test and test after repair of medical electrical equipment".

Besides the cleaning procedure(s) described in chapter 5 and battery charging for pumps with optional batteries as described in chapter 6, no further maintenance is required.

Besides the inspection procedures described in chapter 3, e.g. visual signs of damage to the pump or power cord, pump dropped or dropped into water, no further inspections are required. As pointed out in chapter 3, service and repair may be performed only by an authorised service agency (centre) in full accordance with the Symphony Service Manual using original Medela spare parts mentioned therein. The service manual, wiring diagrams, and descriptions are available upon request from Medela.

Medela deems that the final production testing replaces on-site testing of the device before putting into service.

The operating life of the Symphony breast pump is 3000 hours. The service life is 7 years if used normally, including the life of the internal batteries. The service life of the accessories is two years (see chapter 11).

## 10. Warranty and maintenance / disposal

40

### Disposal

The Symphony breast pump is made of metals and plastics and should be disposed of in accordance with European directives 2011/65/EU and 2012/19/EU. Additional local guidelines must also be observed. For the AC/DC versions, the electronic components and the rechargeable battery must be disposed of separately, in accordance with the local regulations. Please ensure that you dispose of the Symphony and its accessories in accordance with your local disposal guidelines. The accessories (pump set/bottles) are made of plastic and may be disposed of only in accordance with the local regulations.

### **User information for the disposal of electrical and electronic equipment**

This symbol means that the electrical and electronic equipment must not be disposed of together with unsorted municipal waste. Correct disposal of this device protects and prevents possible harm to the environment or human health.

For more information about disposal, contact the manufacturer, your local caregiver or healthcare provider. This symbol is only valid in the European Union. Please respect the relevant state laws and rules in your country for the disposal of electrical and electronic equipment.

## 11. Complete care during breastfeeding time

**Depending on your region, check local website/shops for availability.**

PersonalFit PLUS Breast shield S (21 mm)\*  
PersonalFit PLUS Breast shield M (24 mm)\*  
PersonalFit PLUS Breast shield L (27 mm)\*  
PersonalFit PLUS Breast shield XL (30 mm)\*  
PersonalFit PLUS Breast shield XXL (36 mm)\*

150 ml breast milk bottle (pack of 3)  
250 ml breast milk bottle (pack of 2)

Articles may not be available in all countries. For more information about all Medela products, contact your local dealer or see [www.medela.com](http://www.medela.com).

\* If the result of expression is unsatisfactory or expression is painful, contact your lactation consultant or breastfeeding specialist. A different size of PersonalFit PLUS breast shield may help make expression more comfortable and successful.

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