# Symphony® Breast Pump Set (Kit) Care

## Cleaning - Drying - Storing

Good hygiene will help protect the breast milk that you give to your baby. For the best protection when handling your breast milk, wash your hands with soap and water, and follow the guidelines below for cleaning, drying and storing your pump set.

*If your baby is premature or ill, consult with your healthcare provider for further cleaning and storage instructions.*

*See Medela's Instructions for Use for detailed breast pump set assembly, cleaning & pumping information.*

### How do I clean my breast pump set?

#### 1. Finish pumping.

- While the pump is still running, disconnect tubing from breast shields.\*

- Continue to let pump run for 2 minutes with tubing attached. This will eliminate condensation that naturally results from humidity.\*

#### 2. Take apart pump set and store breast milk.

- With tubing still connected to pump, separate all parts of your pump set that come in contact with your breast or breast milk.

**A.** Remove bottles from breast shields and store breast milk.

**B.** Twist and pull the yellow valves off the breast shields.

**C.** Remove the white membranes from the valves.

**D.** Separate breast shields if using the PersonalFit™ breast shields that have two pieces.

\*Not recommended for some Personal Use Breast Pumps. Refer to your breast pump's instructions for use.

## How do I clean my breast pump set?

### 3. Washing.

- Rinse all separated parts in cool water to remove breast milk
- Wash and Rinse
  - Soak all separated parts in warm, soapy water for 5 minutes.
  - Wash each part with clean dish-cloth or soft brush.
  - Rinse all separated parts with clear water.
- Once rinsed, parts that have touched the breast or breast milk should be placed in a clean wash basin, dedicated to only cleaning these items. Do not place parts directly in the sink.

- OR**
- Dishwasher
    - Wash all separated parts on top rack of dishwasher.

If sanitizing your pump set, separate all parts that come in contact with your breast or breast milk and boil for 10 minutes or follow the instructions on Medela's Quick Clean™ Micro-Steam™ bag.

### Quicker Cleaning

You can use Medela's Quick Clean wipes in between pumping sessions and then clean your pump set once you get home. Follow instructions on Quick Clean package.

### Drying & Storing

Allow all parts to air dry in a clean area and store dry parts when not in use. Do **NOT** store wet or damp parts.

### When do I care for my tubing?\*

If your tubing comes in contact with breast milk, clean it by following these directions.

- Rinse tubing in cool water to remove breast milk.
- Wash tubing in warm soapy water.
- Rinse tubing with clear water.
- Hang to air dry.
- Or shake out any drops of water from the tubing and let the pump run with tubing attached until dry. This will eliminate condensation that naturally results from washing.\*

\*Not recommended for some Personal Use Breast Pumps. Refer to your breast pump's instructions for use.

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