## Disassembly:

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## References/Referencias/Références

- <sup>1</sup>IEC 60601-1, Medical electrical equipment – Part 1: General Requirements for basic safety and essential performance, Table D.2 Symbol 2 General warning sign
- <sup>2</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.4.4 Caution
- <sup>3</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.1.1 Manufacturer
- <sup>4</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.1.3 Manufacturing Date
- <sup>5</sup>Not derived from a standard | No derivado de un estándar | Non dérivé d'une norme
- <sup>6</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.1.6 Catalogue number
- <sup>7</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.1.5 Batch code
- <sup>8</sup>Not derived from a standard | No derivado de un estándar | Non dérivé d'une norme
- <sup>9</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.2.3 Sterilized using ethylene oxide
- <sup>10</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.2.8 Do not use if package is damaged and consult instructions for use
- <sup>11</sup>ISO 7000 – Graphical symbols for use on equipment – Registered symbols, Reference 3079 Open here
- <sup>12</sup>ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied, Part 1: General requirements, Clause 5.4.3 Consult instructions for use
- <sup>13</sup>Not derived from a standard | Not derived from a standard | Non dérivé d'une norme

Medela LLC  
1101 Corporate Drive  
McHenry, IL 60050, USA  
+1 (800) 435-8316 or | o al | ou le +1 (815) 363-1166  
customer.service@medela.com  
[www.medela.us](https://www.medela.us)

Distributed by | Distribuido por | Distribuído por :  
Medela Canada, Inc.  
4160 Sladerview Crescent, # 8  
Mississauga (Ontario), L5L 0A1 Canada  
+1 (800) 435-8316 or | o al | ou le +1 (905) 608-7272  
info@medela.ca  
[www.medela.ca](https://www.medela.ca)

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<sup>†</sup>Jones, F. Best Practice for Expressing, Storing, and Handling Human Milk in Hospitals, Homes, and Child Care Settings. 4th ed Fort Worth, TX: Human Milk Banking Association of North America, Inc. (2019), pg 59

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### Product Description | Descripción del producto | Description du produit :

The sterile, 24-hour use 6 mL Colostrum Protector is a container with lid. The Colostrum Protector is designed to fit seamlessly between the Symphony® breast pump connector and breast shield and is for use with the Symphony breast pump.

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#### **WARNING**

Can lead to serious injury or death.

- Do not use if the packaging is open or damaged prior to use.
- The packaging should be inspected for holes, tears, broken seals, etc.
- This device is intended for short term use and should be discarded after 24 hours.
- The Colostrum Protector is a personal care item intended for use by a single user only and should not be shared between users.

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#### **CAUTION**

Can lead to minor injury.

- Use Medela Symphony breast pump kit only.

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

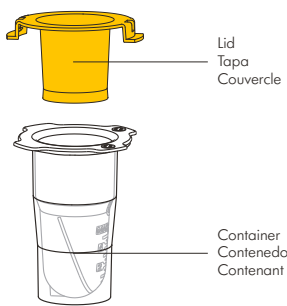
**EN Assembly:**

The Colostrum Protector is sterile and does not need to be cleaned prior to use.

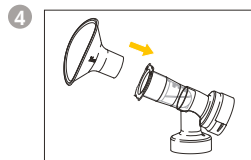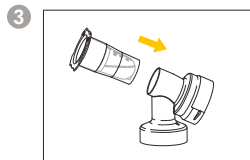- 1 Remove the Colostrum Protector from the packaging.
- 2 Avoid touching the inside of the lid or collector and set aside.

3 Attach the bottom of Colostrum Protector to the Symphony breast pump kit connector. Orient Colostrum Protector with the two raised marks (smiley faces) facing up.

4 Attach the breast shield to the top of the Colostrum Protector.

5 Assemble the remaining breast pump kit as directed in the Symphony breast pump kit instructions.

**Note:** Orientation is important to ensure the vent hole is facing up. Orienting any other way may result in colostrum spilling from vent hole and into the connector.

**Disassembly:**

- 1 Gently remove Colostrum Protector from connector and leave the breast shield attached.
- 2 Place the Colostrum Protector on a flat surface.
- 3 Remove the breast shield.
- 4 Secure the lid to the Colostrum Protector.
- 5 Store colostrum according to hospital guidelines.

**Wash After Each Use\***

Wash in a Dedicated Wash Basin

- 1 Rinse with clear water
- 2 In a dedicated wash basin for breast pump parts, soak in warm, soapy water for 5 minutes.
  - Do not place directly in the sink.
- 3 Wash with a clean dishcloth or soft brush.
- 4 Rinse with clear water.
- 5 Place on a clean surface and/or towel.
- 6 Allow to air dry.
- 7 Store in a clean, cool place when not in use. Do NOT store wet or damp parts.
- 8 Discard after 24-hours of use according to local regulations or hospital protocol.

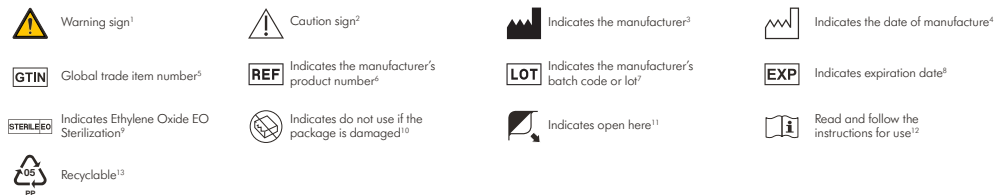
\*When using the Colostrum Protector for infants in the NICU, follow HMBANA best practice, which recommends using a new container for each pumping session.<sup>1</sup>

**Warranty:** Limited 90-day warranty. For complete warranty information visit the product page at Medela.us or Medela.ca.

**Material:** Polypropylene

**Disposal:** According to local regulations or hospital protocol.

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15400187 A 0723 © 2023 Medela 101046292
**Content appearance may vary.** Medela wordmark and logo is registered in the U.S. Patent and Trademark Office and the Canadian Intellectual Property Office. **El aspecto del contenido puede variar.** La marca denominativa y el logotipo de Medela están registrados en la Oficina de Patentes y Marcas de los EE. UU. y en la Oficina de Propiedad Intelectual de Canadá.