EN ASSEMBLY AND USE OF PUMP KIT

- Carefully push the white membrane onto the yellow valve head until it engages and is laying flat.
- Push yellow valve head onto inner ring of connector bottom.
- Screw connector onto the 35 mL container.
- Attach the bottom of Colostrum Protector to the connector. Orient Colostrum Protector with the two raised marks (smiley faces) facing up. (Using Colostrum Protector is optional)
- Attach the breast shield to the top of the Colostrum Protector or connector if Colostrum Protector is not being used.
- Insert the tubing into the corresponding opening in the connector.
- Place the protective membrane onto a flat surface and carefully push the membrane cap down until it engages.
- Insert the tubing adapter into the opening in the membrane cap.
- Open the lid of the breast pump. Press the oval button on the top of the Symphony® and lift the handle.
- Position the membrane cap firmly on the breast pump membrane to form a seal. Center the tubing connector in the recess in the pump housing. Make sure that the membrane cap engages and lies flush to the surface.
- Center the tubing connector in the recess in the pump housing. Make sure that the membrane cap engages and lies flush to the surface.
- Close the lid.
- Center breast shields over your nipples and start pumping. (Caution: pushing shields too firmly onto breasts may affect milk flow)

Note: If single pumping, only seat one membrane cap.
Note: Discontinue use of Colostrum Protector after milk volumes exceed 5 mL.
Refer to Symphony® breast pump Instructions for Use for pump operation information.

WASH AFTER EACH USE*

Separate all parts that come into contact with the breast and breast milk before cleaning.

- Rinse all separated parts in cool water in order to remove breast milk residue.

Wash in a Dedicated Wash Basin

- Once rinsed, parts that have touched the breast or breast milk should be placed in a clean wash basin, dedicated to only cleaning these items. Do not place parts directly in the sink.
- Soak all separated parts in warm, soapy water for 5 minutes.
- Wash each part with a clean dishcloth or soft brush.
- Rinse all separated parts with clear water.
- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Alternative, wash in dishwasher

- Wash all separated parts, except the Colostrum Protector, on top rack of dishwasher.
- Allow all breast pump parts to air dry in a clean area.
- Store dry parts in a clean, cool place when not in use.

Note: If the individual components of the pump kit are cleaned in a dishwasher, parts may be discolored by food pigments. This will not impact part function.

Note: Store the pump kit in a clean bag/container until next use.
Note: Only wash tubing if milk is present or tubing is soiled.

SANITIZE ONCE PER DAY*

Medela pump kit parts, except the Colostrum Protector, can be sanitized by boiling or using the Medela Quick Clean™ Micro-Steam™ bag (sold separately).

- Separate all parts that come in contact with breast milk.
- Discard the Colostrum Protector after 24-hours of use according to local regulations or hospital protocol.
- Wash hands thoroughly.
- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water for 10 minutes.
- Allow water to cool and gently remove parts from with tongs.
- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Note: Store the pump kit in a clean bag/container until next use.
Note: If you notice a white residue on your parts after boiling, you may have a high mineral content in your water. Remove residue by wiping parts with a clean washcloth and air dry. We recommended that you boil parts with distilled water to prevent substantial mineral build-up over time which may compromise your parts

If sanitizing your pump kit parts with the Quick Clean Micro-Steam bag, separate all parts that come in contact with your breast or breast milk and follow the instructions on the Quick Clean Micro-Steam bag.

Not for use with Colostrum Protector.

TUBING CARE

Inspect tubing after each pumping session.

Tubing should be washed if milk is present or tubing is soiled.

- If tubing becomes moldy, discontinue use and replace tubing. Replacement parts can be purchased www.ShopMedela.com.
- Remove tubing from breast shield.
- Rinse tubing in cool water to remove breast milk.
- Once rinsed, place in a clean wash basin, dedicated to only cleaning breastpump parts. Do not place parts directly in the sink.
- Soak tubing in warm soapy water for 5 minutes.
- Rinse tubing with clear water.
- Hang to air dry.
- Or shake out any drops of water from the tubing and let the Symphony breast pump run with tubing attached for 1-2 minutes or until dry.

Note: If there is condensation in the tubing: Shake water droplets out of the tubing and hang it up to dry or let the Symphony breast pump run with tubing attached for 1–2 minutes or until dry.

⚠ WARNING:

- Close supervision is necessary when this product is used near children.
- Use the product only for its intended use as described in this manual.
- Never drop or insert any object into any opening or tubing.
- Use Medela original accessories only.
- Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.
- Inspect the breast pump kit components prior to each use.
 - Do not use the components if they are damaged.
 - Do not use the components if dirt, mold or other contamination is visible.
- Inspect all appropriate pump components before each use.
- Do not modify any breast pump kit components
- Wash all parts that come in contact with your breast and breast milk after every use.
- If problems or pain occur, consult your lactation consultant or physician.

⚠ CAUTION:

- Wash hands thoroughly with soap and water before touching breast pump, kit and breasts. Avoid touching the inside of containers or lids.
- Pump only with the breast pump in an upright position.
- Separate and wash all parts that come in contact with breast and breast milk immediately after use to avoid dry up of breast milk residue and prevent growth of bacteria.
- Do NOT shorten tubing between breast shield and pump.
- Do NOT use abrasive cleaners/detergents when cleaning breast pump or breast pump parts.
- Do not attempt to remove the breast shield from your breast while pumping. Turn the breast pump off and break the seal between your breast and breast shield with your finger, then remove breast shield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.
- Do not store wet or damp parts as mold may develop.
- Inspect the tubing for condensation or milk after each use. Wash or replace the tubing immediately after use, if milk is observed in the tubing.
- If tubing becomes moldy, discontinue use and replace tubing.
- Do not use the breast pump kit with a non-compatible breast pump.
- Do not use the breast pump kit for any other purpose than expressing breast milk.
- Only use drinking-quality tap or bottled water for cleaning.
- While some discomfort may be felt when first using a breast pump, using a breast pump should not cause pain.

NOTE

- *Sterile kits are ready to use and do not require washing or sanitizing prior to first use.
- Wash breast pump kit parts after each use.
- Take care not to damage parts of the pump kit during cleaning.
- Only wash tubing if milk is present or tubing is soiled.
- Store the pump kit in a clean bag/container until next use.
- Sanitize breast pump kit parts once daily.
- When using the Colostrum Protector for infants in the NICU, follow HMBANA best practice, which recommends using a new container for each pumping session.¹

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EN INTENDED USE

The Symphony® breast pump kits are intended to use with Symphony™ Breast pump by lactating women to express and collect milk from their breast.

PLEASE SAVE THESE INSTRUCTIONS.

Use Symphony® breast pump kit Instructions in association with Symphony™ breast pump Instructions for Use

IMPORTANT

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breast milk if bottles or components become damaged.

1. Jones, F. Best Practice for Expressing, Storing, and Handling Human Milk in Hospitals, Homes, and Child Care Settings. 4th ed Fort Worth, TX: Human Milk Banking Association of North America, Inc. (2019), p. 59

See Medela website or contact Medela Customer Service at 1-800-435-8316 for Breast Shield Sizing, Breast milk storage guidance, Supplemental information and Warranty

WARRANTY

Limited 90-day warranty. For complete warranty information visit the product page at Medela.us or Medela.ca.

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*Medical devices - Symbols to be used with information to be supplied by the manufacturer -Part 1: General requirements

*This symbol is not derived from standards

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EN PARTS

1. Breast Shields
2. Colostrum Protector
3. Connectors
4. Yellow Valves
5. White Membranes
6. Breast milk bottle/collection containers (bottle size may vary based on breast pump kit)
7. Lids (not shown)
8. Tubing (not shown)

Kit components may vary from pictures in this instruction manual.

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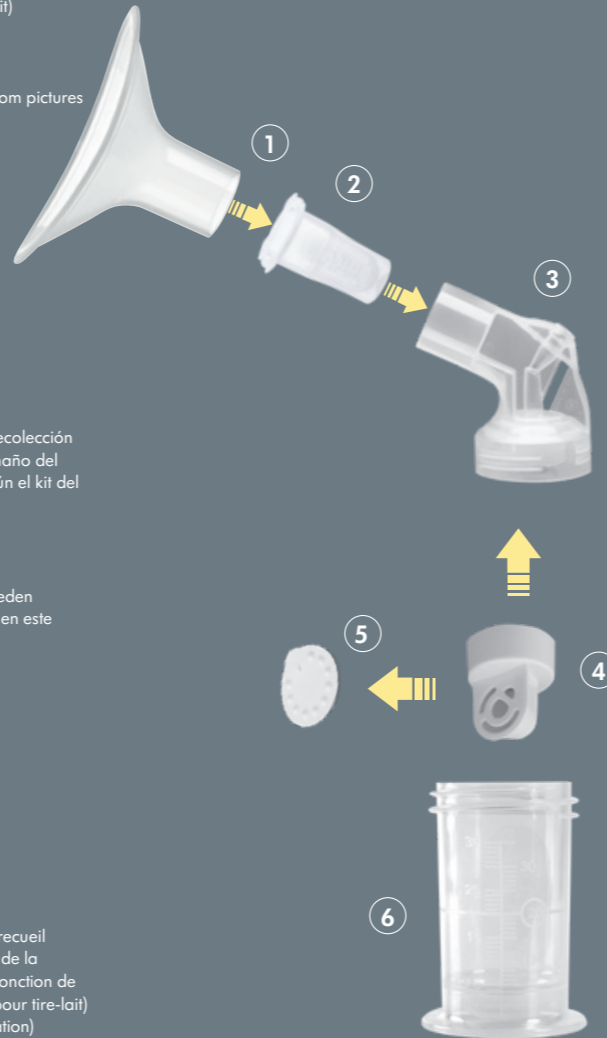
EN Instructions

Symphony® Breast Pump Kit

- ▶ For use with Symphony breast pumps
- ▶ For double or single pumping

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