## EN INTENDED USE

The Symphony® & Harmony® Breast Pump Kit is intended to use with Symphony & Harmony Breast pump by lactating women to express and collect milk from their breast.

## PLEASE SAVE THESE INSTRUCTIONS.

Use Symphony® & Harmony® breast pump kit Instructions in association with Symphony & Harmony breast pump Instructions for Use

## IMPORTANT

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breast milk if bottles or components become damaged.

## EN SYMBOL GLOSSARY

	Manufacturer (ISO 15223-1, Clause 5.1.1)
	Date of Manufacture (ISO 15223-1, Clause 5.1.3)
	Use-by-date (ISO 15223-1, Clause 5.1.4)
	Batch code (ISO 15223-1, Clause 5.1.5)
	Catalogue number (ISO 15223-1, Clause 5.1.6)
	Sterilized using ethylene oxide (ISO 15223-1, Clause 5.2.3)
	Do not use if package is damaged and consult Instructions for Use (ISO 15223-1, Clause 5.2.8)
	Single sterile barrier system (ISO 15223-1, Clause 5.2.11)
	Do not re-use (ISO 15223-1, Clause 5.4.2)
	Consult Instructions for Use (ISO 15223-1, Clause 5.4.3)
	Caution (ISO 15223-1, Clause 5.4.4)
	General Warning Sign

See Medela website or contact Medela Customer Service at 1-800-435-8316 for Breast Shield Sizing, Breast milk storage guidance, Supplemental information and Warranty

## WARRANTY

Limited 90-day warranty. For complete warranty information visit the product page at Medela.us or Medela.ca.

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## EN PARTS

1. Breast Shields
2. Connectors/Pump body
3. Yellow Valves
4. White Membranes
5. Breast milk bottle/collection containers (bottle size may vary based on breast pump kit)
6. Harmony handle
7. Lids (not shown)
8. Tubing and membrane caps (not shown)

Kit components may vary from pictures in this instruction manual.

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medela   
THE SCIENCE OF CARE™

## EN Instructions

### Symphony® and Harmony® Breast Pump Kit

- ▶ For use with Symphony & Harmony breast pumps
- ▶ **Symphony:** for double or single pumping
- ▶ **Harmony:** for single pumping

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