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8–12x in 24h

Pumping session ≈ 15 mins

Stimulation phases, pumping phases and breaks

When to switch from INITIATE to MAINTAIN?

1

Once at least 20 ml or more is expressed in three **consecutive** pumping sessions

or

2

When physiological markers of secretory activation* become visible

*feeling of breast engorgement (milk comes in)

or

3

Day 6 +

Switch to the **MAINTAIN program** no later than day 6

MAINTAIN program

BUILDING AND MAINTAINING LACTATION

from day 6

8–12x in 24h

Pumping session ≥ 15 mins


2-Phase Expression (stimulation and pumping phase)

Note: This quick guide does not replace the complete instructions for use. Please refer to the Symphony instructions for use for further information.

INITIATE program

You can tell this program is running from these symbols on the display:

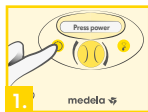Press power


Press the "Let-down" button to use the **INITIATE program**

Set vacuum


NOTE: Program ends automatically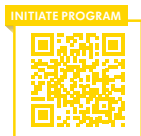

youtu.be/1Jzah7CEaEQ

MAINTAIN program

You can tell this program is running from these symbols on the display:

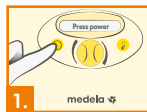Press power

Set a vacuum. After 2 minutes, system switches automatically to the pumping phase (4.)

Re-set vacuum

NOTE: Program does not end automatically

youtu.be/5A0M6Gth0W6

After 10 seconds, the **MAINTAIN program** starts

If milk begins flowing before the two minutes have passed, press the "Let-down" button to switch straight to pumping mode (4.).

IMPORTANT! SET THE VACUUM CORRECTLY AND CHECK/ADJUST IF NEEDED

Always use the maximum vacuum level comfortable. This helps produce optimal milk flow and makes pumping most effective.

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