



Colostrum Collection and Storage Containers – 35 ml

Important Safeguards

- *** Caution:** This is a single use product not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and / or biological characteristics. Reuse could cause cross contamination.
- Use a new disposable bottle for each pumping session.
- Do not use when package is damaged.
- Use Medela original accessories only.
- Never leave accessories unattended and keep out of reach of children.
- If problems or pain occur, consult your lactation consultation or physician.

Storing / Thawing Breastmilk

- Refer to Medela breastpump instructions for storing guidelines or check with your hospital for specific instructions.
- Storage capacity up to 35ml / Pumping capacity up to 20ml
- Bisphenol-A Free, Latex Free

Disposal

The bottles and lids are made of plastics and may only be disposed of in accordance with local regulations.

REF **200.2110** = Multibox of 50 Sets (total 100 bottles/lids)



Unless opened or damaged



810400

03/05/09

03/05/12



Medela, Inc., McHenry, IL 60098-0660 USA