⚠️ Please read all instructions before using this product. Keep these instructions for future reference.

Intended use

In combination with a Medela breast milk bottle, the teat is intended to feed human milk and other liquids of similar consistency. Breastfeeding should be well established before introducing bottle feeding.

Important safety information

⚠️ For your child's safety and health WARNING!

- Continuous and prolonged sucking of fluids will cause tooth decay.
 - Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the baby is allowed to use the teat for long periods through the day and particularly through the night, when saliva flow is reduced or if it is used as a soother.
- Always check food temperature before feeding.
- Inspect the feeding teat before each use and pull the feeding teat in all directions.
 - Throw away at the first signs of damage or weakness.
 - Keep components not in use out of the reach of children.
 - Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- Always use this product with adult supervision.
 - Accidents have occurred when babies have been left alone with drinking equipment due to the baby falling or if the product has disassembled.
- Never use feeding teats as a soother.
- Due to strangulation hazards do not extend cords or loops.
- Do not warm milk in a microwave oven as this may cause uneven heating and could scald the baby.
- Only touch the locking ring of the teat in order to avoid contamination.
- Do not expose the product to direct sunlight or heat.
- Use only Medela original accessories to ensure the proper function.
- Other food heating, cleaning, storage and use methods, different than the one recommended, have not been validated, are not applicable and may damage the product.

Cleaning and Hygiene

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Take care not to damage parts with using a brush.
- Inspect all items before each use.
- Replace parts if damaged or worn out.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Note: Food colorings may discolor components which will not affect performance.
- Follow these cleaning instructions unless you are told otherwise by your healthcare professional.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant („sterilizing solution“) for longer than recommended, as this may weaken the teat.

Prior to first use and once daily

- Disassemble, wash and rinse all parts.
- Place parts in a saucupan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes. This is to ensure hygiene.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

As an alternative to boiling:

Diswasher: Clean parts in top rack.
 Dishwasher: Use Medela Quick Clean™ Microwave Bags or other microwave-safe device.
 Steam sterilizer: Follow the manufacturer's instructions.

After each use:

- Disassemble and wash all parts in warm, soapy water.
- Rinse in clear water. Dry parts with a clean towel.
- After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid.
- Alternatively, the parts can be wrapped in a clean paper or cloth towel.

- Do not place in direct sunlight.
- Avoid touching the inside of bottles and lids.

Material Silicone
 Disposal According to local regulations
 Conforms to EN 14350.

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