

# Colostrum Protector

Every Drop Counts™



Colostrum is a powerful therapy for the NICU infant and can lead to improved patient outcomes.<sup>1</sup> Every drop of colostrum is valuable and should be protected from loss. The Colostrum Protector is easy to use and seamlessly connects to the Symphony® Breast Pump, without impacting pump performance. It catches all expressed drops of colostrum without getting lost in the breast pump parts, making it more effective than most traditional methods.

## Inspired by insights from nurses

- **Simple and easy to use:** The Colostrum Protector is easy to use and seamlessly connects to the Symphony® Breast Pump, without impacting pump performance.
- **Minimizes loss of colostrum:** The Colostrum Protector minimizes loss of colostrum by catching it prior to reaching the pump parts; ensuring baby can get every drop.
- **Effective:** 84% of nurses find the Colostrum Protector to be more effective at collecting colostrum than pumping without it, allowing you to collect colostrum with confidence.<sup>2</sup>
- **Easy to transport:** The Colostrum Protector’s snap on lid makes it easier to transport and protects from contamination, allowing you to focus on patient care.
- **Increases mom’s sense of accomplishment:** The Colostrum Protector is perfect for holding the small amounts of colostrum produced in those first few days. 78% of nurses felt mom had a greater sense of accomplishment using the Colostrum Protector.<sup>2</sup>

## How does the Colostrum Protector work?\*

- 1 Colostrum Protector is attached to the Symphony® kit, between the breast shield and the connector.
- 2 Mom uses the Symphony breast pump to initiate breast milk production.
- 3 Colostrum is collected in the Colostrum Protector before reaching the rest of the pump parts, minimizing loss.  
**Tip:** In the event mom comes to volume sooner than expected and milk volumes exceed the 5 mL capacity; it will overflow into the container below.
- 4 After pumping, remove the Colostrum Protector from the Symphony kit keeping the breast shield on and set on a flat surface. Remove the breast shield and snap the lid onto the Colostrum Protector.



## Product Specifications:

Product Name	Item Number	Description	Unit (Each Information)		Shipper Information		
			Dimensions (H x W x D Inches)	Weight (lbs)	Qty/Case	Dimensions (H x W x D Inches)	Weight (lbs)
Colostrum Protector 2-Pack	101046344	Includes 2 Colostrum Protectors and lids	2 x 5.5 x 4	0.06	40 packs of 2	16 x 10.875 x 9.25	2.4
Symphony Breast Pump Kit with Colostrum Protector	101044744	Includes 1 Symphony Initiation Kit and 4 Colostrum Protectors and lids	4 x 9.5 x 11.5	0.5	10 kits	22 x 12 x 19	5

### Content appearance may vary.

<sup>1</sup>Meier PP et al. J Pediatr. 2017;180:15-21. <sup>2</sup> Medela 2022 Colostrum Collector Evaluation Survey  
\*Refer to Instructions for Use for complete instructions.