## EN Instructions for use Milk Collection Shells

The Medela Milk Collection Shells collect leaking milk and are comfortable to wear. The skin-friendly, flexible material mean that they are pleasant and discrete to wear.

### Indication

- During breastfeeding, when the milk flow starts in the other breast
- When too much milk leaks
- Avoid wearing at night to prevent tender spots.

### Assembly and Use

Put the silicone membrane on the shell. Be sure it snaps on all the way around. Place the assembled milk collection shell inside your bra with the opening in the silicone membrane centered over your nipple. We recommend wearing a nursing bra; nursing bras are roomier and the milk collection shell applies less pressure on the breast. Product must be used in the upright position, ensure that the spout is facing upwards. Avoid sudden movements while wearing the shell. When removing the shell, lean forwards somewhat. The collected milk can then be poured into a different container.

**Caution:** Only milk collected during breastfeeding should be saved using a sterile container. If the milk is not fed to the baby right away, immediately place the container into the refrigerator and use within 24 hours. To keep moisture from damaging the nipples, we recommend to remove the shells every 2-3 hours and let the breast air dry.

### Cleaning Instructions

#### Before first use and subsequently once daily

Place the separated parts in a small pot filled with cold water. Distilled water is recommended - if tap water is used, a little citric acid may be added to the water to avoid a build-up of lime scale. Bring the water to a boil and allow it to continue boiling for approximately 5 minutes. Drain off the water and rinse the parts thoroughly. Place the parts on a clean cloth to dry.

### After each use

Separate the parts. Shells and membranes should be washed with a mild cleaning solution, rinsed thoroughly in cold water and dried on a clean cloth.

### Note

Shells and membranes can be put in a dishwasher or autoclave.

### Material

Shell: Polypropylene  
Silicone membrane: Silicone

### Disposal

According to local regulations.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.

**We recommend the Medela Nipple Formers for flat or inverted nipples.**  
**Sore and cracked nipples can be protected from further irritation by using the Medela Breast shells.**

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