ENGLISH

Instructions for use SpecialNeeds™ Feeder

Read all instructions before use. SAVE THIS INSTRUCTION FOR USE.

Intended purpose

The SpecialNeeds Feeder is intended to feed breast milk and other liquids of similar consistency to infants.

Indications for use

The SpecialNeeds Feeder is indicated for use for infants with:
A weak condition such as low muscle tone;
I Neurologic, cardiac or respiratory disorders resulting in difficulties in coordinating sucking, swallowing and breathing.
I Oral-facial anomalies such as cleft lip/palate or strong tongue-tie;
I Pierre Robin Syndrome or Down Syndrome.

Important safety information

For a patient/ user/ third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

For the infants child safety and health – WARNING!

I Only use this product with adult supervision.
I Do not warm liquid in a microwave oven as this may cause uneven heating and could scald the infant.
I Use the product only for its intended purpose as described in this instructions for use.
I Thoroughly clean the device before and after each use.
I Use by more than one infant without adequate reprocessing may present a health risk and could cause cross-contamination.
I Use Medela's original accessories only.
I Inspect before each use. Throw away at the first signs of damage, mould or weakness.
I Never leave packaging and components unattended. Keep them out of reach of children.

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Retunjuk penggunaan SpecialNeeds™ Feeder

Cleaning

Important
I Only use drinking-quality water for cleaning.
I Repress all parts prior to first use (see below).
I Clean all parts immediately after use.

The cleaning instructions below are general instructions. For country-specific regulations and guidelines, ask your Medela agent or visit our website (www.medela.com).

Do not use the liquid if bottles or components become damaged.

Before first use and once a day

I Cover all parts with water and boil for five minutes.
I Leave to dry on a clean cloth.

After each use

I Disassemble the feeder into its individual parts. Make sure to separate the yellow valve plate from the rest. Also separate the white membrane from the valve plate.

Note: You can re-use the milk that is left in the bottle. However, you must throw away the milk left in the teat. Therefore, hold the feeder above the sink. Carefully unscrew the lid and remove it together with the teat and valve plate, so as not to spill milk. Discard the leftover milk from the teat.

I Rinse all parts with cold water (approx. 20 °C). Clean all parts with plenty of warm, soapy water (approx. 30 °C). Use a commercially available washing-up liquid, preferably without artificial fragrances and colouring. Rinse all parts with cold water (approx. 20 °C).

I Alternatively, clean the product in the dishwasher. Place the parts on the top rack or in the cutlery section. Use a commercially available dishwasher detergent.
I Leave to dry on a clean cloth.

Storage

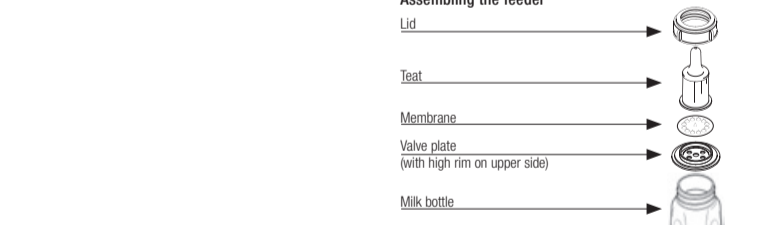
Put the clean parts in a clean storage bag or a clean environment. It is important that all residual moisture dries. Do not store parts in an airtight container/bag.

How to store

It is recommended that the mother performs her first feedings with the SpecialNeeds Feeder under the supervision of a professional/trained staff.

I Always check the temperature of the liquid before feeding.

Assembling the feeder

I Hold the valve plate with the high rim facing upwards. Place the white membrane onto the valve plate, inside the rim. Push the membrane's knob completely through the central hole of the valve plate.

I Fill the bottle with the required amount of liquid.
I Place the assembled valve with the high rim upwards onto the bottle opening.

I Place the teat on the valve plate, such that the rim is inside the teat.

I Slide the lid over the teat. Screw all parts together with the lid.

I Hold the feeder upright. Squeeze some air out of the teat.
I Keep squeezing while turning the feeder upside down.
I Now release the teat. Some liquid will enter it.
I Repeat the above steps until the teat is full.

ENGLISH

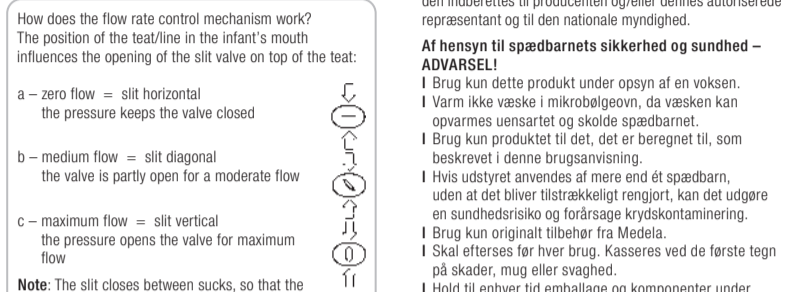
ENGLISH

Controlling the milk flow

The SpecialNeeds Feeder offers the opportunity to adapt the liquid flow to your infant's needs. Use the lines on the teat to set the liquid flow. The shortest line corresponds with zero flow, the middle line with a medium flow, the largest line for maximum flow.

To select a flow rate, point the corresponding line on the teat towards the infant's nose (see the picture at the right). Choose zero flow (the shortest line) if the infant needs to get used to the teat before the feeding starts.

Note: The slit closes between sucks, so that the baby will not be flooded.

Note: The feeder supports a sliding flow rate. To get any liquid flow in between zero and maximum, just choose the corresponding position in between the lines.

Material: Test and membrane: silicone; rest: polypropylene.
Disposal: According to local regulations.

Indicates compliance with the EU's requirements for medical devices

Indicates that the packaging contains products intended to come in contact with food according to regulation (EC) No 1935/2004

Indicates the material from which an item is made

Indicates the number of products included in the package

Indicates that an item can technically be recycled

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