www.medela.com

Medela, Inc.
1101 Corporate Drive
McHenry, IL 60050
e-mail: Customer.Service@medela.com

Medela Canada, Inc.
4090B Sladeview Crescent, Unit 2
Mississauga, Ontario,
Canada L5L 5Y5
e-mail: info@medela.ca

Phone: 1-800-435-8316 or
815-363-1166
Fax: 815-363-1246

SNS and Supplemental Nursing System are trademarks of Medela, Inc.
Medela is a registered trademark of Medela Holding AG.

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
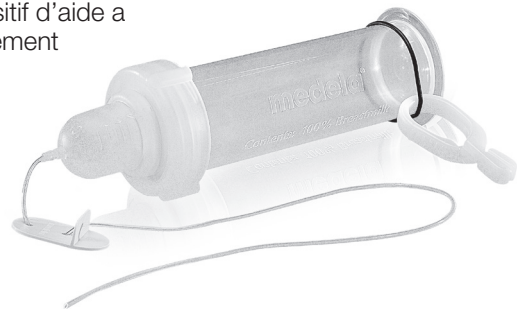
Article #101040421, #00902S, #0097003S (container included)
In Canada #20902, #20903 (container included)

Parts • Piezas de repuesto • Pièces

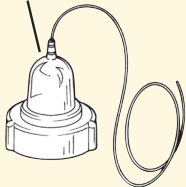
Teat-Tubing Assembly
Conjunto de tetilla y
manguera
Ensemble tétine-tube

Container (only with Article #0097003S and #20903)
Recipiente (incluido sólo con los artículos 0097003S y 20903)
Contenant (inclus seulement avec le produit no.
0097003S et 20903)

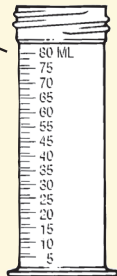Tubing Clamp
Abrazadera para tubo
Presse-tube


White Membrane
Membrana blanca
Membrane blanche

Disc
Disco
Disque

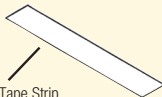Tape Strip
Cinta adhesiva
Bande de ruban adhésif

80ml Container
Recipiente de 80 mL
Contenant de 80 ml

Quick Clip and Elastic Ring
Pinza "Quick Clip" y anillo elástico
Attache «Quick Clip» et bande élastique

General Information

Contents:

Teat-Tubing Assembly; White Membrane, Disc, Quick Clip and Elastic Ring; Tubing Clamp; Tape Strip; Container (Container included with #0097003S or #20903).

CAUTION:

You are using your Starter SNS™ to assist you in breastfeeding your baby. Your baby should gain weight each week.

If your baby is not gaining weight, or loses weight, consult your healthcare professional immediately.

Visit your healthcare professional throughout the time you use your Starter SNS as often as he/she wishes to check the baby. Please make sure you read this manual well, for it contains information you need to know.

Extra assistance:

Very accurate hospital grade baby scales are available for rent at low daily costs. Call 1-800-TELL YOU for locations in your area.

Flow Regulation:

One way to adjust the flow is by the height of the device. When the bottom of the Starter SNS is above the level of the nipple, the supplement will flow more quickly and will continue to flow into the baby's mouth even when sucking has stopped. Such a non-stop flow can be uncomfortable, so use extreme caution when the device is above the level of the nipple, especially if the baby seems to gulp loudly or choke. Most babies will stop and pull away if this should happen.

Similarly, when the SNS is placed below the level of the nipple, the baby must suck against gravity. It could be difficult for the baby to suck the supplement in this case. Your healthcare professional may suggest this to promote a stronger suck. Make sure the baby swallows with every suck or every other suck if this technique is used. The flow can also be increased by very gently squeezing on the teat. This could be used when the infant has a very weak suck or needs extra assistance, such as with the reluctant nurser.

Some mothers have found that warming the supplement in the Starter SNS will increase the flow. It is usually not necessary to do this. However, it is an additional way to increase flow.

Cleaning Instructions

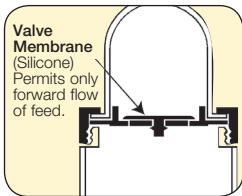
After each use:

1. Fill container full of warm soapy water and force water through tubing, teat and disc/membrane assembly by squeezing teat (making sure disc/membrane assembly is inside teat lid while cleaning).
2. Refill with clean water and again force water through tubing, teat and disc/membrane assembly.
3. Squeeze the empty teat a few times to remove water droplets from the tubings.
4. Completely disassemble and clean remaining parts with soapy water. Squeeze teat tubing assembly until the disc/membrane assembly falls out. Use fingernail of forefinger and thumb to remove membrane from disc.
5. Rinse well and drain dry on a clean towel.
6. Reassemble.


This device is intended for short term use and should be discarded after 24 hours.

Instructions for Use


1. Place elastic ring with QuickClip around bottom. (Container included with #0097003S, #20903). Use a Medela Collection and Storage Container or other graduated feeder. Have extra paper tape available if necessary.
2. Fill container with supplement recommended by your physician. Normally your body will heat the supplement as it passes through the tubing. However, if the supplement was taken from the refrigerator, warm it to approximately body temperature before pouring it into the container.
3. Press white membrane onto the upper side of the disc, so that the stud goes completely through the center hole.
4. Screw the Teat/Tubing assembly onto the container.

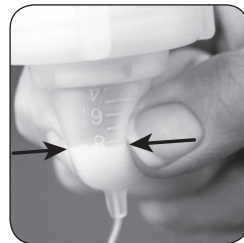
5. To prime the system, hold the unit in the upright position and, while gently squeezing the teat, invert the tubing assembly. Allow a small amount of the supplement in the teat. Next, gently squeeze the teat a second time to fill the tubing to help prevent the infant from swallowing air. Only a small amount of supplement should be in the teat. Do not fill teat completely.

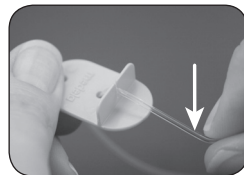6. Close off tubing using Tubing Clamp to avoid milk flow.

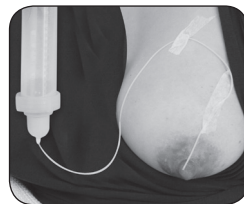7. Clip the container to the shoulder of the gown or bra. The bottom of the teat should be level with the mother's nipple.

8. Place tubing on the top of the nipple making sure it extends about 1/4 inch beyond nipple and secure it vertically with a piece of tape extending down to the base of the areola. Another piece of tape may be placed horizontally at the base of the areola if necessary. Make

sure the tubing is positioned so the baby keeps the tube in the center of its upper lip and not in the corner of its mouth.

9. Position baby at breast. Place all your fingers underneath the breast away from the areola. To begin milk flow, lift tubing off Tubing Clamp.

10. Tickle baby's lower lip with your nipple and wait until the baby opens wide. Place the tube under the baby's upper lip and gently but quickly pull the baby onto your breast. If you have difficulty, don't worry as this may take a little practice. Remove baby and start again.

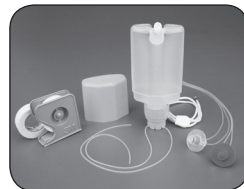
Note: Make sure the baby keeps the tube in the center of its upper lip and not in the corner of its mouth.

Useful Accessories

Supplemental Nursing System™

A reusable supplemental feeding device for extended use. Comes with three sizes of tubing.

Part #

(US) 00901S and (Canada) 20901

Medications and Mother's Milk

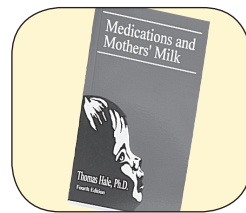
By Thomas W. Hale, Ph.D.

The effect of over 1000 drugs on the nursing infant.

Ordering Information

To order any of these useful accessories, contact your local authorized Medela Rental or Retail Station.

Products may also be ordered by calling 1-800-435-8316 and ordering with Discover Card, MasterCard, VISA, American Express, cashier's check or money order.

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art.1907576 D 1019 © 2019 Medela Printed in the USA.