



**neonatal
solutions**

Directions for Use

Enteral Syringe
TwistLok®

R_x Only

Not made from natural rubber latex.

Not made with Diethylhexylphthalate (DEHP).

WARNING:

- The TwistLok syringe is not intended for intravascular or intravenous use.
- This device has the potential to misconnect with small bore connectors of other healthcare applications.

CAUTIONS:

- Contents are sterile in an unopened, undamaged package. Do not use this device if opened or damaged.
- Do not re-sterilize.
- Do not use excessive force to remove the hub from the syringe.
- The patient's age, condition, and ability to feed orally will determine the method of feeding.

INDICATIONS FOR USE:

- The TwistLok Syringe is intended to be attached to a nasogastric feeding tube and/or extension set for introduction of enteral fluids and/or oral medication. The TwistLok Syringe is disposable and single use only.

RECOMMENDED PROCEDURE:

- Use clean technique.
- Ensure this device is only connected to an enteral port and not to an I.V. set.
- Attach TwistLok male connector to TwistLok female connector of feeding tube or extension set. Check to ensure the syringe connector is fully engaged to the feeding tube or extension set.
- Secure syringe to syringe pump.

SYRINGE REMOVAL:

- Remove the syringe and discard per hospital protocol.

 Medela LLC
1101 Corporate Drive
McHenry, IL 60050

(800) 435-8316

1548708 A 0717 © 2017 Medela DFU-018 Medela
wordmark and logo and TwistLok are registered in the
U.S. Patent and Trademark Office.