

Directions for Use

Enteral Extension Set

TwistLok[®]

R_x Only

Not made from natural rubber latex.
Not made with Diethylhexylphthalate (DEHP).

WARNING:

- The TwistLok Enteral Extension Set is not intended for intravascular or intravenous use.
- This device has the potential to misconnect with small bore connectors of other healthcare applications.

CAUTIONS:

- Contents are sterile in an unopened, undamaged package. Do not use this device if opened or damaged.
- Do not re-sterilize.
- Do not remove the hub from the enteral extension set.
- The patient's age, condition, and ability to feed orally will determine the method of feeding.

INDICATIONS FOR USE:

- The TwistLok Enteral Extension Set is intended to be attached to a nasogastric feeding tube and/or syringe for introduction of enteral fluids and/or oral medication. The TwistLok Enteral Extension Set is disposable and single use only.

RECOMMENDED PROCEDURE:

- Use clean technique.
- Ensure this device is only connected to an enteral port and not to an I.V. set.
- Attach TwistLok male connector to feeding tube.
- Attach TwistLok female connector to syringe.

SYRINGE REMOVAL:

- Remove the enteral extension set and discard per hospital protocol.