



**neonatal  
solutions**

## Directions for Use

GraviFeed® Enteral Syringe

TwistLok®

R<sub>x</sub> Only

Not made from natural rubber latex.

Not made with Diethylhexylphthalate (DEHP).

### WARNING:

- The TwistLok GraviFeed syringe is not intended for intravascular or intravenous use.
- This device has the potential to misconnect with small bore connectors of other healthcare applications.

### CAUTIONS:

- Contents are sterile in an unopened, undamaged package. Do not use this device if opened or damaged.
- Do not re-sterilize.
- The patient's age, condition, and ability to feed orally will determine the method of feeding.

### INDICATIONS FOR USE:

- The TwistLok GraviFeed syringe is intended to be attached to a nasogastric feeding tube and/or extension set for introduction of enteral fluids and/or oral medication. The TwistLok GraviFeed syringe is disposable and single use only.

### RECOMMENDED PROCEDURE:

- Use clean technique.
- Ensure this device is only connected to an enteral port and not to an I.V. set.
- Pour breast milk or formula into the TwistLok GraviFeed syringe and ensure the lid is securely closed.
- Place a thumb over the hole on the top of the lid and remove the non-vented cap.
- Attach TwistLok male connector to TwistLok female connector of feeding tube or extension set. Check to ensure the syringe connector is fully engaged to the feeding tube or extension set.

### SYRINGE REMOVAL:

- Remove the syringe and discard per hospital protocol.

 Medela LLC  
1101 Corporate Drive  
McHenry, IL 60050  
(800) 435-8316

1548669 A 0717 © 2017 Medela DFU-017  
Medela wordmark and logo, TwistLok and GraviFeed  
are registered in the U.S. Patent and Trademark Office.