

Silicone Breast Milk Collector

The new breastfeeding companion. Because every drop counts.



Did you know that two out of three mums experience breast milk leakage up to 6 months after giving birth? Within the first two months as much as 90 % of mums are affected.¹ Especially during breastfeeding or while the mother is pumping on one breast, the natural let-down effect can cause milk to leak from the other side.

To make sure every drop of milk is protected, Medela has developed the new Silicone Breast Milk Collector.

It is designed to fit all breast sizes. Made entirely of soft, 100 % food-grade silicone, the collector easily latches on to the breast and uses the natural let-down-effect to collect every drop of precious breast milk while the mother is feeding or pumping on the other side. To prevent baby's kicking feet from knocking the collector over, the adjustable clip on lanyard can be worn around the neck. The double leak-proof design with a lid and stopper ensures that the inside of the collector is kept clean and hygienic at all times, while the secure suction base keeps it stable and prevents spilling. Easy to clean and safe to use, made without BPA.

¹ Morse JM, Bottorff JL. Leaking: a problem of lactation. J Nurse Midwifery. 1989 Jan-Feb;34(1):15-20. doi: 10.1016/0091-2182(89)90124-9. PMID: 2926511


WITHOUT BPA

NEW

- SPILL PROOF DESIGN
- EASY TO USE
- CLIP ON LANYARD
- LIGHT AND PORTABLE
- EASY TO CLEAN
- CAPACITY 100 ml / 3.4 oz
- QUIET AND DISCREET
- SECURE SUCTION BASE

EASY TO CLEAN:

After each use, wash the Silicone Breast Milk Collector by hand or dishwasher. Before using it for the first time and once per day, sanitize, by boiling in water for approximately 3 minutes or using the Medela's Quick Clean™ Micro-Steam™ bags.

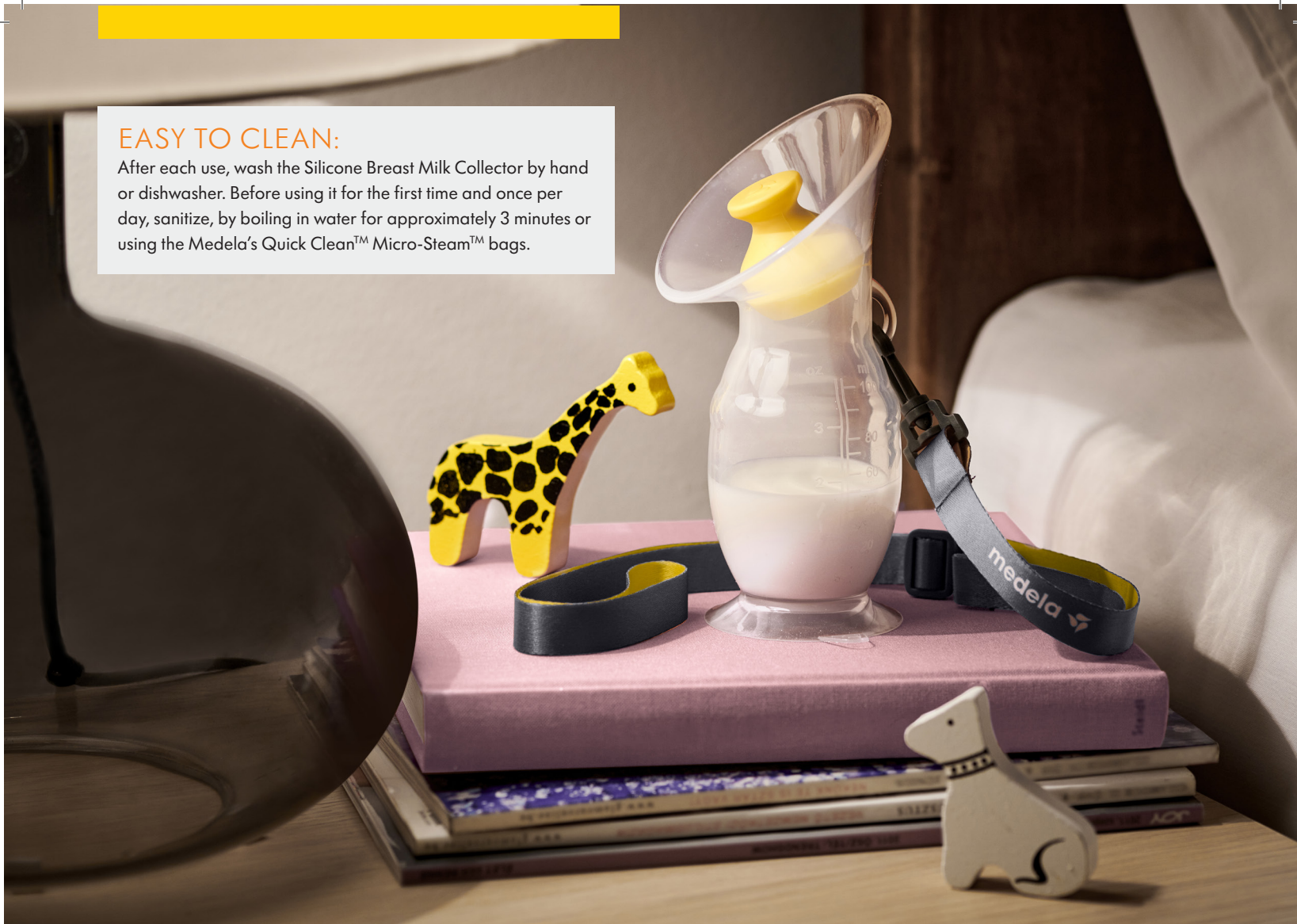

WHEN TO USE THE NEW SILICONE BREAST MILK COLLECTOR

The Silicone Breast Milk Collector is not intended to replace a breast pump, as it does not actively and effectively drain the breast. Mothers who depend on pumping to initiate, build or maintain their milk supply, should always use a hospital-grade double electric pump such as Medela Symphony™. If the mother needs to regularly have pumped milk available for her baby and also wants to pump on the go (when starting work again for instance) a personal pump such as Swing Maxi or Freestyle Flex is a good recommendation.



The Silicone Breast Milk Collector is a companion tool to breastfeeding or pumping to assist in collecting naturally flowing breast milk which may otherwise be lost onto clothing or into nursing pads.

Milk supply can vary over weeks and months and even throughout the day. A mother with high milk supply may decide to use the Silicone Breast Milk Collector to build a freezer stash of milk for future feeds or times when she cannot be with her baby. Another clever way is to use the excess milk for adding it to bathwater because breast milk has plenty of added benefits over and above its nutritional value. A mother with lower milk supply will likely decide to feed the collected milk directly.

Whatever the individual circumstances, with the new Silicone Breast Milk Collector every precious drop of milk is now being protected.

For the comfort of mothers and the healthy development of every baby.

*Thank you for your
invaluable efforts for mums
and babies, every day!*



For more information on Medela's new Silicone Breast Milk Collector visit our website on:

