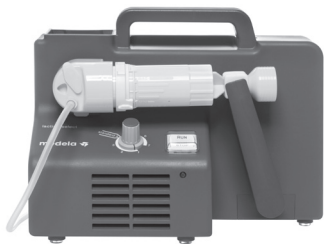
medela 
mother's milk,
everyday amazing™

Symphony® & Lactina® Breast Pump Kit

EN Instructions for use

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*14 parts that touch breast or are made without BPA.

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Table of Contents

1. Important Safeguards	4
2. Accessories	5
3. Cleaning Overview	6-7
4. Wash – Before first use & after each use	8-10
5. Sanitize – Before first use & once per day	12-13
6. Tubing Care	14-15
7. Assembly of the Pump Kit	16-25
7A. Symphony®	16-19
7B. Lactina®	20-23
7C. Manual Breast Pump Assembly	24-25
8. Breast Shield Sizing	26
9. Breast Milk Collection & Storage	27
10. Troubleshooting	28-29
11. Supplemental Information	30
12. Warranty	31

Important

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breast milk if bottles or components become damaged.

Symphony® Intended Use

The Symphony Breast pump is a powered breast pump to be used by lactating women to express and collect milk from their breast.

Symphony Product Description

Symphony is a multi-user, hospital grade electric breast pump that includes 2-Phase Expression® technology and is capable of single and double pumping.

Lactina® Intended Use

Mother's milk is the best nutrition for your baby. The Lactina Breast pump is a clinically proven pump suitable for frequent and long-term pumping.

Lactina Product Description

The Lactina Breast pump is a powered breast pump:

- To be used by lactating women to express and collect milk from their breast.
- To initiate and maintain a milk supply as well as collect breast milk at work or during other absences from the baby.
- To initiate and maintain a milk supply if direct breastfeeding is not possible.
- To relieve engorgement.
- For use in a hospital or home.

The warning symbol identifies all instructions that are important to safety. Failure to observe these instructions can lead to injury or damage to the breast pump. When used in conjunction with the following words, the warning symbols stand for:

WARNING Can lead to serious injury or death.

CAUTION Can lead to minor injury.

NOTE Can lead to material damage.

TIP Useful or important information that is not related to safety.

PLEASE SAVE THESE INSTRUCTIONS.

1. Important Safeguards

When using electrical devices, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

WARNING: To avoid health risk and reduce the risk of injury:

- Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.
- Inspect the breast pump kit components prior to each use.
 - Do not use the components if they are damaged.
 - Do not use the components if dirt, mold or other contamination is visible.
- Inspect all appropriate pump components before each use.
- Clean and sanitize all parts that come in contact with your breast and breast milk prior to first use. Sterile kits are ready to use and do not require washing prior to first use.
- Do not modify any breast pump kit components.
- Wash all parts that come in contact with your breast and breast milk after every use.
- Do NOT continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.
- If tubing becomes moldy, discontinue use and replace tubing.
- If breast milk backup occurs, you must clean tubing before your next pumping session.
- If problems or pain occur, consult your lactation consultant or physician.


CAUTION: Can lead to minor injury:

- Pump only with the breast pump in an upright position.
- Do NOT shorten tubing between breast shield and pump.
- Make sure the voltage of the device is compatible with the power source.
- Plug the power cord into the breast pump first and then into the wall socket.
- Do NOT use abrasive cleaners/detergents when cleaning breast pump or breast pump parts.
- Never put breast pump motor in water or a sterilizer, as you can cause permanent damage to the breast pump.
- Do not attempt to remove the breast shield from your breast while pumping. Turn the breast pump off and break the seal between your breast and breast shield with your finger, then remove breast shield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.
- Do not store wet or damp parts as mold may develop.
- Do not use the breast pump kit with a non-compatible breast pump.
- Do not use the breast pump kit for any other purpose than expressing breast milk.

2. Accessories

Symphony Protective Membrane and Cap (Lactina® to Symphony Conversion Kit contains 2 protective membranes and 2 membrane caps)	67091
PVC Tubing - Symphony, Lactina®, Classic™	101034773
PersonalFit™ Connectors	87071
PersonalFit 21 mm Breast Shield	87072
PersonalFit 24 mm Breast Shield	87073
PersonalFit 27 mm Breast Shield	87274
PersonalFit 30 mm Breast Shield	87075
PersonalFit 36 mm Breast Shield	87084
Replacement White Membranes	87088
Yellow Valve Heads & White Membranes	87089
Lactina Piston Cylinder Assembly (Symphony® to Lactina® Conversion Kit)	67090

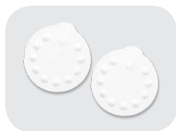Authentic Medela Spare Parts are made with the quality materials and workmanship you expect and trust, only from Medela.

Non-Medela spare parts can vary significantly in terms of product design, materials, and workmanship; all of which may affect the performance of your Medela breast pump.

Quality & performance
you can trust

3. Cleaning Overview

! Note

It's important for you to do the following before using Symphony® or Lactina® for the first time*:

1. For Symphony models that include a rechargeable battery, fully charge the battery 12 hours uninterrupted.
2. Separate all parts.
3. Clean - see instructions:
 - Wash – **section 4**
 - Sanitize – **section 5**

! Note

*Sterile kits are ready to use and do not require washing or sanitizing prior to first use.

Breast shield assemblies come assembled.
Separate all parts that will come into contact with the breast and breast milk before cleaning.

Parts to clean:

- Breast shields
- Breast milk bottle/collection containers
- Lids
- Connectors
- White membranes
- Yellow valve heads
- Piston
- Cylinder

Kit components may vary from pictures in this instruction manual.

	Breast pump kit & bottles/containers 	Tubing 	Lactina® piston/cylinder 
Before 1st use*	<ul style="list-style-type: none"> ✓ wash ✓ sanitize 		
After each use	<ul style="list-style-type: none"> ✓ wash 		
Once per day	<ul style="list-style-type: none"> ✓ sanitize 		
As needed		<ul style="list-style-type: none"> ✓ wash only if milk is in tubing or tubing is soiled 	<ul style="list-style-type: none"> ✓ wash every few pumping sessions

Detailed cleaning instructions on pages 8-15:

- Wash – pages 8-10
- Sanitize – pages 12-13
- Tubing care – pages 14-15
- Lactina piston cylinder cleaning care – page 10

! **Note**
 *Sterile kits are ready to use and do not require washing or sanitizing prior to first use.

! **Warning**
 Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

4. Wash - Before first use & after each use*

Supplies needed:

- Mild dish soap
- Clean dish cloth or soft brush
- Wash basin
- Drinking-quality water

Parts to wash:

- Breast shields
- Breast milk bottles/containers
- Lids
- Connectors
- White membranes
- Yellow valve heads
- Piston
- Cylinder

Separate parts for washing.

Caution

Separate and wash all parts that come in contact with breast milk immediately after use to avoid dry up of breast milk residue and prevent growth of bacteria.

Note

- Wash breast pump kit parts after each use.
- Take care not to damage parts of the pump kit during cleaning.
- It is not necessary to wash tubing, membrane cap or protective membrane prior to first use.
- Only wash tubing if milk is present or tubing is soiled.

Note

*Sterile kits are ready to use and do not require washing or sanitizing prior to first use.

Warning

Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

Wash in dishwasher

- Wash all separated parts on top rack of dishwasher.
- Allow all breast pump parts to air dry in a clean area.
- Store dry parts in a clean, cool place when not in use.

Note

If the individual components of the pump kit are cleaned in a dishwasher, parts may be discolored by food pigments. This will not impact part function.

Wash in sink with dedicated basin

2

Rinse all separated parts that came in contact with breast and breast milk in cool water in order to remove breast milk residue.

3

OR

- Once rinsed, parts that have touched the breast or breast milk should be placed in a clean wash basin, dedicated to only cleaning these items. **Do not** place parts directly in the sink.
- Soak all separated parts in warm soapy water for 5 minutes.
- Wash each part with a clean dish-cloth or soft brush.
- Rinse all separated parts with clear water.

Caution

Only use drinking-quality tap or bottled water for cleaning.

5

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Note

Store the pump kit in a clean bag/container until next use.

You may also wash your kit parts by following the instructions on the Quick Clean™ Breast Milk Removal Soap bottle. (sold separately)

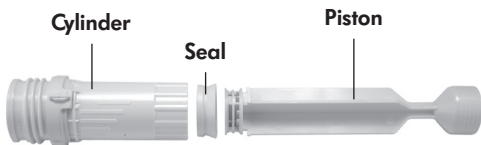
Questions? Visit www.medela.com or call us at 1-800-435-8316.

4. Wash - Before first use & after each use (cont.)

Lactina Piston/Cylinder Cleaning

Care for your piston and cylinder after every few pumping sessions with the Lactina breast pump or manual breast pump.

- Wash the cylinder, seal, and vacuum regulator ring with warm soapy water.
- Rotate the piston in the cylinder.
- If your piston/cylinder comes in contact with breast milk, remove and wash with breast pump kit.
- Do not use Quick Clean™ Micro-Steam™ bag to sanitize the piston and cylinder.

Questions? Visit www.medela.com or call us at 1-800-435-8316.

5. Sanitize – Before first use & once per day*

Supplies needed:

- Clean pot for boiling water
- Drinking-quality water
- Tongs

Parts to sanitize:

- Breast shields
- Breast milk bottles/
containers
- Lids
- Yellow valve heads
- Connectors
- White membranes

Note

*Sterile kits are ready to use and do not require washing or sanitizing prior to first use.

Warning

Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

1

- Separate all parts that come in contact with breast milk.

Note

Sanitize breast pump kit parts once daily.

4

Allow water to cool and gently remove parts from water with tongs.

- Wash hands thoroughly.

- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water for 10 minutes.

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Note

Store the pump kit in a clean bag/container until next use.

Note

If you notice a white residue on your parts after boiling, you may have a high mineral content in your water. Remove residue by wiping parts with a clean washcloth and air dry. We recommend that you boil parts with distilled water to prevent substantial mineral build-up over time which may compromise your parts.

You may also sanitize your kit parts by following the instructions on Medela's Quick Clean™ Micro-Steam™ bag. (sold separately) Not for use with piston and cylinder.

6. Tubing Care

Supplies needed:

- Mild dish soap
- Clean dish towel
- Clean sink or bowl
- Drinking quality water

Parts needed:

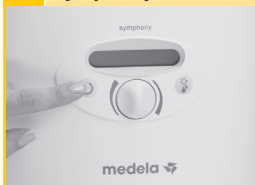
- Tubing
- Symphony® or Lactina® pump
- Membrane cap (Symphony)
- Protective membrane (Symphony)
- Piston (Lactina)
- Cylinder (Lactina)
- Pump connector (Lactina)

- Inspect tubing after each pumping session.

Note

If there is condensation in the tubing: Shake water droplets out of tubing and hang it up to dry or let the pump run with tubing attached for 1-2 minutes or until dry.

1 Symphony

Turn off breast pump.

1 Lactina

Turn off breast pump.

2 Lactina

Unplug breast pump from power source.

5 Lactina

Rinse tubing in cool water to remove breast milk.

6 Lactina

- Wash tubing in warm soapy water.
- Rinse tubing with clear water.
- Hang to air dry or shake out any drops of water from the tubing and let the pump run with tubing attached until dry.

Caution

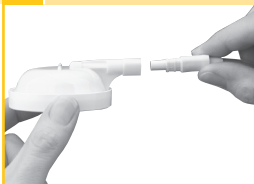
- Inspect the tubing for condensation or milk after each use.
- Wash or replace tubing immediately after use if soiled or milk is observed in the tubing.
- If tubing becomes moldy, discontinue use and replace tubing. Replacement parts can be purchased at www.shopmedela.com or call customer service at 1-800-435-8316.

2 Symphony

Unplug breast pump from power source.

3 Symphony

Remove tubing, protective membrane, and membrane cap. Clean following steps in **section 4**.

3 Lactina

Remove tubing from port on pump connector. Follow cleaning instructions for remaining kit parts in **section 4**.

4 Lactina

Remove tubing from connector.

Warning

Breast pump kits are a personal care item intended for use by a single user only and should not be shared between users.

Warning

Refer to breast pump instructions for use for important safeguards and operating information.

7. Assembly of the Pump Kit

T Tip

Carry out all steps with care and assemble the pump kit correctly for optimum performance.

Parts Needed:

- Tubing
- Symphony® pump
- Breast shields
- Breast milk bottles/containers
- Lids
- Connectors
- Yellow valve heads
- White membranes
- Membrane cap
- Protective membrane

! Note

- Use Authentic Medela Spare Parts only. See **section 2** for details.
- Check pump kit components for wear or damage before use and replace if necessary.
- All components must be completely dry before use for proper performance and to prevent damage to the pump.

7A. Symphony®

! Caution

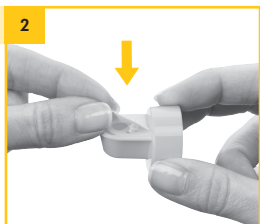
Wash hands thoroughly with soap and water before touching breast pump, kit and breasts and avoid touching the inside of containers or lids.

Push yellow valve head onto inner ring of connector bottom.

Place the protective membrane onto a flat surface and carefully push the membrane cap down until it engages.

Carefully snap white membrane onto yellow valve head until membrane lies flat.

Push the breast shield onto the connector.

Screw connector onto the bottle/container.

Insert the tubing into the corresponding opening in the connector.

Insert the tubing adapter into the opening in the membrane cap.

Warning

Refer to pump instructions for use for important safeguards and operating information.

7. Assembly of the Pump Kit (cont.)

Open the lid of the breast pump.
→ Press the oval button on the top of the Symphony® and lift the handle.

Position the membrane cap firmly on the breast pump membrane to form a seal. Center the tubing connector in the recess in the pump housing.
→ Make sure that the membrane cap engages and lies flush to the surface.

Note

If single pumping, only seat one membrane cap.

Center breast shields over your nipples and start pumping.
(Caution: pushing shields too firmly onto breasts may affect milk flow.)

To turn the pump on, press the power button. You will be starting in the Stimulation Phase. After 2 minutes, the pump will automatically change to the Expression Phase.*

*Pumping instructions are for Symphony Standard 2-Phase Expression® or when using the MAINTAIN program of Symphony PLUS™ program card. Please refer to the Symphony Breast Pump Instructions for Use, item # 1907724, for more detailed pumping instructions.

Center the tubing connector in the recess in the pump housing.
→ Make sure that the membrane cap engages and lies flush to the surface.

Close the lid.

 **Warning**
Refer to pump instructions for use for important safeguards and operating information.

 **Note**

- **For step 12:**
Always close the lid when pumping. The lid keeps the membrane cap pushed down to prevent a loss of vacuum that could occur if the membrane cap came off.

If let-down occurs before the Expression Phase begins, press the let-down button to begin the Expression Phase.*

When your pumping session is over, store your breast milk and clean your kit. See **sections 3-6** for cleaning instructions. Refer to **section 9** for storage guidelines.

7. Assembly of the Pump Kit (cont.)

T Tip

Carry out all steps with care and assemble the pump kit correctly for optimum performance.

Parts Needed:

- Tubing
- Lactina® pump
- Breast shields
- Breast milk bottles/containers
- Lids
- Connectors
- Yellow valve heads
- White membranes
- Piston
- Cylinder
- Pump Connector

! Note

- Use Authentic Medela Spare Parts only. See page 5 for details.
- Check pump kit components for wear or damage before use and replace if necessary.
- All components must be completely dry before use for proper performance and to prevent damage to the pump.

7B. Lactina®

⚠ Caution

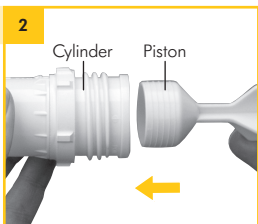
Wash hands thoroughly with soap and water before touching breast pump, kit and breasts, avoid touching the inside of containers or lids.


Attach pump connector to Lactina, and rotate cylinder down into position.


Snap white membranes onto yellow valve heads until white membranes lie flat.

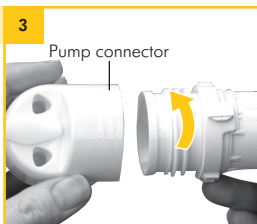
Slide the vacuum regulator ring onto cylinder. Push handle end of piston into cylinder.

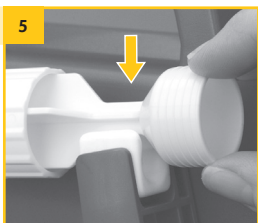Screw cylinder tightly into Lactina pump connector.

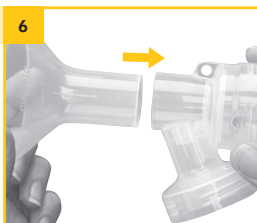Slip piston into pumping arm cradle, and push down until snug.

Securely attach breast shields to connectors.

Push yellow valve heads onto inner rings of connector bottoms.

Warning

Refer to breast pump instructions for use for important safeguards and operating information.

7. Assembly of the Pump Kit (cont.)

9
Screw connector onto the bottle/
container.

10
Insert pale yellow round ends of
tubing into openings in back of
connectors.

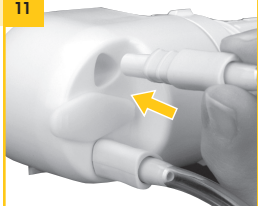
13
Turn pump on and set vacuum
regulator ring to "MIN." Rotate
vacuum regulator towards max to
adjust for comfort once pumping
begins.

! **Note**

- If single pumping, insert port plug into unused port.

11

Insert the triangular two-toned yellow ends of tubing into ports on pump connector.

12

Center breast shields over your nipples and start pumping. (Caution: pushing shields too firmly onto breasts may affect milk flow.)

14

When your pumping session is over, store your breast milk and clean your kit. Refer to **sections 3-6** for cleaning instructions. Refer to **section 9** for breast milk storage guidelines.

7. Assembly of the Pump Kit (cont.)

T Tip

Carry out all the steps with care and assemble the pump kit correctly for optimum performance.

Parts Needed:

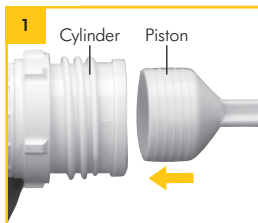
- Breast shield
- Breast milk bottles/container
- Lid
- Connector
- Yellow valve head
- White membrane
- Piston
- Cylinder

! Note

- Use Authentic Medela Spare Parts only. See page 5 for details.
- Check pump kit components for wear or damage before use and replace if necessary.
- All components must be completely dry before use for proper performance and to prevent damage to the pump.

7C. Manual Breast Pump Assembly

For manual breast pumping follow the instructions below.


Slide the vacuum regulator ring onto the cylinder. Push handle end of piston into the cylinder.

Push yellow valve head onto inner ring of connector bottom.

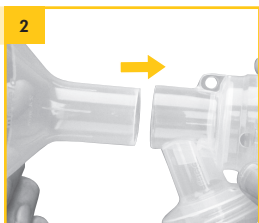Set vacuum regulator ring to "MIN".

2
Securely attach breast shield to connector.

3
Snap membrane onto yellow valve head until white membrane lies flat.

5
Screw connector onto the bottle/ container.

6
Screw cylinder into back of breast shield.

8
Center breast shield over your nipple and start pumping by pushing and pulling on the piston end.

9
When your pumping session is over, store your breast milk and refer to **section 9**. Refer to **sections 3-6** for cleaning instructions.

8. Breast Shield Sizing

1. Medela's PersonalFit™ sizing

Pumping should not hurt. For maximum comfort and pumping efficiency, Medela offers five breast shield sizes.

21
mm

24
mm

27
mm

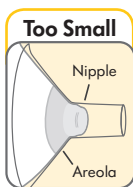
30
mm

36
mm


Visit MedelaBreastshields.com to view the full guide to help you determine your optimal size based on your nipple diameter.

2. Test your breast shield size


1. Start with the 24 mm that came with your pump, or the size determined by measuring.
2. Center nipple and gently hold breast shield against your breast.
3. Adjust for Maximum Comfort Vacuum™, to achieve optimum suction level.
4. Refer to images while pumping in expression.

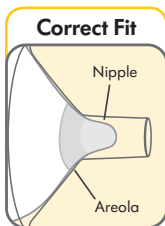- Nipple rubs along side of tunnel
- Try a larger size

- Nipple is centered and moves freely

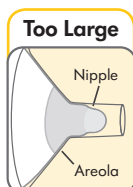- Nipple and excessive areola are pulled into the tunnel
- Try a smaller size

3. Should you try a new size?

- Does your nipple rub sides of tunnel, to the point of causing discomfort?
- Do you see excessive areola being pulled into tunnel?
- Do you see any redness?
- Is your nipple or areola turning white?
- Do you feel unexpressed milk after pumping?

If you answered **"YES"** to any of these questions, consider trying a new size by following the measuring instructions above.

If you are still unsure if you selected the correct size, see a lactation consultant, breastfeeding specialist, or visit MedelaBreastshields.com for assistance in choosing the right size breast shield.

Caution

- While some discomfort may be felt when first using a breast pump, using a breast pump should not cause pain. If you are unsure about breast shield sizing, please contact a health care professional or breastfeeding specialist who can help you get a proper fit.
- If you are experiencing discomfort at the base of the nipple due to rubbing of your breast tissue against the breast shield tunnel, use of a lubricant such as Tender Care™ Lanolin may be beneficial.

9. Breast Milk Collection & Storage

9. Storing breast milk

Storage Guidelines for Freshly Expressed Breast Milk (For Healthy Term Babies)			
Room Temperature 60°F to 77°F (16° C to 25° C)	Refrigerator 39°F (4°C) or colder	Freezer 0°F (-18°C) or colder	Breast Milk Thawed in the Refrigerator
Up to 4 hours is best *Up to 6 hours for milk expressed under very clean conditions	Up to 3 days is best *Up to 5 days for milk expressed under very clean conditions	Up to 6 months is best *Up to 9 months for milk expressed under very clean conditions	At room temperature: Up to 2 hours Refrigerator: Up to 24 hours DO NOT REFREEZE

References: www.BreastmilkGuidelines.com

*Guidelines for expressing milk under very clean conditions:

- Before expressing milk, wash your hands with soap and water or use an alcohol-based hand sanitizer.
- Pump parts, bottles and pumping area must be clean.
- Breasts and nipples do not need to be washed before pumping.

These guidelines for storage and thawing of breast milk are a recommendation. Contact your lactation consultant or breastfeeding specialist for further information

For additional breast milk collection and storage information, please visit: www.BreastmilkGuidelines.com

10. Troubleshooting

If the breast pump is not turning on	<p>Check whether electrical outlet power/battery is available.</p> <p>Check that the device is switched on.</p> <p>Check whether program card is inserted correctly in the card compartment on the back of the breast pump. (Symphony® breast pump only)</p>
If there is low or no suction	<p>Assemble the pump kit in accordance with the instructions in Section 7.</p> <ul style="list-style-type: none">• Check whether all connections of the pump kit and breast pump are secure.• Make sure that the breast shield forms a complete seal around the breast.• The white membrane must lie flat against the yellow valve head.• Check that the yellow valve head and white membrane are clean and not damaged.• The ends of the tubing must be firmly connected to the connector and the opening in the membrane cap. (Symphony® Breast Pump only)• The protective membrane and membrane cap should be correctly assembled and the membrane cap must be firmly fitted to the breast pump membrane to form a seal. (Symphony Breast Pump only)• The protective membrane must be intact. (Symphony® Breast Pump only)• Do not kink or pinch the tubing during expression.• All components must be completely dry. <p>Yellow Valve Head and White Membrane Check</p> <ul style="list-style-type: none">• Remove and separate yellow valve head from the white membrane.• Inspect the yellow valve head for chips or cracks. Inspect the white membrane for holes or tears. If flawed or damaged, replace the valve and/or white membrane prior to pumping.• Wash yellow valve head and white membrane per Cleaning Instructions in Sections 3-6.• Reassemble. Make sure the white membrane lays flat on the yellow valve head, and if it does not, replace the yellow valve head and/or white membrane prior to pumping. <p>Breast Shield and Breast Shield Connector Check</p> <ul style="list-style-type: none">• Inspect breast shield and breast shield connector to make sure there are no cracks and they securely attach to each other.• Make sure the breast shield forms a complete seal around the breast. <p>Other Checks</p> <p>Inspect the following to check for damage and secure connections.</p> <ul style="list-style-type: none">• Secure tubing adapter to the breast shield connector.• Test vacuum. If suction is not improved, contact Medela Customer Service at 1-800-435-8316.• Inspect piston/cylinder and refer to piston/cylinder cleaning in Section 4. (Lactina® Breast Pump only) <p>Tubing Check</p> <ul style="list-style-type: none">• If damaged replace prior to pumping.• Inspect tubing for damage and secure connections.• Check that the ends of the tubing are firmly connected to the connector and the opening in the membrane cap.• If single pumping, make sure only one membrane cap is seated. (Symphony® Breast Pump only)• If single pumping, insert port plug into unused port. (Lactina® Breast Pump only) <p>Piston/Cylinder/Tubing Check (Lactina® Breast Pump Only)</p> <ul style="list-style-type: none">• Make sure the ends of the tubing are fit securely into the back of the breast shield and into the opening on the pump connector.• Make sure the rubber seal is correctly attached onto piston.• Make sure the piston/cylinder is correctly attached onto the pump connector or breast shield.• Make sure the pump connector and piston/cylinder are assembled properly onto the Lactina.• If single pumping, make sure the port cap is inserted into the unused opening of cylinder holder. <p>Take care not to kink or pinch tubing.</p>

If there is an overflow	<ol style="list-style-type: none"> 1. Place the pump set into the bottle holder. 2. Turn off the pump and unplug the power cord from the electrical outlet. 3. Use a damp (not wet) cloth to wipe the pump and membrane area. 4. If there is milk in the tubing: Remove tubing and clean in accordance with Sections 4 & 6. If there is condensation in the tubing: Let the pump run with tubing attached for 1–2 minutes or until dry. 5. When components are completely dry, reassemble breast pump and accessories set.
--------------------------------	--

Condensation or Milk in Tubing	<ul style="list-style-type: none"> • Turn off the pump and unplug from power source. • Use a damp (not wet) cloth to wipe the pump. • Refer to cleaning instructions "Tubing Care" Section 6. • If necessary, clean the pump kit. • When components are completely dry, reassemble breast pump and kit.
No "Let-Down" or Breast Milk Expression	<ul style="list-style-type: none"> • Ensure your breast pump kit is assembled correctly and suction is available. • Relax and take a 10-15 minute break if let-down is not achieved. • Consult with a healthcare or lactation professional if expression does not occur after 2 consecutive pumping sessions.

If you have not resolved the problem with your breast pump or you have further questions, please contact Medela Customer Service at 1-800-435-8316 or fill out the contact form at www.medelabreastfeedingus.com.

11. Supplemental Information

Supplemental Information

Following are some common breastfeeding related conditions. If you experience any of these symptoms, contact a healthcare professional or lactation professional.

	Symptom	Potential Causes
Engorgement	Breasts are hard and uncomfortable, possibly with reddened areas.	Milk not draining properly from breast. Milk can drain back into the tissue causing areas to swell and become tender.
Blocked/Plugged Ducts	Area of breast will look reddened and may be tender to touch.	Milk not being drained from a specific duct. The area becomes "clogged" up and milk is then prevented from flowing.
Mastitis	It often follows engorgement but can occur suddenly. The initial symptoms are similar to flu – fatigue, headache, muscle aches, fever and localized breast tenderness. It can be in one or even both breasts and needs immediate treatment.	A bacterial infection in the breast tissue which is often accompanied by cracked nipples.

A Healthcare Professional or Lactation Professional can give you guidance in breastfeeding your baby and using a breast pump. Their guidance is valuable in terms of the lifetime of health benefits for both you and your baby. A listing of lactation professionals are available at www.medelalocator.com or calling 1-800-TELL-YOU (1-800-835-5968).

12. Warranty

This product is warranted by Medela to the original purchaser to be free from defects in material and workmanship for the period of 90 days from the date of purchase. Warranty can only be claimed in the country of purchase. In the event of defect, Medela will replace this product, without charge for such replacement parts. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. THE LIABILITY OF MEDELA FOR BREACH OF ANY WRITTEN OR IMPLIED WARRANTY IS LIMITED TO REPAIR OR REPLACEMENT OF THIS PRODUCT, AND MEDELA WILL HAVE NO LIABILITY UNDER ANY CIRCUMSTANCES FOR ANY INCIDENTAL, PUNITIVE, OR CONSEQUENTIAL DAMAGES. SOME AREAS DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS OR DAMAGE LIMITATIONS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE OR IN YOUR COUNTRY.

Before you make a claim under this warranty, it may save time and expense to call Medela Customer Service (toll free) at 1-800-435-8316. You may also call this number for additional information concerning this warranty. If you wish to make a claim under this warranty, you must call first for authorization number; returns not accepted without an authorization number. Return this product to Medela with a return authorization number received from Medela Customer Service, prepaid, together with your dated bill of sale or other proof of purchase.

Customer Service toll free number: 1-800-435-8316

Medela LLC – Returns, Door 4501
1101 Corporate Dr.
McHenry, IL 60050
ATTENTION: RETURNS

Medela Canada, Inc.
4160 Sladeview Crescent, #8
Mississauga, ON, L5L 0A1
ATTENTION: RETURNS

Call first for authorization number. Returns not accepted without an authorization number.

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ART.1908210 G 1018 © 2018 Medela 1908210 Printed in the U.S.A. Impreso en los EE. UU.
Medela wordmark and logo, Lactina and Symphony are registered in the U.S. Patent and Trademark Office. PersonalFit, Quick Clean, Tender Care and Micro-Steam are trademarks of Medela.