Harmony®

Manual breast pump
Instructions for use

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ESPAÑOL

31 – 44

EN Illustrated items or instruction steps are marked with the figure and detail number (e.g. 2-01).

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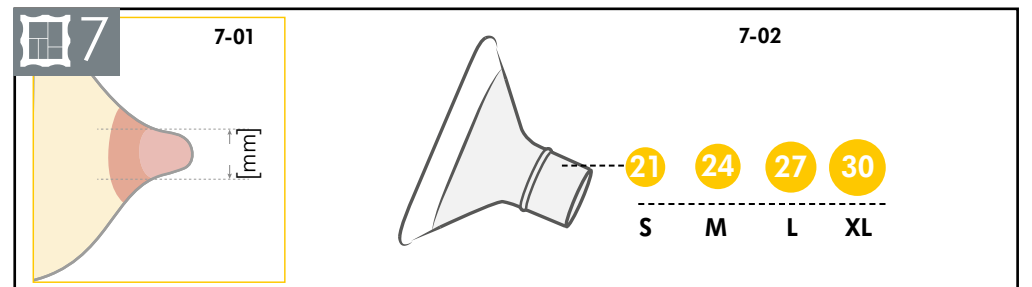
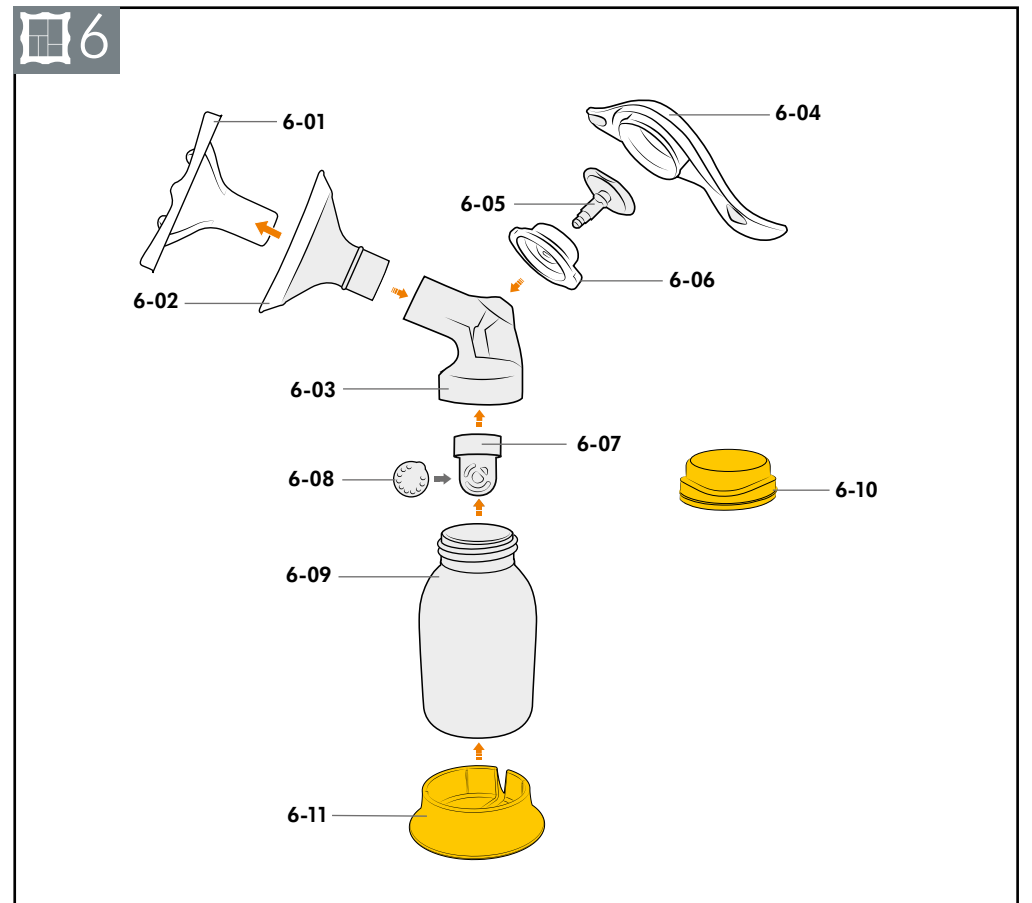
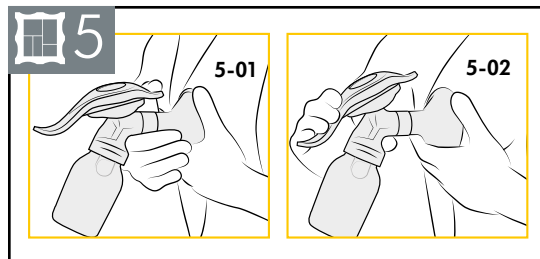
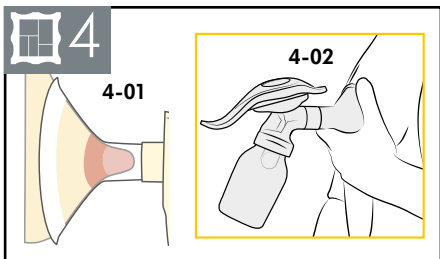
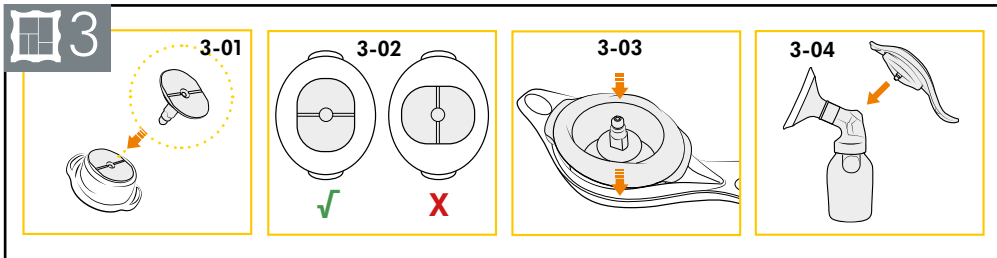
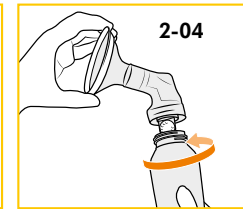
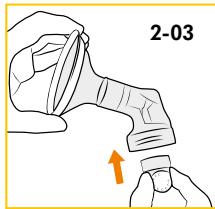
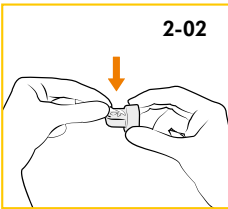
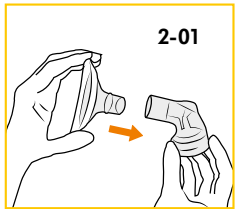
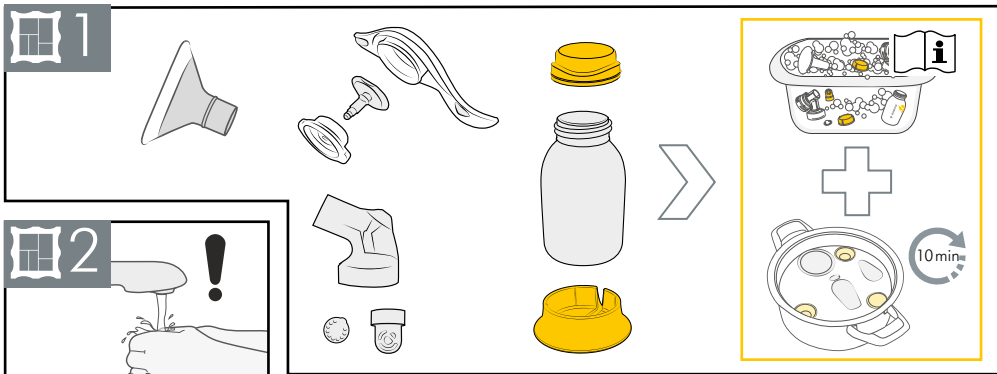
EN Fold this page out.

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1. Important safety information

**Please read all instructions before using this product.
Keep these instructions for future reference.**

Warnings identify all instructions that are important for safety. Failure to observe these instructions can lead to personal injury or damage to the product.

WARNING

Can lead to serious injury or death.

To avoid health risks and reduce the risk of severe injury:

- Pumping can induce labor. Do not pump until after giving birth. If you become pregnant while breastfeeding or breast pumping, consult with a licensed healthcare professional before continuing.
- If infected with Hepatitis B, Hepatitis C, or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
- Wash all parts that come into contact with your breast and breast milk after every use.
- Small parts may be swallowed by toddlers. Close supervision is necessary when the breast pump or accessories are used in the vicinity of children.

CAUTION

Can lead to minor injury.

To avoid health risks and reduce the risk of injury:

- This is a single-user product. Use by more than one person may present a health risk.
- The device and its accessories are not heat-resistant. Keep away from heated surfaces or open flames.
- Do not expose the device to direct sunlight.
- Before each use visually inspect the individual components for damage or wear. Never use a defective device. Throw away parts at first signs of damage or weakness.
- Do not repair the device by yourself. Do not modify the device.
- Use the device only for its intended purpose as described in these instruction for use.
- Do not microwave or boil breast milk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving (Microwaving can also change the composition of breast milk).

⚠ CAUTION

Can lead to minor injury.

To avoid health risks and reduce the risk of injury:

- Clean and sanitize all parts that come into contact with your breast and breast milk prior to first use.
- Only use Medela recommended parts with the device.
- While some discomfort may be felt when first using a breast pump, it should not cause pain. If you are unsure about correct breast shield sizing please visit **MedelaBreastShields.com** (U.S.) or **Medela.ca/breastshields** (Canada), or see a lactation consultant/breastfeeding specialist who can help you get a proper fit.

NOTICE

Can lead to material damage.

- Take appropriate care in handling bottles and components:
- Plastic bottles and parts become brittle when frozen and may break when dropped.
- Bottles and parts may become damaged if mishandled (e.g., dropped, over-tightened, or knocked over).
- Do not use the breast milk from bottles or components that show signs of damage.

Contents

1. Important safety information	3
2. Intended purpose	6
3. Product description	6
3.1 The Harmony® breast pump includes	6
4. Getting started	6
5. Cleaning	7
5.1 Washing and sanitizing the pump	7
6. Preparing for pumping	9
6.1 Selecting the correct breast shield size	9
6.2 Assembling the connector and breast shield	9
6.3 Assembling the handle	10
6.4 Placing the breast shield	10
7. Operating your breast pump	10
7.1 Glossary	10
7.2 Mimicking the baby's rhythm (2-Phase Expression®)	10
7.3 Expressing your breast milk	11
7.4 After pumping	11
8. Resources for you	12
9. Troubleshooting	14
10. Warranty	15
11. Disposal	16
12. Meaning of symbols	16
13. Technical specifications	16

2. Intended purpose

The Harmony® breast pump is a manual breast pump to be used by lactating women to express and collect milk from their breasts. The Harmony® breast pump is intended for a single user.

3. Product description

The Harmony® is a personal-use manual breast pump with a PersonalFit Flex™ breast shield and an ergonomic 2-Phase Expression® pump handle.

This enables you to alternate between short low-volume movements to stimulate the milk flow and long, deep movements to optimally extract your milk – the way your baby does it.

The Harmony® breast pump

- is easy to handle and clean.
- enables you to silently and discreetly extract your milk.
- includes a 24 mm PersonalFit Flex™ breast shield to give you more comfort and more milk.

3.1 The Harmony® breast pump includes

Refer to figure  6 on the foldout pages.

- | | |
|--|---|
|  6-01 (Disposable) protective cover |  6-07 Valve head |
|  6-02 PersonalFit Flex breast shield 24 mm |  6-08 Valve membrane (2 pcs) |
|  6-03 Connector |  6-09 Bottle |
|  6-04 Handle |  6-10 Lid |
|  6-05 Stem with O-ring |  6-11 Bottle stand |
|  6-06 Diaphragm | |

4. Getting started

To be ready for expressing milk for the first time, separate the parts of the breast pump, then wash and sanitize them – see **section 5.1**.

5. Cleaning

5.1 Washing and sanitizing the pump

Washing and sanitizing are two different activities. They must be done separately to protect you and your baby, and to maintain the performance of the device.

Wash – To clean the surfaces of the parts by physically removing contamination.

Sanitize – To kill microorganisms, such as fungi, bacteria or viruses, that may be present on the surfaces of the parts.

CAUTION

Can lead to minor injury.

- Separate and wash all parts that are exposed to breast milk immediately after use. This will help to remove breast milk residue and prevent growth of bacteria.
- Wash hands thoroughly with soap and water.
- Only use drinking-quality tap water or bottled water for cleaning.
- Do not place pump parts directly in the sink to rinse or wash. Use a clean wash basin used only for infant feeding items.
- Do not use a dish towel to rub or pat items dry.
- Do not use disinfectants for cleaning.

NOTICE

Can lead to material damage.

Pay attention to the following:

- Take care not to damage parts of the pump during cleaning. Do not use a sharp object (such as a toothpick) for cleaning.
- Some parts of the Harmony® breast pump are very small (e.g. the O-ring or the valve membrane). Make sure that no parts are missing when you reassemble the pump after cleaning.
- Store the dried pump parts in a clean bag/container until next use.
- Do not store wet or damp parts as mould may develop.
- If you notice a white residue on the parts after boiling, your water may have a high mineral content. Remove residue by wiping parts with a clean towel and allow to air dry.
- Distilled water is recommended when boiling parts to prevent substantial mineral build-up over time, which may compromise your parts.

i Information

If using the dishwasher, parts may become discolored. This will not impact the part function.

What to do when: See detailed instructions on the next page.

Before first use, remove and dispose of the protective cover (6-01) from the breast shield. Wash and sanitize the pump parts and bottles following the steps 1 to 4 below. Wash your pump parts and bottles **after each use** and sanitize the parts **once per day** after they have been washed.

1. Disassemble

Refer to figure 1 on the foldout page.

- i. Disassemble the pump into its individual parts (breast shield, connector, handle, valve head and milk bottle).
- ii. Remove the diaphragm from the handle and pull the stem out.
- iii. Remove the valve membrane from the valve head.
- iv. Separate the milk bottle, lid and bottle stand.

2. Wash (before first use and after each use)

Rinse and wash by hand

- i. Rinse the disassembled parts with cool water.
- ii. Clean these parts with plenty of warm, soapy water. Use a commercially available dish soap.
- iii. Rinse the parts with cool water for 10 to 15 seconds.

or

Wash in dishwasher

Place the disassembled parts on the top rack or in the cutlery section. Use a commercially available dishwashing detergent.

3. Sanitize (before first use and once per day)

Sanitize on stovetop

Cover the disassembled, washed parts with water and boil at least for 10 minutes.

or

Sanitize in microwave

(except for the handle)

Use Quick Clean™ Micro-Steam™ bags* in the microwave in accordance with the instructions on the bags.

*Refer to local website/shops for availability in your country.

4. Dry

Allow to air-dry on a clean, unused dish towel or paper towel. Store the dry parts in a clean environment. Do not store parts in an airtight container/bag if moist. It is important that all residual moisture dries.

6. Preparing for pumping

Only use clean and dry parts for assembly. Refer to **chapter 5** for cleaning.

CAUTION

Can lead to minor injury.

To prevent contamination of your milk:

- Wash hands thoroughly with soap and water before touching the pump parts and breasts.
- Dry your hands with a fresh towel or a single-use paper towel.
- Avoid touching the inside of bottles and lids.

To make sure that the device works properly and safely:

- Check pump components for wear or damage before use. Replace if necessary.
- Always inspect all parts prior to use for cleanliness. If dirty, clean according to **chapter 5**.

NOTICE

Can lead to material damage.

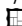
The O-ring ensures a sealed connection between the handle and the connector. This is necessary to build up a vacuum.

Make sure that the white O-ring is present in its groove of the stem.

If it is missing or defective, contact the Medela Customer Service.

6.1 Selecting the correct breast shield size

Refer to figure  7 on the foldout pages.

1. For proper sizing, measure the diameter of your nipple ( 7-01).
2. Select the breast shield that is slightly larger than your nipple ( 7-02).

For more information on breast shield sizing, visit **MedelaBreastShields.com** (U.S.) or **Medela.ca/breastshields** (Canada).

6.2 Assembling the connector and breast shield

Refer to figure  2 on the foldout pages.

1.  2-01 Carefully push the breast shield into the top opening of the connector.
 - Check to see if you are using the correct size breast shield. Visit **MedelaBreastShields.com** (U.S.) or **Medela.ca/breastshields** (Canada).
2.  2-02 Place the valve membrane in a horizontal position on the valve head. Carefully push the knob on the membrane's back into the smallest hole of the head until it engages.
 - The membrane must lie flat on the head.
3.  2-03 Push the valve head onto the inner sleeve of the connector.
 - The valve head must be positioned sideways as shown in the picture.
4.  2-04 Screw the bottle into the connector.

6.3 Assembling the handle

Refer to figure  3 on the foldout pages.

1.  3-01 Insert the stem (with the O-ring) into the hole on the upper side of the diaphragm.
2.  3-02 Make sure that the oval shaped base of the stem is aligned correctly to the diaphragm.
3.  3-03 Place the diaphragm into the pump handle. Mind the orientation (the recess in the diaphragm engages on the tongue in the handle) and press down firmly around the entire edge to ensure a proper seal.
4.  3-04 Insert the stem of the handle assembly in the opening on top of the connector. Push until the stem locks into place.

6.4 Placing the breast shield

Refer to figure  4 on the foldout pages.

1. The oval breast shields can be rotated (360°) and placed in the desired position to have the most comfortable fit for you.
2.  4-01 Place the breast shield on the breast so that the nipple is properly centered in the tunnel.
3.  4-02 Hold the breast shield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.
4. Start pumping as described in **section 7.3**.

7. Operating your breast pump

7.1 Glossary

2-Phase Expression® technology – research-based technology that mimics a baby’s natural nursing rhythm.

Expression Phase – slower sucking/pumping rhythm for gentle and efficient milk removal as quickly as possible.

Maximum Comfort Vacuum™ – highest vacuum setting where a mother still feels comfortable during pumping. It is different for every mother.

Stimulation Phase – fast sucking/pumping rhythm to stimulate the milk ejection reflex and to start the milk flowing.

7.2 Mimicking the baby’s rhythm (2-Phase Expression®)

Babies use a “two-speed” sucking action. At first, they suck quickly, and then they switch to a slower, deeper sucking rhythm once milk starts to flow.

The quicker sucking action stimulates the milk ejection reflex and triggers the milk flow; the slower sucking draws the milk out for the feeding phase.

2-Phase Expression® is a “two-speed” pumping action that mimics the baby’s natural sucking rhythms. The pump’s handle design allows you to begin with fast sucking action called the **Stimulation Phase**. For this, you actuate the short side of the handle with your thumb in a fast manner. When you see milk flow or feel milk ejection, you can then switch to the **Expression Phase** by pressing down on the longer side of the handle with your fingers in a slower, full stroke manner.

7.3 Expressing your breast milk

CAUTION

Can lead to minor injury.

- Do not hold the pump by the bottle. This can lead to engorgement and blockage of the milk ducts.
- Only fill the bottle to the 150 ml mark.
- Contact your healthcare professional or breastfeeding specialist if you can express only minimal or no milk or if expression is painful.

Refer to figure  5 on the foldout pages.

1.  5-01 Start pumping with stimulation: Using your thumb, press down the smaller part of the handle. This is the part with the milk drops on top. Press and release quickly, again and again, until your milk starts to flow.
2.  5-02 Once your milk starts to flow, switch to expression: With your fingers, press down the lower, longer part of the handle.
By doing so, you build up a vacuum that facilitates the milk flow. You control the level of the vacuum with the speed of the handle stroke and the compression that is created. Decide yourself which pumping rhythm is the best for you.
3. Stop pumping when you feel that the milk does not flow anymore or when the bottle is full.

7.4 After pumping

Prepare the milk for storage as follows:

1. Unscrew the bottle from the connector.
2. Use the bottle stand to prevent the bottle from tipping over.
3. Close the bottle with a lid.

For information on storing your breast milk, visit BreastMilkGuidelines.com (U.S.) or Medela.ca/breastfeeding/moms-journey/storing-and-thawing-breast-milk (Canada).

8. Resources for you

Helpful resources to support you throughout your breast milk feeding journey.

Visit our website for helpful videos.

MedelaVideos.com (U.S.) or
[youtube.com/channel/
 UCXu0hfQ0z-B1RNyS-2zKcSQ](https://www.youtube.com/channel/UCXu0hfQ0z-B1RNyS-2zKcSQ) (Canada)

Choosing the right breast shield size can impact your breast milk supply. Find out why and get help choosing the right size.

MedelaBreastShields.com (U.S.) or
[Medela.ca/breastshields](https://www.Medela.ca/breastshields) (Canada)

Complement your breastfeeding journey with our free, personalized, award-winning app, designed to help you achieve your goals. Track baby's key activities, log nursing and pumping details, and get access to breastfeeding tips. Available on Android and iOS devices.

MyMedelaApp.com (U.S.) or
[Medela.ca/breastfeeding/moms-journey/
 mymedela](https://www.Medela.ca/breastfeeding/moms-journey/mymedela) (Canada)

Learn more about storing
your breast milk.

BreastMilkGuidelines.com (U.S.) or
Medela.ca/breastfeeding/moms-journey/
storing-and-thawing-breast-milk (Canada)

An exclusive place for you to find the
right information, tools, and service,
delivered to you at the right time.
Join now to get access to breastfeeding
tips, support and deals for free!
Only available in the United States.

MedelaMomsRoom.com

Whether it's your first latch or you have been
breastfeeding for months, utilize these helpful
solutions to keep you on the path of providing
breast milk to your baby.

MedelaBreastfeedingUS.com/Breastfeeding-Guide (U.S.) or
Medela.ca/breastfeeding/moms-journey (Canada)

9. Troubleshooting

In case of an unexpected behaviour of your breast pump check with the troubleshooting table if you find the issue in the column “Problem” and follow the instructions in the column “Solution”.

Troubleshooting table

Problem	Solution
There is low or no suction	<ul style="list-style-type: none"> • Check that the O-ring on the stem is in place and not damaged. If it is missing or damaged, contact the Medela Customer Service. • Make sure that the breast shield forms a complete seal around the breast. • Make sure that the yellow valve head and the white membrane are clean and not damaged. • Make sure that the membrane lies flat on the valve head. Refer to figure  2-02 on the foldout page. • Make sure that the valve head is positioned sideways on the connector. • Make sure that the components of the handle are assembled correctly. Refer to figure  3-02 on the foldout page. • Check all connections. • If suction does not improve after having followed these steps, contact the Medela Customer Service.
There is no milk flow	<ul style="list-style-type: none"> • Ensure your breast pump is assembled correctly and there is suction. • Relax and take a 10-15 minute break if expression is not achieved after two consecutive pumping sessions. • Consult your lactation consultant/breastfeeding specialist if expression still does not occur.

If you have not resolved the problem with your breast pump or you have further questions, please contact the Medela Customer Service at 800-435-8316.

10. Warranty

This product is warranted by Medela to the original retail purchaser to be free from defects in material and workmanship for the period of 90 days from the date of purchase. Warranty can only be claimed in the country of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement, parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE.

For questions regarding this warranty or instructions on making a warranty claim, please call Medela Customer Service (toll free) at 1-800-435-8316. All returns must be sent with a Return Authorization Number from Medela, with your dated bill of sale or other proof of purchase and a brief statement of the problem to the following address:

Medela LLC – Returns, Door 4501
1101 Corporate Dr.
McHenry, IL 60050
ATTENTION: RETURNS

Medela Canada
4160 Sladeview Crescent Unit #8
Mississauga, ON L5L 0A1
ATTENTION: RETURNS
info@medela.ca

11. Disposal

Properly dispose of the device in accordance with local, state, and federal regulations.

Breast shields, connector and accessories

The parts are made of plastics that are not harmful to the environment when disposed of as household waste. Recycle or dispose of according to local regulations.

12. Meaning of symbols

Symbols used in these instructions

	General safety alert symbol, points to information related to safety.		Read and follow the instructions for use.
---	---	---	---

Symbols found on the product parts and its packaging.

	Identifies the manufacturer.		Indicates the date of manufacturing.
	Indicates the part number of the device.		Indicates the lot/batch number.
	Read and follow the instructions for use.		The packaging contains products intended to come in contact with food according to regulation EC 1935/2004.
	Keep away from rain. Keep in dry conditions.		Indicates that the material is part of a recovery/recycling process.
	Keep away from sunlight.		Indicates that the package is made of cardboard.
	Contains fragile goods. Handle with care.		

13. Technical specifications

Materials touching skin or coming in contact with milk

- Breast shield: Polypropylene, thermoplastic elastomer
- Connector, valve head: Polypropylene
- Diaphragm, valve membrane: Silicone rubber
- Bottle: Polypropylene
- Lid: Polypropylene

All parts that come in contact with breast milk are not made with BPA (Bisphenol A).

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Medela wordmark and logo, Harmony, The Moms' Room, and 2-Phase Expression are registered in the U.S. Patent and Trademark Office. Mother's Milk, Everyday Amazing, PersonalFit Flex, Quick Clean, Micro-Steam, and Maximum Comfort Vacuum are trademarks of Medela.
Medela Canada Inc.