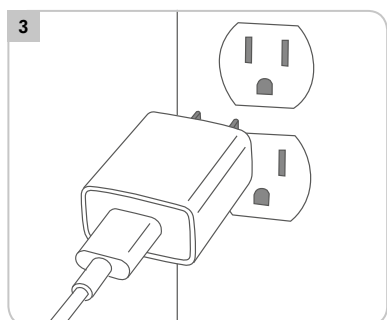
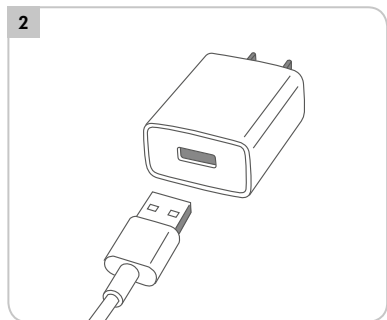
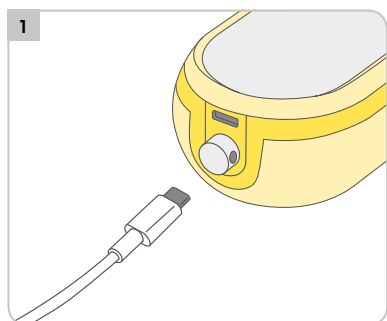
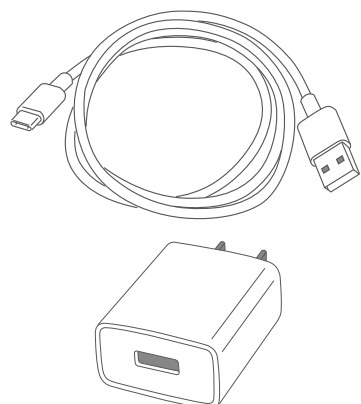
Instructions for use Freestyle Flex power adaptor

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ENGLISH

⚠ **Please read all instructions before using this product. Keep these instructions for future reference. For further information please read the instructions for use of the Freestyle Flex breast pump.**

Product description/Intended purpose

The Freestyle Flex power adaptor is a switched power supply intended to connect the Medela Freestyle Flex breast pump to the mains for powering and battery charging.

Use the power adaptor only for the Freestyle Flex breast pump sold in the US/Canada/Mexico markets.

Important safety information

⚠ WARNING

- Never operate an electrical device if it has a defective cord or plug, if it is not working properly, if it has been damaged or dropped into water. If damage is found, immediately discontinue use of power adaptor and call Medela Customer Service.
- Do not use an electrical product that has been exposed to water or other liquids.

⚠ CAUTION

- Make sure the voltage of the power adaptor is compatible with the power source.
- Always unplug electrical products immediately after use.
- Use only with the original cable that comes with the power adaptor. An inadequate cable may result in a fire hazard or malfunction of the equipment.
- A defective power adaptor or cable cannot be repaired. Disable the damaged power adaptor in an obvious way. Dispose of the power adaptor according to local regulations.
- The power adaptor is not heat-resistant: Keep away from heated surfaces or open flames.
- Allow for sufficient air ventilation around the power adaptor while it is operating otherwise overheating may occur.
- Do not use the power adaptor near flammable materials.
- Close supervision is necessary when the power adaptor is used in the vicinity of children.

How to use

1. Connect the power cable to the motor unit.
2. Plug the power cable into the power adaptor.
3. Plug the power adaptor into a wall outlet.

Specifications

Model: S010WU0500200
Sales article No.: 101038442
Input voltage range: 100 to 240 V~, 50/60 Hz
Max. input current: 400 mA
Rated output voltage: 5 V=
Max. output current: 2 A

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Ambient Conditions

Operating temperature: 0 to 40 °C
Operating relative humidity: 10 to 90 %
Storage temperature: -30 to 70 °C
Storage relative humidity: 10 to 90 %

Disposal

Do not dispose of electric/electronic devices together with unsorted municipal waste (dispose of the device in accordance with local regulations).

Meaning of symbols

- ⚠ General safety alert symbol, points to information related to safety.
- ~ Indicates alternating current.
- = Indicates direct current.
- ⚡ Indicates the polarity of the USB power output.
- ⊞ Indicates that the device is a Class II electrical appliance (double insulated).
- 🏠 Indicates that the device is for indoor use only.
- VI Indicates the efficiency level for external power supplies.
- UL LISTED The UL LISTED mark indicates that the product is manufactured in compliance with UL safety requirements for USA and Canada.
- FC Indicates the compliance with the requirements of the Federal Communications Commission.
- NOM The NOM (Norma Oficial Mexicana) mark indicates that the product is found to be in compliance with the applicable Mexican requirements.
- 📖 Read and follow the instructions for use.
- 🌡️ Defines the temperature range for transport or storage.
- 🌫️ Defines the relative humidity range for transport or storage.

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Medela AG
Medela LLC
Medela Canada Inc.