Clario

SUCTION PUMP

- EN Instructions for use
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- FI Käyttöohjeet
- NO Bruksanvisning
- ES Instrucciones de uso
- PT Instruções de utilização

Table of contents

Congratulations	4
1 Warnings and safety instructions	5
Safety instructions	7
2 Power supply and battery operation	8
Battery low (only AC/DC model)	8
Battery empty (only AC/DC model)	8
3 Description	9
Introduction	9
Intended use/indications	9
Contraindications	9
Intended user	9
Intended patient population	9
Important note	9
4 Overview	10
Definition of vacuum	10
Versions of the pump	10
Main elements of the pump	10
Position of the membrane	11
Valve discs in the SafetyChamber	11
Operating elements and battery status	11
5 Installation	12
Check initial delivery	12
6 Preparation for use	13
Checks before use	13
Assembly of the collection system	13
Assembly of the SafetyChamber	14
Attaching the collection system to the pump	14
7 Operating instructions	15
Connect Clario to mains power	15
Functional check with water	16
Alternative: functional check with the optional vacuum gauge	17
Changing vacuum level	18
Placing out of operation after use	18
8 Emptying the canister	19
9 Replacing the valve discs	20

10	Replacing the rechargeable battery	21
11	Battery data and test	22
	Duration of pumping.....	22
	Charging time (charged up to 100%).....	22
	Battery status.....	22
	Testing the rechargeable battery.....	22
12	Battery calibration	23
	Calibrating NiMH rechargeable battery.....	23
	Storage of the rechargeable battery.....	23
13	Troubleshooting	24
	Motor not running.....	24
	Pump is too loud.....	24
	Insufficient vacuum.....	24
	Running time from battery insufficient.....	24
	No LED lit (AC/DC version only).....	24
	LED lit up (AC/DC version only).....	24
	LED blinks slowly (AC/DC version only).....	24
	LED blinks rapidly (AC/DC version only).....	24
14	Warranty and servicing	25
	Warranty.....	25
	Servicing.....	25
15	Disposal	25
16	Cleaning guidelines	26
	General notes.....	26
	Medela recommended surface cleaning agents for pump housing.....	26
	Water.....	26
	Cleaning/disinfection machines.....	26
	Disposable products.....	26
17	Accessories overview	28
18	Accessories legend	29
19	Signs and symbols	30
20	Technical specifications	32
21	Technical documentation	33
	Technical documentation/EMC.....	326

Congratulations

By choosing the Clario, you have acquired a high-quality suction pump which is adaptable to your needs on order of a health professional/ physician only. This product provides you with a unique threefold hygienic design and optimal protection from an overflow. It is operated by a membrane system with an electric motor. Using the 3 vacuum settings, the suction strength can be adjusted to the required level. As all Medela suction pumps, this pump provides reliable and simple suction. Its simple handling and cleaning as well as the safety features are additional advantages you receive. Contact us – we will be pleased to advise you.

1 Warnings and safety instructions

WARNINGS

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTIONS

Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury.

Safety related tip

Indicating useful information about the safe use of the device.

The Clario is approved exclusively for the use as described in these instructions for use. Medela can only warranty the safe functioning of the system when the Clario is used in combination with the original Medela accessories. See chapters 17/18.

Please read and observe these warning and safety instructions before operation. These instructions for use must be kept with the device for later reference.

Please note that these instructions for use are a general guide for the use of the product. Medical matters must be addressed by a physician. Medela considers herself only responsible for the effect on BASIC SAFETY, reliability and performance of the Clario if it is used in accordance with the instructions for use.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Subject to change.

WARNINGS

- For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators.
- For patients for whom a breakdown of the device can lead to a critical situation (e.g. patients with acute dyspnoea or severe catarrhal congestion), a replacement device must always be available.
- The device must not be used for suctioning explosive, easily flammable or corrosive liquids.
- The connecting tubing supplied with the device must never come into direct contact with the suction area. A sterile suction catheter must always be used (risk of infection).
- Before cleaning the device, pull the plug out of the fixed mains socket.
- No modification of this equipment is allowed.
- Do not connect this device to a passive drainage tube.
- Consult the indications for use and consider risk factors and contraindications before using the Clario. Failure to read and follow all instructions in this manual prior to use may result in serious or fatal injury of the patient.
- Do not use Clario in MRI environment (Magnetic Resonance Imaging).

CAUTIONS

- Incorrect use can cause pain and injury to the patient.
- Do not use sterile accessories when the sterile packaging is damaged.
- Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the Clario and should be kept at least a distance 1ft (30 cm) away from the equipment.
- Portable and mobile RF communications equipment can affect medical devices.

Safety related tip

- For safety tests, it is assumed that the device is serviced and repaired throughout its service life in accordance with the service manual. Because the Clario is a device of safety class II (EN IEC 60601-1, A1, A2:1995), the safety tests are confined to visual inspection of the housing and power cord. This test must be carried out before each use.
- Testing the patient leakage current: Please see service manual for details.
- The protection of the Clario against the effects of the discharge of a cardiac defibrillator is dependent upon the use of appropriate cables.
- Separation from the mains is only assured through the disconnection of the mains plug and the fixed mains socket.
- Before you plug in the device, please check that your local power supply is the same as the voltage given on the specification plate.

Safety instructions

- The Medela Clario is a medical device that requires special safety measures in regard to EMC. It must be installed and put into operation in accordance with the attached EMC information.
- In each of the following cases, the device must not be used and it must be repaired by Customer Services:
 - if the power cord or the plug are damaged
 - if the device is not functioning perfectly
 - if the device is damaged
 - if the device shows clear safety defects.
- Keep the power supply cord away from hot surfaces.
- The mains plug and the on/off switch must not come into contact with moisture.
- Never pull the mains plug out of the fixed mains socket by pulling on the power supply cord!
- Never leave the device unattended when it is switched on.
- The pump must stand upright during use.
- Never place the power supply cord around your neck.
- Never use the device at high room temperatures, while bathing or showering, if you are very tired or in an environment where there is a risk of explosion.
- Never place the device in water or other liquids.
- When using single use, sterile products, please note that they are not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.
- Contact your local Medela customer service representative for assistance with product operations.
- Wear gloves for all operations.

These instructions for use must be kept for later reference.

2 Power supply and battery operation

The Clario is a mains- and battery powered suction pump. Before you plug in the device, please check that your local power supply is the same as the voltage given on the specification plate. Used as described in this user guide, the Clario AC/DC model has a maximum of 50 minutes of battery time before it is necessary to recharge. If the Clario AC/DC model is not in use, the battery must be calibrated every 60–90 days.

Battery low (only AC/DC version)

During battery operation a slow blinking of the LED indicator indicates, that the rechargeable battery is soon discharged. The function of the Clario is maintained for a short time (~10 minutes), but the battery must be recharged as soon as possible.

Battery empty (only AC/DC version)

As soon as the rechargeable battery is empty, the pump comes to a stop. The proper function of the pump is no longer maintained. The rechargeable battery must be recharged or replaced.

3 Description

Introduction

The Clario is a high-quality suction pump. It is powered by the well-proven membrane system. It ideally combines easy handling and cleaning with safety features to ensure optimal operation. Use only according to the instructions from your doctor and after getting instructions from trained medical specialists.

Intended use/indications

The Clario is intended for use in the medical field for suctioning secretions, body liquids and foreign objects in the nasal, pharyngeal and tracheal areas. This suction pump is suitable for use at home, in the hospital, in the car (AC/DC version) or outdoors (AC/DC version).

Contraindications

The device is not suitable for use for prolonged suctioning.

Intended user

The Clario should only be operated by properly instructed users. These persons must not be hard of hearing or deaf and must have adequate visual faculty. The instruction has to be refreshed at least once a year.

Intended patient population

The Clario is intended to be used on patients only exhibiting conditions as described in the indications for use.

Important note

Compliance with proper surgical procedures and techniques is the responsibility of the physician. Each physician must evaluate the appropriateness of the treatment based on his own knowledge and experience.

4 Overview

Definition of vacuum

By the application of medical aspiration devices, vacuum is normally given as the difference (in absolute figures) between absolute pressure and atmospheric pressure or as negative values in Kilopascal (kPa). In this document, the indication of -10 kPa for example always refers to a pressure range in kPa below atmospheric ambient pressure (according to EN ISO 10079:1999).

Versions of the pump

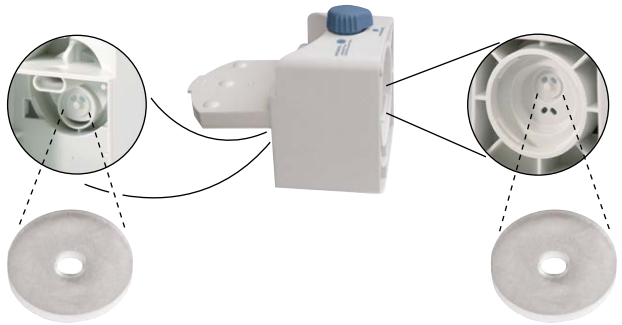
Main elements of the pump

Position of the membrane

Valve discs in the SafetyChamber

Operating elements and battery status
(only AC/DC version)

Vacuum regulator

On/off button

Rechargeable battery LED
(only AC/DC version)

- not lit** Battery is fully charged
- lit up**
 - Battery is being charged
 - Pump is plugged in
- blinks slowly** Battery is soon empty
- blinks rapidly** Battery is defective

5 Installation

Check initial delivery

Check the delivery package of the Clario for completeness and general condition.

Clario versions

REF	Clario AC	014.00XX
	Clario AC/DC	014.01XX
	Clario Deluxe	014.02XX

Disposable fingertip tubing

REF	074.0003
------------	----------

Power adapter

REF	014.0004
------------	----------

Power cord

099.0011 (Euro)
014.0007 (UK)
014.0008 (AUS/NZ)
014.0009 (USA)

Instructions for use

REF	177.1405
------------	----------

WARNINGS

The connecting tubing supplied with the device must never come into direct contact with the suction area. A sterile suction catheter must always be used (risk of infection).

6 Preparation for use

WARNINGS

For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Wear gloves for all operations.

CAUTIONS

- The Clario must remain in an upright position during use.
- Sterile accessories must be checked on the integrity of the packaging before use.
- Non sterile and reusable accessories must be cleaned, disinfected and/or sterilized according to the Medela cleaning guide (product code 200.2391).

6.1 Checks before use

- Check the Clario system before use for damage of the power cord or plug, obvious device damage or safety defects and proper functioning of the device.
- Check for completeness and general condition of the Clario delivery package.
- Only for AC/DC versions: make sure that the rechargeable battery is calibrated according to chapter 12.
- Check all accessories prior to use:
 - Canister and lid for cracks, brittle and flawed spots. Replace if necessary.
 - Tubing for cracks, brittle areas and that connectors are firmly attached. Replace if necessary.

6.2 Assembly of the canister

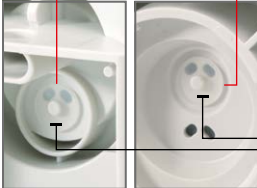
- 1.1 Attach the floater to the canister.

- 2.1 Attach the lid to the canister. Mind the markings on the front of the canister and the lid.

6.3 Assembly of the SafetyChamber

1.  1.1 Open the lid of the SafetyChamber.
2.  2.1 Make sure that both valves (silicone discs) are attached to the SafetyChamber. See chapter 4.
3.  3.1 Make sure that the lid of the SafetyChamber is closed well.
4.  4.1 Hold the SafetyChamber and tilt 45° to the left to attach it to the motor unit.
4.2 Turn the SafetyChamber to an upright position until you hear a click.

6.4 Attaching the canister to the pump

1.  1.1 Attach the canister to the pump. Press firmly.
2.  2.1 Attach the disposable fingertip tubing to the suction port.

7 Operating instructions

CAUTIONS

The Clario is to be set up in such a way, that a separation from the mains supply can be easily managed. Calibrate the rechargeable battery before the first use.

WARNINGS

For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Wear gloves for all operations.

7.1 Connect Clario to mains power

1. Check the pump before use following the instruction in chapter 6.1.

2. **All models: if the pump is connected to a fixed mains socket:**

Plug in the mains plug of the power cord to a fixed mains socket.

or

AC/DC version: if the pump is operated with a 12 VDC power source:

Connect the optional 12 VDC connection cable to the pump and to the 12 VDC power source.

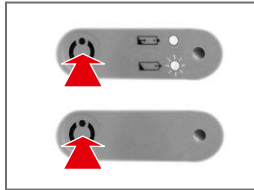
or

AC/DC version: if the pump is operated with the rechargeable battery:

No connection to a power source is needed. Make sure that the battery was charged for five hours.


7.2 Functional check with water

1.


1.1 Press the On/off button to switch on the pump.

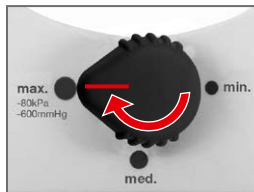2.


2.1 Set vacuum regulator to position «max.».

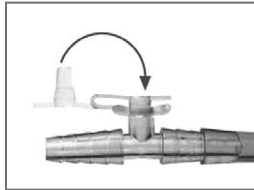3.

3.1 Attach the fingertip tubing.
See chapter 6.4, step 2.1.
3.2 Plug fingertip port.

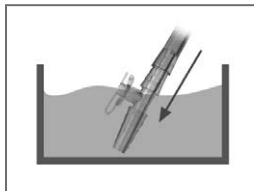4.

4.1 Place fingertip into water and check if there is suction.

5. If there is suction, proceed with step 6. If not, see chapter 13.

6. Use catheter and vacuum setting as prescribed by a physician.


7.3 Alternative: functional check with the optional vacuum gauge

1.


1.1 Attach the optional vacuum gauge to the suction port.

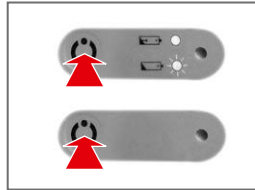2.

2.1 Press the On/off button to switch on the pump.

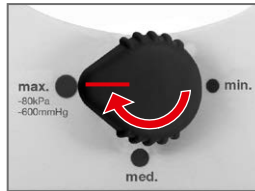3.

3.1 Set vacuum regulator to position «max.».

Altitude above sea level: Max. Vacuum:

+ 2000 m	- 63 kPa - 471 mmHg
+ 1000 m	- 71 kPa - 532 mmHg
+ 500 m	- 76 kPa - 566 mmHg
0 m	- 80 kPa - 600 mmHg

(Tolerance: - 15%)

4.

4.1 Compare the vacuum with the values in the table on the left.

5. If the vacuum is ok, proceed with step 6. If not, see chapter 13.

6. Attach the fingertip tubing.

7. Use catheter and vacuum setting as prescribed by a physician.

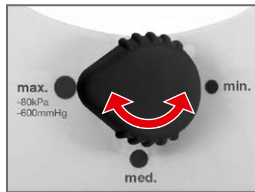
WARNINGS

For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Wear gloves for all operations.

7.4 Changing vacuum level


1.

1.1 Set vacuum regulator to the necessary position.

7.5 Placing out of operation after use

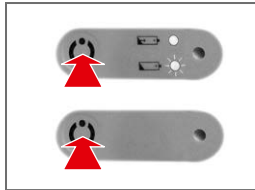1.

1.1 Press the On/off button to switch off the pump.

2. **If the pump is connected to a fixed mains socket:**

Disconnect the mains plug from the fixed mains socket.

or

If the pump is connected to a 12 VDC power source:

Disconnect the 12 VDC plug from the 12 VDC power source.

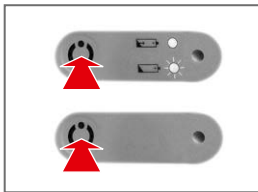
3. Clean and disinfect the Clario. See chapter 16.

8 Emptying the canister

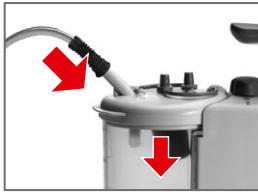
Safety related tip

On a daily basis or latest when the fluid level reaches 550ml and the floater stops suction into the canister the canister should be emptied.

1. 

1.1 Press the On/off button to switch off the pump.
2. 

2.1 Disconnect the disposable fingertip tubing from the suction port.
2.2 Remove canister from the pump.
3. 

3.1 For transportation always seal the canister with the plugs.
4. Empty canister and dispose of fingertip tubing in accordance with local guidelines and inhouse guidelines.
5. 

5.1 Attach a clean canister.
5.2 Attach a new disposable fingertip tubing to the suction port.
6. Switch on the pump and set the vacuum as prescribed by a physician.

9 Replacing the valve discs

1.  1.1 Remove the canister.

2.  2.1 Turn SafetyChamber 45° to the left.
2.2 Remove SafetyChamber by pulling it off the pump.

3.  3.1 Open the SafetyChamber.

4.  4.1 Replace defective valve discs. Make sure that the new valve discs are attached correctly.

5. Attach the SafetyChamber to the pump as described in chapter 6.3.

10 Replacing the rechargeable battery

(AC/DC version only)

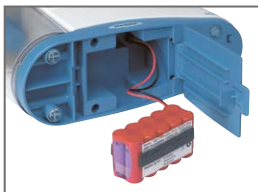
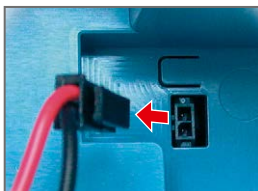
WARNINGS

Before replacing the rechargeable battery, switch off the Clario and pull the mains plug from Clario out of the fixed mains socket.

Before this operation, pull the mains plug out of the fixed mains socket.

1.  1.1 Open the lid of the battery compartment on the bottom of the Clario.
2.  2.1 Detach the battery cable by pulling the plug.
3. Remove the old rechargeable battery.
4. Insert the new rechargeable battery.
5.  5.1 Reattach the battery cable by plugging in the plug.
6. Close the lid of the battery compartment. Make sure that the lid is closed well and that the battery cable is not clamped or pinched.
7. Calibrate and test the rechargeable battery (see chapters 11 / 12).

11 Battery data and test

(AC/DC version only)

CAUTIONS

Only charge the rechargeable battery when the charging indicator blinks slowly.

Duration of pumping

If the battery is fully charged, the operating time at maximum vacuum is approximately > 50 minutes.

Charging time (charged up to 100%)


Approx. 5 hours.

Battery status


not lit	Battery is fully charged
lit up	– Battery is being charged – Pump is plugged in
blinks slowly	Battery is soon empty
blinks rapidly	Battery is defective

Testing the rechargeable battery

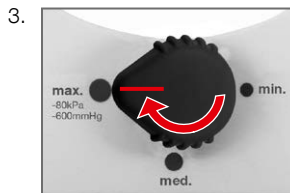1. Make sure that the rechargeable battery is fully charged.
2. Pull out the mains plug from the fixed mains socket.

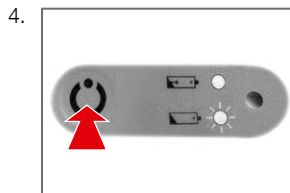3.1 Set vacuum regulator to position «max.».

4.1 Switch on the Clario. The pump will start running.

5. Measure the time until the pump comes to a stop. The battery is ok, when the running time is > 50 minutes.

If the running time is < 50 minutes, repeat test or calibrate the battery (see next page).

12 Battery calibration

(AC/DC version only)

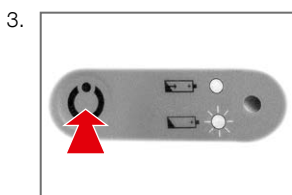
CAUTIONS

Calibrate the rechargeable battery before first use and after storage duration > 2 months. Replace battery if the calibration is not successful.

Calibrating NiMH rechargeable battery

1. Make sure that the rechargeable battery is fully charged.
2. Pull out the mains plug from the fixed mains socket.

3.1 Switch on the Clario. The pump will start running.

4. Let the pump run until the rechargeable battery is empty and the pump comes to a stop.
5. Connect the Clario to a fixed mains socket. The rechargeable battery is being charged.
6. Wait until the charging indicator goes out.
7. Repeat step 1 to 6. The Clario AC/DC is then ready to use.

Storage of the rechargeable battery

In order to keep the self-discharge at a minimum, store the Clario and the removed replacement rechargeable batteries at temperatures below 25°C (77°F). Repeat the calibration process every 60–90 days.

13 Troubleshooting

Motor not running

Check if:

- the Clario is switched on.
- the mains voltage is correct and if the mains plug is inserted correctly into the fixed mains socket.
- the internal battery is charged (only AC/DC version under battery operation).

Pump is too loud

Check the position and seal of the valve discs inside the SafetyChamber.

Insufficient vacuum

Check if:

- the vacuum regulator is set correctly.
- the tubings are not defective or broken. If necessary, replace.
- all plug-in connections are tight.
- the suction canister/lid have no cracks, brittle areas, discolouration. If necessary, replace.
- the seal of the valve discs inside the SafetyChamber is tight.
- the membrane is clean and has no tears.

Running time from battery insufficient

Calibrate the rechargeable battery. See chapter 12.

No LED lit (AC/DC version only)

The rechargeable battery is fully charged.

LED lit up (AC/DC version only)

- The rechargeable battery is being charged.
- Pump is plugged in.

LED blinks slowly (AC/DC version only)

The rechargeable battery is soon empty. The battery power is sufficient for about 10 minutes of pumping.

LED blinks rapidly (AC/DC version only)

The rechargeable battery is defective. For replacing the defective battery see chapter 10.

14 Warranty and servicing

Warranty

Medela AG warrants the device will be free from defects in materials and workmanship for a period of 2 years (6 months for the rechargeable battery) from the date of delivery.

Faulty material will be replaced free of charge during this period if not resulting from abuse or misapplication. This will not apply to parts subject to wear and tear in use. To ensure compliance with this warranty as well as optimum service from Medela products, we recommend the exclusive use of Medela accessories with our appliances.

In no event shall Medela AG be liable for claims which exceed the scope of warranty described including liability for consequential damages, etc. The right to the replacement of faulty parts will not be recognized by Medela if any work has been carried out on the pump by unauthorized persons. This warranty is subject to the appliance being returned to a Medela service centre.

Servicing

Service work is only to be carried out by positions authorised by Medela. The Medela service manual is available upon request.

15 Disposal

The Clario comprises metals and plastics and should be disposed of in accordance with the European directives 2011/65/EU and 2012/19/EU. Additional, local guidelines must also be observed. With the AC/DC versions, the electronic components and the rechargeable battery must be disposed of separately, in accordance with the local regulations.

Please take care that you dispose the Clario and its accessories in accordance with your local disposal guidelines.

User information for the disposal of electrical and electronic equipment

This symbol means that the electrical and electronic equipment must not be disposed as normal household refuse. A correct disposal of this device protects and prevents possible damage to the environment or human health. For more information about the disposal contact the manufacturer, your local caregiver or healthcare provider. This symbol is only valid in the European Union. Please respect the relevant state laws and rules in your country for the disposal of electrical and electronic equipment.

16 Cleaning guidelines

WARNINGS

After each use, the parts that had been in contact with the aspirated secretions are to be cleaned, disinfected, sterilised or disposed of **according to the table on the next page**.

Before cleaning the device, pull the mains plug out of the fixed mains socket.

General notes

- These are general recommendations only that may be adjusted individually, based on the hospital's specific directives and cleaning practices and policies.
- Detailed information according to Medela cleaning instructions (product code 200.2391).
- Wear protective gloves for cleaning/disinfection.
- Dispose of fluids such as blood and secretions and the parts contaminated with them in accordance with internal hospital guidelines.

Medela recommended surface cleaning agents for pump housing

- **CaviWipes**, Metrex Research, www.sybrondental.com
- **mikrozid® AF wipes**, Schülke&Mayr GmbH, www.schuelke.com

Water

Use only the purest quality of water for cleaning. Water hardness is a serious consideration since deposits left on medical products may not be properly decontaminated. Use deionised water in order to reduce this problem. The final rinse water is bacterial free and contains no endotoxins.

Cleaning/disinfection machines

Can be used to disinfect parts from the table on the next page. A hot water rinse (maximum temperature 100 °C) may provide a medium-to high level of disinfection. Every section of the constituent parts must be accessible in order to ensure efficient cleaning. We recommend using a cleaning/disinfecting machine that has been approved by the Robert Koch- Institute and complies with ISO 15883. Recommended temperature for noncritical medical devices (i.e. those that only come into contact with uninjured skin) is 90 °C for 1 minute. The time is increased to 5 minutes for all medical devices that are considered to be critical.

Disposable products

These are single use products not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.


Canister, SafetyChamber	Pump housing Mains cable	
x	x	<p>1. Disassembly Separate all individual parts before cleaning, disinfecting and sterilising.</p>
x		<p>2. Cleaning canister, SafetyChamber Clean components in hot water (60–70 °C) containing a detergent with a pH range between 6.0 and 8.0 only, in order to avoid damaging the instruments and containment devices. Enzymatic detergents help with the removal of organic matter, such as blood. Detergents should be used according to their manufacturer's recommended concentration levels. Some alkaline detergents have been formulated to be safe for reprocessing medical devices. The manufacturers of these detergents should provide information about specific materials that may be damaged by their detergents. Soak all parts thoroughly with warm, soapy water or in enzymatic detergent for 1–5 minutes.</p> <p>1. Remove visible dirt with a cleaning tool – general purpose cleaning brushes, such as pipe cleaners or non-abrasive lint cloths. Brushes and pipe cleaners should fit snugly but still be able to be moved around easily within the area to be cleaned. Rinse thoroughly in clear water. Allow to dry</p> <p>2. Check the instruments for visible dirt and repeat these steps if necessary.</p>
	x	<p>3. Cleaning pump housing, mains cable Wipe with detergent mentioned above. Use clean cloth to dry.</p>
x		<p>4. Disinfect Soak at room temperature for 30 minutes in a disinfection/cleaning solution (A). After the reaction time, use water to rinse the solution residue from the individual parts and rinse the tubing. Rinse for at least 2 minutes with the cleaning solution (B).</p> <p>(A) fluid aldehyde-free disinfection solution with cleaning effect for use as a soaking bath, bactericide, fungicide, limited virucide, with good material compatibility for stainless steel, non-ferrous material and plastics including silicone, slightly alkaline. e.g. 2% neodisher® Septo MEDsolution</p> <p>(B) Fluid, pH-neutral, enzymatic cleaning solution for the treatment of instruments by machine or manually with very good material compatibility for stainless steel, non-ferrous material and plastics including silicone.</p> <p>or use cleaning/desinfecting machine. or use a dishwasher.</p>
x		<p>5. Storing components After disinfection store the components on a clean and dry cloth.</p>

17 Accessories overview

WARNINGS

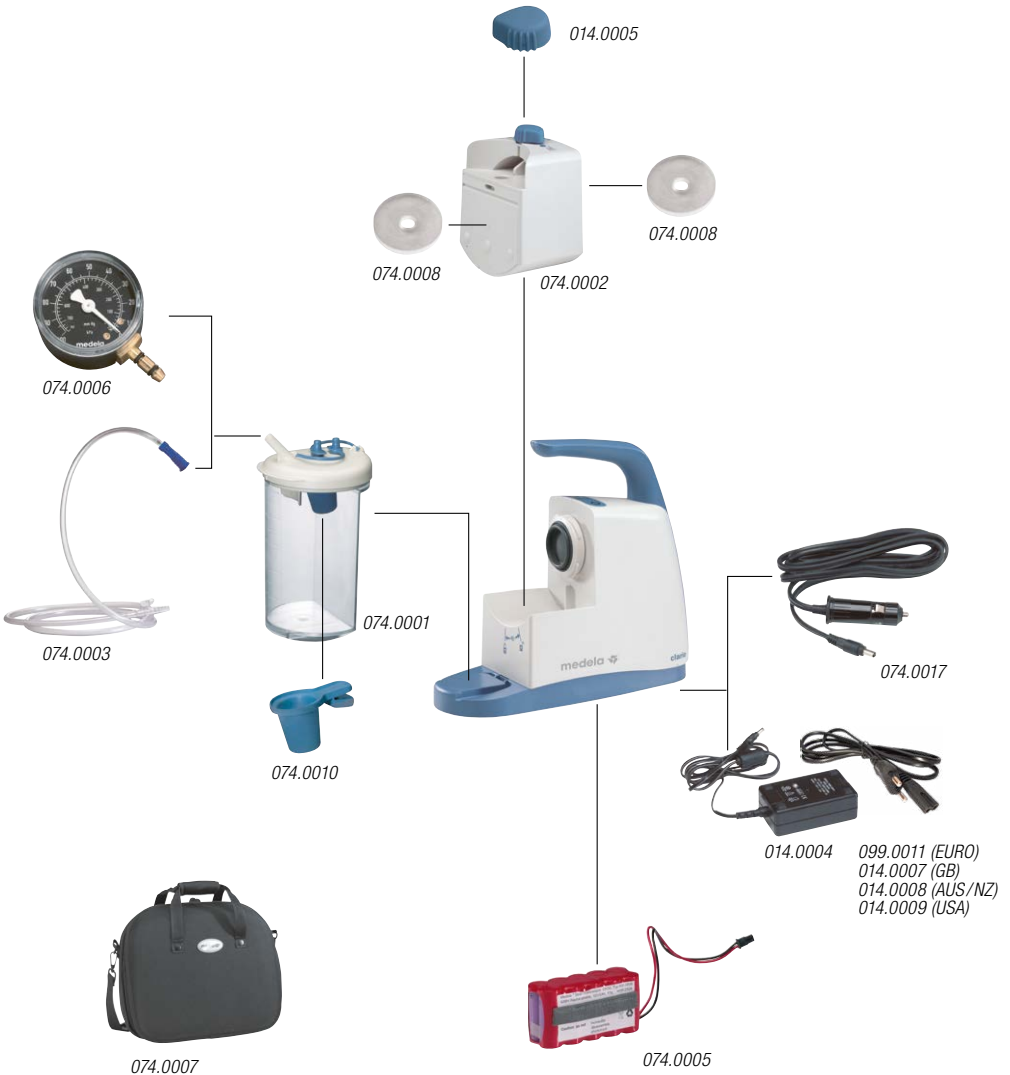Clario was verified in combination with the accessories listed below. For a correct and safe operation use Clario with these accessories only. Further information is supplied on the instruction sheet of the individual accessory.

18 Accessories legend

Safety related tip

If the pump is used together with non-Medela applied parts, they must:

- have CE mark and (if necessary) local registration
- be able to be attached to Medela accessories safely without impacting the performance of the pump.

Note: when combining Medela parts and non-Medela parts, you take over the responsibility of the entire system.

REF

074.0002	SafetyChamber
014.0005	Vacuum regulator
074.0008	Valve discs (10 pcs.)
074.0001	Canister set
074.0010	Floater (10 pcs.)
074.0003	Disposable fingertip tubing
074.0005	Rechargeable battery
014.0004	Power adapter
099.0011	Power cord (EURO)
014.0009	Power cord (USA)
014.0007	Power cord (GB)
014.0008	Power cord (AUS/NZ)
074.0007	Carrying bag
074.0017	Car connection cable for 12V DC
074.0006	Vacuum gauge

19 Signs and symbols

This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

This symbol indicates the class of the pump.

This symbol indicates a class II device.

IP21

This symbol indicates the protection against harmful effects due to ingress of solid foreign objects and against harmful effects due to the ingress of water.

This symbol indicates that the device should not be used after the end of the year and month shown.

This symbol indicates a single use device. Do not reuse the device.

This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.

REF

This symbol indicates manufacturer's catalogue number.

This symbol indicates the legal specifications of the pump.

This Symbol indicates a type BF applied part.

SN

This symbol indicates manufacturer's serial number.

This symbol indicates to follow instructions for use.

This symbol indicates MR UNSAFE.

LOT

This symbol indicates manufacturer's batch code.

This symbol indicates that interferences may occur in the vicinity of equipment marked with this symbol.

STERILEEO

This symbol indicates the device is sterilized using ethylene oxide.

This symbol indicates to consult instructions for use.

This symbol indicates a CAUTION or WARNING associated with the device.

This symbol indicates the manufacturer.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates safety related tip.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

pcs This symbol indicates number of items.

This symbol indicates do not use the device if package is damaged.

This symbol indicates the maximum vacuum level of the pump.

This symbol indicates the number of items n that the content is sufficient for.

This symbol indicates the flow levels of the pump.

This symbol indicates the electrical specifications of the pump.

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates alternating current.

This symbol indicates direct current.

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates the weight of the pump.

This symbol indicates a carton package.

This symbol indicates the dimensions (h x w x d) of the pump.

This symbol indicates to keep the device away from sunlight.

This Symbol indicates a Prescription Device. CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician (for US only).

This symbol indicates to handle the fragile device with care.

20 Technical specifications

min. vacuum, - 18 kPa / - 135 mmHg (+/- 15 %)
 med. vacuum, - 36 kPa / - 270 mmHg (+/- 15 %)
 max. vacuum, - 80 kPa / - 600 mmHg (- 15 %)

Calculated at 0m, atmospheric pressure: 1013 hPa
 Please note: vacuum levels may vary depending on location (meters above sea level, atmospheric pressure and temperature).

low flow: 15 l/min. (+/- 10 %)

AC 1.7 kg
 AC/DC 2.0 kg


100–240 V, 50–60 Hz
 12 V, 3.4 A, NiMH
 Power adapter

ISO 13485
 93/42 EEC Class IIa

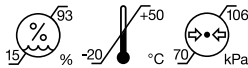
h x w x d
 223 x 255 x 95 mm

Transport/Storage Conditions

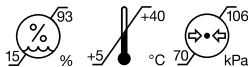Transport/Storage
 Rechargeable battery

Operating Conditions

IP21

21 Technical documentation

WARNINGS

Do not use other accessories than those specified or sold by the manufacturer as replacement parts for internal components as it may result in increased emissions or decreased immunity of the Clario pump.

HF (high-frequency) surgical equipment, radio networks or the like can influence the operation of the device and may not be operated in combination with the Clario.

EMC

Clario is EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 and IEC 60601-1-2:2014 4th Edition according clause 7 and 8.9.

Clario is a medical device that requires special safety precautions and must be installed and placed in operation in accordance with the attached EMC information.

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Electromagnetic Compatibility
 (EMC, IEC60601-1-2:2007 3rd Edition Table 1 and IEC 60601-1-2:2014 4th Edition)

Electromagnetic Emissions

Clario is intended for use in the electromagnetic environment specified below. The customer or the user of Clario should assure that they are used in such environment.

Emission Tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	Clario uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference with nearby electronic equipment.
RF emissions CISPR 11	Class B	Clario is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

WARNINGS

Clario should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, Clario should be observed to verify normal operation in the configuration in which it will be used.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007 3rd Edition, Table 2)

Electromagnetic Immunity

Clario is intended for use in the electromagnetic environment specified below. The customer or the user of Clario should assure that they are used in such environment.

Immunity Tests	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input / output lines	± 2 kV for power supply lines ± 1 kV for input / output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % U_T for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U_T for 1 cycle 70 % U_T for 25 cycles at 50 Hz Single phase: at 0° 0 % U_T for 250 cycles at 50 Hz	0 % U_T for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U_T for 1 cycle 70 % U_T for 25 cycles at 50 Hz Single phase: at 0° 0 % U_T for 250 cycles at 50 Hz	Mains power quality should be that of a typical commercial or hospital environment. If the user of Clario requires continued operation during power mains interruptions, it is recommended that Clario be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels of a typical commercial or hospital environment.

Note: U_T is the a.c. mains voltage prior to application of test level.

Electromagnetic Immunity

Clario is intended for use in the electromagnetic environment specified below. The customer or the user of Clario should assure that Clario is used in such environment.

Immunity Tests	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of Clario, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1.2 \sqrt{P}$ $d = 0.35 \sqrt{P}$ <p>80 MHz to 800 MHz</p> $d = 0.7 \sqrt{P}$ <p>800 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m)</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	10V/m 80 MHz to 2.5 GHz	10V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which Clario are used exceeds the applicable RF compliance level above, Clario should be observed to verify normal operation. If abnormal operation is observed, additional measures may be necessary, such as reorienting or relocating Clario.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

Recommended separation distance between portable and mobile RF communications equipment and Clario

Clario is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of Clario can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and Clario as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 0.35 \sqrt{P}$	800 MHz to 2.5 GHz $d = 0.7 \sqrt{P}$
W			
0.01	0.12	0.035	0.07
0.1	0.38	0.11	0.22
1	1.2	0.35	0.7
10	3.8	1.11	2.21
100	12	3.5	7.0

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Medela UK Ltd.
Medela LLC