EN

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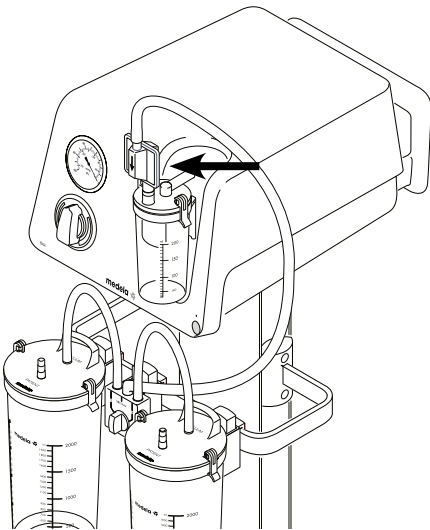
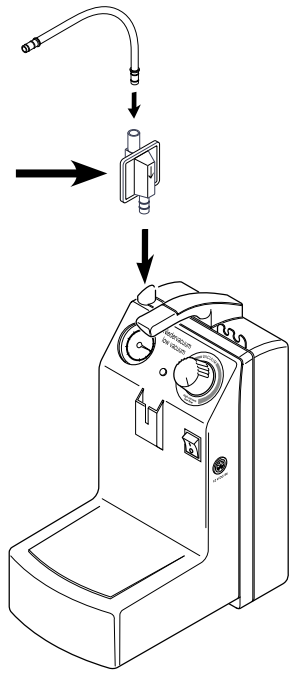
## Test filter

Filter open!

1. Select maximum vacuum
2. Switch on pump
3. Read/compare the vacuum

Olympus  
KV-6  
high  
vacuum

Vario 8 low vacuum	Vario 18 medium vacuum	Vario 18 high vacuum	Basic 30 high vacuum	Dominant 50 – high vacuum	Basic/ Dominant Flex – high vacuum
-3kPa	-10kPa	-10kPa	-15kPa	-20kPa	-20kPa

If the above mentioned values are exceeded, the filter must be replaced.

## Filter exchange

- In case of overflow or clogging, or
- weekly with single patient use, or
- daily with multiple patient use.

To detect a clogged filter, perform filter test before each patient use.  
Comply with in-house directives on hygiene.

## Filter models

Filters with Overflow Protection

### Bacteria filter with overflow protection

077.0571	Medela connections
077.0572	Conical connections
077.0577	Medela + conical connection

### Virus filter with overflow protection

101035263	Medela connections
101035264	Conical connections

### Bacteria/odour filter with overflow protection

077.0092	Medela connections
101035285	Conical connections

### Virus/odour filter with overflow protection

101035286	Medela connections
101035287	Conical connections

CAUTION: Suctioning will be interrupted when the filter is wet/moist or clogged. Perform filter test before each patient as described above.

Filters

### Bacteria filter

077.0573	Medela connections
077.0574	Conical connections

### Virus filter

101035265	Medela connections
101035284	Conical connections

### Bacteria/odour filter

077.0575	Medela connections
077.0576	Conical connections

### Virus/odour filter

101035288	Medela connections
101035289	Conical connections

### Particle filter

077.0582	Medela connections
----------	--------------------

CAUTION: Suctioning will be interrupted when the filter is clogged. Perform filter test before each patient as described above.

## Technical data

- Filtering capacity (Bacteria filter) 99.99999% for particles larger than 0.3 µm
- Filtering capacity (Virus filter) 99.99984% for particles larger than 0.03 µm
- Length/Width/Depth 100x62x36 mm (filtering pad 50x50 mm)
- Weight ~17–23g
- Material (PP, filtering pad)
- Packaging unit 10 pcs

### WARNING

- This is a single use product not intended to be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.
- When combining Medela parts and a third party interfacing device, the user takes responsibility for the entire system and should test the combination to ensure that the performance and safety are properly maintained.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician. Any serious incident that has occurred in relation to the device should be reported to Medela AG and the relevant Competent Authority.

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Medela UK Ltd.
Medela LLC