ENGLISH
<b>SINGLE USE BIRD CUP</b>
<span></span>

## SINGLE USE BIRD CUP

**Intended Use**

This obstetrical instrument is utilized to assist a clinician in the delivery of an infant.

**Intended User**

Vacuum extraction must only be performed or supervised by trained and experienced clinicians.

Proper surgical procedures and techniques are the responsibility of the medical practitioner.

Each physician must evaluate the appropriateness of the procedure based on his or her medical training and experience. The neonatal staff must be aware of the indications, contra-indications and precautions for assisted delivery and must observe the infant for signs of complications of vacuum assisted delivery.

Pump requirements
<ul style="list-style-type: none"><li>This obstetrical instrument has been developed for use in conjunction with an external vacuum source capable of precise vacuum regulation to ensure proper vacuum levels (up to –80kPa/–600<span> </span>mmHg).</li> <li>This device should be designed for medical purposes and may be either electric or manual.</li> <li>Safety and performance of cups used in combination with Medela electric and manual pumps has been verified.</li></ul>

**Tubing requirements**

The tubing connecting the instrument to the vacuum source is vital for safe and proper function of the instrument. To ensure this only Medela original tubing should be used.

Material
<ul style="list-style-type: none"><li>Cup: Polycarbonate [PC]</li> <li>Bottom plate: Polypropylene (PP)</li> <li>Traction handle: Nylon</li> <li>Locking piece: ABS Plastic</li></ul>

GENERAL INFORMATION

### Safety rules

**Warnings**

To reduce the risk of exposure to biological hazards of mother and infant:

- Sterile product – do not use if:
  - The product has exceeded the expiration date on the packaging.
  - Packaging is opened or damaged.
  - Device is single-use and must not be reused. Re-use could cause cross-contamination.
- Do not prevent risks of injury for mother and infant:
  - Do not use if:
    - The device is damaged.
    - This device shows obvious safety defects.
  - Follow the Pump and Tubing Requirements and the Safety Notes.
- No modification of this equipment is allowed.
- Wul suction is never to be used.
- Device is single-use and must not be reprocessed. Reprocessing could cause loss of mechanical, chemical and/or biological characteristics.

**Safety notes**

- Ensure against intrusion of extraneous tissues between cup and head and ensure proper placement of cup.
- Never apply the cup to any portion of the infant’s face as this can cause fetal trauma.
- Insertion of the cup should be performed carefully to avoid malfunctions, maternal or fetal trauma and maternal discomfort.
- The head, not just the scalp, must descend with each pull.
- Pull only during a contraction.
- Always apply traction at an angle allowing the stem of the cup to remain in-line or on-axis.
- Never apply rocking movements or torque to the device; only steady traction should be used.
- Discontinue vacuum extraction if there is evidence of significant scalp trauma or damage.
- The head must be delivered, or almost completely delivered, with no more than three pulls.
- The head must be completely delivered within 15–20 minutes of first applying the cup.
- Never exceed 10 minutes of cumulative traction time.
- Never exceed maximum recommended vacuum –60 to –80 kPa (–450 to –600 mmHg). During “pop offs” – scalp trauma can occur.
- Discontinue vacuum extraction if the vacuum cup becomes disengaged or pops off twice. Discontinue the procedure if traction does not cause descent of the fetal head.
- Do not pull three times, if the second pull fails to produce good descent.

**Tips**

- Parents should be instructed to check the baby’s health.
- The infant has to be observed for several days for possible problems.

**Disposal**

Safely discard potentially contaminated medical device as healthcare (clinical) waste in accordance with local healthcare guidelines, country and government legislation or policy.

Separation, re-cycling or disposal of packaging material shall follow local country and government legislation on packaging and packaging waste, where applicable.

Any serious incident that has occurred in relation to the device should be reported to Medela AG and the relevant Competent Authority.

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