Apgar Timer

EN	Instructions for use	DE	Gebrauchsanweisung	FR	Mode d'emploi
IT	Istruzioni per l'uso	NL	Gebruiksaanwijzing	SV	Bruksanvisning
DA	Brugsanvisning	FI	Käyttöohjeet	NO	Bruksanvisning
ES	Instrucciones de uso	PT	Instruções de utilização	EL	Οδηγίες χρήσης
BG	Инструкции за употреба	CS	Návod k použití	ET	Kasutusjuhend
HR	Upute za upotrebu	HU	Kezelési utasítás	LT	Naudojimo instrukcijos
LV	Lietošanas instrukcija	PL	Instrukcja obsługi	RO	Instrucțiuni de utilizare
SK	Návod na obsluhu	SL	Navodila za uporabo	SR	Упутство за употребу

Table of Contents

1	Intended use	3
2	Contraindications	3
3	Safety instructions	3
4	Operating elements	4
5	Operation of the Apgar Timer	5
6	Cleaning and Reprocessing	7
7	Disposal	7
8	Warranty	8
9	Technical Data	9
10	Signs and symbols	10
11	Technical Documentation	294

1 Intended use

The Medela Apgar Timer is used during childbirth in the hospital to measure and indicate a length of time at predetermined intervals from the setting time; this being when the baby is born.

The Apgar score is used to assist a healthcare professional to determine the physical health of a newborn infant and its response to birth and life outside the womb at intervals of 1, 5 and 10 minutes.

The total numerical score is 10. In addition, the exact time of birth can be stored for later retrieval.

2 Contraindications

No contraindications are currently known.


3 Safety instructions

CAUTION

Do not touch the patient and the battery contacts simultaneously.

4 Operating elements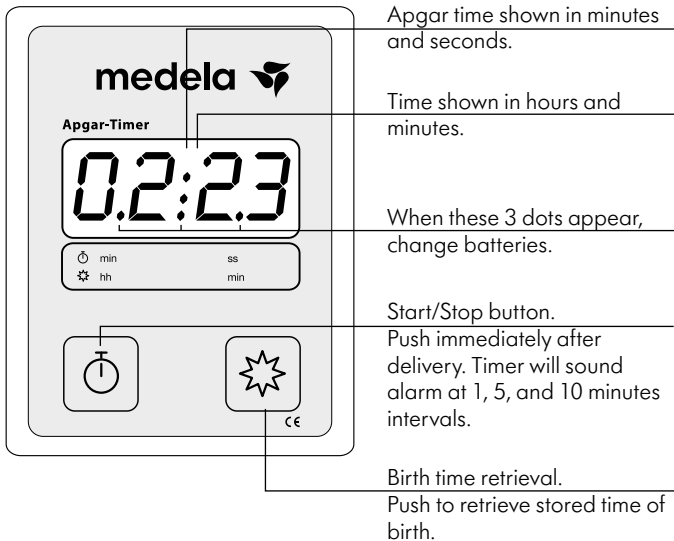

5 Operation of the Apgar Timer

1. Starting the Apgar Timer

Push the Start/Stop button.

2. Switching off the device

The Apgar Timer can be turned off by pushing the Start/Stop button. The Timer will automatically shut off after 1 hour.

3. Storing the birth time

When turning on the Apgar Timer the time of its internal clock is automatically stored as birth time. The birth time is shown in hours and minutes. The birth time will be changed as soon as the next Apgar score is taken.

4. Retrieval of the birth time

The birth time is called up in hours and minutes by pushing the birth time retrieval button.

5. Battery Check

A set of batteries should last for 1000 Apgar score measurements. When the batteries become weak, 3 additional dots will appear on the display.

6. Change of Batteries

The Apgar Timer has been designed so that the internal clock continues to run for approx. 2 minutes when the batteries are changed. This means that when you change the batteries no adjustment of time will be necessary.

To change the batteries, proceed as follows:

1. Remove old batteries.
2. Insert new batteries (two 1,5 volt alkaline batteries) following the polarity markings.

7. Retrieval of the Internal Time

If you wish to check the time of the internal clock in the Apgar Timer proceed as follows:

If the Apgar Timer is running (Retrieval of the internal time is not available for the first ten minutes after the Apgar Timer is switched on.)

– Push the birth retrieval button twice. The Apgar Timer will return automatically to the timer after three seconds.

If the Apgar Timer is not running

– Push the birth retrieval button twice. Push the Start/Stop button to end showing the internal time.

8. Adjusting the time

The time of the internal clock can be set as follows:

1. Start with the Apgar Timer switched off.
2. Push the birth time and Start/Stop button simultaneously.
A black dot will appear behind the first digit.
3. Push the birth time retrieval button until the correct number appears in the first digit.
4. Push the Start/Stop button, the back dot will jump to the second digit. Again push the birth time retrieval button until correct number appears in the second digit.
5. Follow the same steps to correct the third and fourth digit.
6. Leave the time adjusting mode by pushing the Start/Stop button.
This will automatically save any changes done to the internal time.

Note: Please adjust the time as described above once every month.

6 Cleaning and Reprocessing Instructions

Reusable devices and components from Medela are delivered non-sterile and are intended for reuse. Prior to use, clean and reprocess the product by following the procedure in the Cleaning and Reprocessing Instructions below.

The following cleaning process has been validated according to international standards and guidelines.

It is the responsibility of the user/processor to ensure that the cleaning/reprocessing procedure is performed using equipment, materials and personnel which are suitable to ensure the effectiveness of the process. Any deviation from the following instructions should be validated by the user/processor to ensure effectiveness of the process.

Materials and equipment:

- Personal Protective Equipment (PPE: disposable gloves, proper protective gear)
- Lint free nonwoven wipes
- Sterile water (<40 °C, <104 °F)

Notes

- Never use steel brushes or steel wool for cleaning.
- Never place the device in water or other liquids.

Water temperatures higher than 45 °C (113 °F) can cause protein coagulation which may negatively affect the efficacy of the cleaning process.

Initial cleaning: Perform initial cleaning directly after the use of the device (within a maximum of 2 hours). Wipe the device surface to remove all gross debris by using a moistened lint free nonwoven wipe and water (<40 °C, <104 °F).

Thorough manual cleaning: Use a lint free nonwoven wipe moistened with tap flowing water (<40°C, <104°F).

Manual disinfection: For manual disinfection, use only disinfectant wipes with active substances such as alcohol, quaternary ammonium compounds (Quats), hypochlorite, hydrogen peroxide, or glutaraldehyde. For effective and safe disinfection please follow the instructions provided by the disinfectant manufacturer.

Drying: After the manual disinfection, store the product on a clean dry cloth and let it dry. Alternatively, follow the instructions provided by the disinfectant manufacturer.

Inspection: After reprocessing according to the steps described above, the device and its components must be inspected for signs of degradation that may limit the useful life and/or performance of the device, such as the following: visible corrosion, mechanical wear, abrasion, damage, or deformation. Discard the device if any signs of degradation are evident.

Storage: Store the product dry and dust-free.

7 Disposal

The Apgar Timer comprises metals and plastics and should be disposed of in accordance with the European directives 2011/65/EU and 2012/19/EU. Additional, local guidelines must also be observed. The electronic components and the batteries must be disposed of separately, in accordance with the local regulations. Please take care that you dispose the Apgar Timer and its accessories in accordance with your local disposal guidelines.

User information for the disposal of electrical and electronic equipment

This symbol means that the electrical and electronic equipment must not be disposed as normal household refuse. A correct disposal of this device protects and prevents possible damage to the environment or human health. For more information about the disposal contact the manufacturer, your local caregiver or healthcare provider. This symbol is only valid in the European Union. Please respect the relevant state laws and rules in your country for the disposal of electrical and electronic equipment.

8 Warranty

Medela AG warrants the device will be free from defects in materials and workmanship for a period of 2 years from the date of delivery. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage as well as use by unauthorized persons.

9 Technical Data

Batteries:	2 pcs 1,5V AA
Accuracy:	Quartzstability, <100ppm
Signal:	Buzzing signal after 1, 5 and 10 minutes
Display:	4-digit Liquid crystal display (LCD)
Operation:	2 foil buttons
Measurements:	160x 100x35 mm (HxWxD)
Weight:	250g (including batteries)

Operating conditions

Temperature limitation:	+5°C to +40°C
Pressure limitation:	70kPa–106kPa
Humidity limitation:	30–75%

Transport/Storage conditions

Temperature limitation:	–20°C to +50°C
Pressure limitation:	70kPa–106kPa
Humidity limitation:	20–95%

The Apgar Timer fulfils standard IEC 60601-1-2 where electromagnetic compatibility is concerned. While the Apgar Timer is in use, other equipment which also fulfils this standard will not be affected nor will the other equipment affect the operation of the Apgar Timer.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Any serious incident that has occurred in relation to the device should be reported to Medela AG and the relevant Competent Authority.

10 Signs and symbols

Start/Stop button and birth time memory. An alarm will sound after 1, 5 and 10 minutes.

Birth time retrieval button.

This symbol indicates the date of manufacture.

This symbol indicates manufacturer's catalogue number.

This symbol indicates manufacturer's serial number.

This symbol indicates the manufacturer.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates to keep the device dry.

This symbol indicates to follow instructions for use.

This symbol indicates the item is a medical device.

This symbol indicates a CAUTION or WARNING associated with the device.

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11 Technical Documentation

The Apgar Timer has been EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 and IEC 60601-1-2:2014 4th Edition according to clause 7 and 8.9.

Electromagnetic compatibility (EMC, IEC 60601-1-2:2007 3rd Edition and IEC 60601-1-2:2014 4th Edition, Table 1)

Electromagnetic Emissions		
The Medela Apgar Timer is intended for use in the electromagnetic environment specified below. The customer or the user of the Medela Apgar Timer should assure that it is used in such environment.		
Emission Tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Medela Apgar Timer uses RF energy only for their internal function. Therefore, its RF emissions is very low and is not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Medela Apgar Timer is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	not applicable	

CAUTION

Wireless communication equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, and walkie-talkies can interfere with the Apgar Timer and should be kept at a minimum distance of 30 cm away from the Apgar Timer.

Electromagnetic compatibility (EMC, IEC 60601-1-2:2007 3rd Edition, Table 2)

Electromagnetic Immunity			
The Medela Apgar Timer is intended for use in the electromagnetic environment specified below. The customer or the user of the Medela Apgar Timer should assure that they are used in such environment.			
Immunity Tests	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	± 1 kV differential mode ± 2 kV common mode not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	$<5\% U_T$ ($>95\%$ dip in U_T) for 0,5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $<5\% U_T$ ($>95\%$ dip in U_T) for 5 sec	$<5\% U_T$ ($>95\%$ dip in U_T) for 0,5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $<5\% U_T$ ($>95\%$ dip in U_T) for 5 sec not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the electric breast pump Freestyle requires continued operation during power mains interruptions, it is recommended that the electric breast pump Freestyle is powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE U_T is the a.c. mains voltage prior to application of the test level.			

Electromagnetic Immunity

The Medela Apgar Timer is intended for use in the electromagnetic environment specified below. The customer or the user of the Medela Apgar Timer should assure that they are used in such environment.

Immunity Tests	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz 3 V/m 80 MHz to 2.5 GHz	3 Vrms not applicable 3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Medela Apgar Timer, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800 MHz to 2,5 GHz <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m)</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			
<p>^a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Medela Apgar Timer is used exceeds the applicable RF compliance level above, the Medela Apgar Timer should be observed to verify normal operation. If abnormal operation is observed, additional measures may be necessary, such as reorienting or relocating the Medela Apgar Timer.</p>			
<p>^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.</p>			

Recommended separation distance between portable and mobile RF communications equipment and the Medela Apgar Timer

The Medela Apgar Timer are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Medela Apgar Timer can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Medela Apgar Timer as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz $d = 0,35 \sqrt{P}$	800 MHz to 2,5 GHz $d = 0,7 \sqrt{P}$
0,01	not applicable	0,04	0,07
0,1	not applicable	0,11	0,22
1	not applicable	0,35	0,70
10	not applicable	1,11	2,21
100	not applicable	3,50	7,00

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Medela UK Ltd.
Medela LLC