

Bird Suction Cups

Product Information



REUSABLE BIRD CUPS

The Bird Cups are available as three autoclavable stainless steel suction cups in the sizes 40, 50 and 60 mm diameter. The cups of each model are identical, but the mode of traction and tube connectors vary. The mesh inside the cups distributes the vacuum evenly. The locking ring can be used to secure the tubing to the cup.

SINGLE USE BIRD CUPS

The Single Use Bird Cup made of Polycarbonate (PC) consists of three pieces: The cup, a bottom plate mesh and a traction handle. The mesh enables the air between the cup and the child's head to be evacuated and ensure that the cup sucks firmly against the child's head.

Intended use

This obstetrical instrument is utilized to assist a clinician in the delivery of an infant during childbirth. Special Bird cups are made to be used for Anterior and Posterior baby's position.

MATERNAL AND FETAL INDICATIONS

Maternal Indications

- Shortening of second stage labour (e.g. maternal exhaustion, reduced maternal effort ie hypertensives, cardiacs, previous caesarean section, poor progress of 2nd stage, malposition of presenting part, deflected head, asynclitism, inadequate expulsive effort)
- Special Cases (e.g. vacuum caesarean section, delivery of second twin)

Fetal indications

- Fetal distress (eg expedite delivery with abnormal CTG, bradycardia)

Maternal Contraindications

- Unengaged fetal head
- Cephalopelvic disproportion (CPD)
- Cervix not fully dilated
- Uncooperative subject
- Failed forceps
- Cup disengaged twice (or more)
- HIV positive status

Fetal Contraindications

- Prematurity < 36 weeks gestation or fetal weight < 2,500 g
- Abnormal presentation
- Fetal abnormalities
- Bleeding disorders/coagulopathies
- Osteogenesis imperfect
- Excessive caput/moulding
- Fetal scalp damage

Obstetric Product Range – Reusable (250 Reprocessing cycles)**Bird Anterior Cup Ø 40 mm**Consisting of: Chain (without handle), length 11 cm
Material: Stainless steel

077.1041

Bird Anterior Cup Ø 50 mmConsisting of: Chain (without handle), length 11 cm
Material: Stainless steel

077.1042

Bird Anterior Cup Ø 60 mmConsisting of: Chain (without handle), length 11 cm
Material: Stainless steel

077.1043

Bird Posterior Cup Ø 50 mmConsisting of: Chain (without handle), length 11 cm
Material: Stainless steel

077.0063

Steel Handle

Traction handle with hook to Bird Suction Cups

077.1044

Tubings, reusable

Silicone tubing, transparent, 150 cm (for Bird and Silc Cups) Ø 7 x 12 mm

077.0056

Silicone tubing, transparent, 200 cm (for CaesAid Cup)

077.0051

Additional reusable cups

Silc Cup, Ø 50 mm, blue

077.0761

Silc Cup, Ø 60 mm, yellow

077.0762

Silc Cup with trumpet valve, Ø 60 mm

077.0078

CaesAid Cup

077.0080

Additional Single use cups

Single Use Silc Cup, Ø 60 mm (Polymethylmethacrylat)

077.0791

Tubings, disposable

Disposable tubing PVC, 150 cm, CH25, with two connectors (for reusable Bird Cups)

077.0184

Disposable PVC tubing, sterile, 150 cm for Malmström Cup and Silc Cups

077.0170

Accessories

Apgar Timer for standard rail

017.0015

Tubing holder, left side

077.0751

Tubing holder, right side

077.0752

Clampholder, plastic

077.0521

Recommended with Basic suction pump**Controls**

Foot on/off switch

077.0723

Foot vacuum regulator

077.0731

Recommended with Basic suction pump

Basic Suction pump	Item number
Basic suction pump, rack version	071.0001
Trolley	071.0034

Main compatible accessories

Disposable Collection System	Item number
Suction jar PC, 1.5l	077.0082
Disposable suction liner 1.5l	077.0083
Disposable suction liner 1.5l with solidifier	077.0084
Suction jar PC, 2.5l	077.0085
Disposable suction liner 2.5l	077.0086
Disposable suction liner 2.5l with solidifier	077.0087

Filters

	Item number
Disposable virus filter with Medela connection	101035265
Disposable overflow protection/bacteria filter with Medela connection	077.0571
Disposable bacteria filter with Medela connection	077.0573
Disposable bacteria/odour filter with Medela connection	077.0575
Disposable virus filter with overflow protection with Medela connection	101035263

Vacuum Tubing, silicone

	Item number
Silicone tubing Ø 7 x 12 mm, without coupling pieces, 25 m	077.0900
Silicone tubing Ø 7 x 12 mm, without coupling pieces 1 m	077.0901
Silicone tubing Ø 7 x 12 mm, without coupling pieces, 2 m	077.0902
Silicone tubing Ø 7 x 12 mm, without coupling pieces, 5 m	077.0905
Silicone tubing Ø 7 x 12 mm with 1 coupling piece, 60 cm	077.0912
Silicone tubing Ø 7 x 12 mm with 1 coupling piece, 100 cm	077.0913
Silicone tubing Ø 7 x 12 mm with 2 coupling pieces, 30 cm	077.0921
Silicone tubing Ø 7 x 12 mm with 2 coupling pieces, 60 cm	077.0922
Silicone tubing Ø 7 x 12 mm, 25 cm, with 1 coupling piece and 1 coupling piece 90°	077.0931
Silicone tubing green Ø 6.5 x 12.5 mm 180 m with coupling piece	077.0961
Silicone tubing Ø 7 x 12 mm, 60 cm, with 1 coupling piece and 1 coupling piece 90°	077.0185
Silicone tubing Ø 5 x 10 mm, 25 m	077.0970

CE 0123

Medical Vacuum Technology for Healthcare Professionals
Please contact us or your local Medela representative for details.

 Medela AG
Lättichstrasse 4b
6340 Baar
Switzerland
www.medela.com

Local contact

©Medela AG/101047087/2024-08/A